

A photograph of four women dressed as a rock band, performing on a stage. They are all smiling and have their arms raised in the air. The woman on the far left is wearing a white long-sleeved shirt and a bright blue skirt, holding a black electric guitar. The woman next to her is wearing a white dress and a white jacket, holding a white keyboard. The woman in the center is wearing a white dress and a white jacket, holding a white keyboard. The woman on the far right is wearing a white top and a white skirt, holding a black electric guitar. They are all wearing high heels. The background is a stage with lights and a dark floor.

medi

medi
product catalogue
Venous Health



medi. I feel better.





Driving the future in compression therapy

A strong brand. A wide-ranging product range. Convincing concepts.

medi World of Compression: signpost to the future with more than 65 years' competence in the field of compression. Your partner for the highest demands on quality, innovation and design. Let's shape the future together and create positive brand experiences – for you and your clients!

Satisfied customers and new potential

Numerous possible combinations for more successful treatment.

mediven compression garments are available in many colours and variants. So medi offers the right garment for several phlebological indications. The broad assortment of add-ons helps patients enjoy wearing their stockings – for long-term treatment compliance. Join medi in profiting from partner-like goodwill and helpful services.



medi World of Compression

medi provides comprehensive marketing solutions – always keeping our focus on the market, its demands and variables, and the medi vision.

To introduce you to medi World of Compression we created indication related catalogues. This enables you to find the correct product for all the different indications and situations in no time. Please find a short explanation of the different literature below:



Venous Health

mediven round-knitted compression stockings are the cornerstone of the conservative treatment for patients with venous disorders. Discover more about the numerous styles and variants of mediven and circaid products for venous health.

Art. no.: 0097D67



Therapy Concept Lymphology

The medi Therapy Concept Lymphology shows the new approach in the treatment of lymphoedema. It combines the decongestion phase, the transition phase and the maintenance phase and provides the right products for every stage. The circaid and the mediven flat knit products make the treatment of lymphoedema more individual than ever before.

Art. no.: 0097D84



Wound Care

The medi Wound Care therapy chain accompanies the patient during the complete treatment of venous leg ulcer by connecting the acute phase to the prevention of recurrence phase. Discover new potentials with the medi Wound Care Concept.

Art. no.: 0097D68



Clinical Compression

The world of Clinical Compression is more than just thrombosis prophylaxis in the clinical environment. Learn more about the multiple application fields of medi's hospital products and benefit from over 40 years of experience with Clinical Compression.

Art. no.: 0099580





medi

medi produces medical aids for a variety of conditions and life situations. But we don't see ourselves solely as a manufacturer. On the contrary, we handle our own research and development, technology, production, and marketing and sales in-house – always keeping our focus on the market, its demands and variables, and the medi vision.

We have the direct connection

To communicate. To co-operate. To be there for each other. The medi service team respects and values this requirement seriously.

Therefore, our office not only offers consultants, but also a state-of-the art communications system and logistics.

Just-in-time: in consultancy, in service, in ordering and delivery.
We are always here for you.
This is your medi export service team.

Hotline

for consultancy, ordering, problem solutions

Administration

T +49 921 912 -0

F +49 921 912 -781

You can visit our website:	www.medi.de/en
or contact us by email:	export@medi.de
or order by email:	orders@medi.de



6

medi

- 10** More than just a manufacturer
- 11** Quality Certificates
- 12** mediven observational study
- 14** Indication overview – elastic and inelastic compression



18

mediven® medical compression stockings

- 20** Product benefits
- 22** mediven sheer & soft
- 26** mediven elegance
- 30** mediven for men
- 34** mediven comfort
- 38** mediven plus
- 42** mediven active
- 46** mediven forte
- 50** Special models in the round-knitted range
- 60** Gusset variants
- 68** Basic care: Duomed
- 74** mediven harmony



82

circaid®

Adjustable compression

- 86** circaid features
- 88** circaid juxtalite



92

Over the counter products

- 94** medi travel men
- 98** medi travel women
- 102** Commercial products
- 104** medi Donning and doffing aids
- 108** medi skin care
- 112** medi textile care



114

Service

- 117** Measuring

More than just a manufacturer


medi produces medical aids for a variety of conditions and life situations. But we don't see ourselves solely as a manufacturer. On the contrary, we handle our own research and development, technology, production, marketing and sales in-house – always keeping our focus on the market, its demands and variables, and the medi vision.

medi offers products in five business divisions: Phlebology, Hospital, Orthopaedics, Wound Care and Footcare. These products are used and distributed by medical supply stores, hospitals and clinics, doctors' practices and therapists. In other words, specialists who know which users need which aids to improve their quality of life.

What we're proud of and what makes our products so special is that they're free of negative side effects.

Our range of products primarily includes medical compression stockings and adjustable compression for people with vein disorders, lipoedema and lymphoedema, supports and braces for the conservative and post-operative treatment of orthopaedic injuries and disorders, stockings for the prevention of thrombosis, compression garments for use after plastic and cosmetic surgery plus a variety of therapy-supporting products.

We support our partners in the sale and supply of our products and have developed intelligent concepts and services for this purpose.

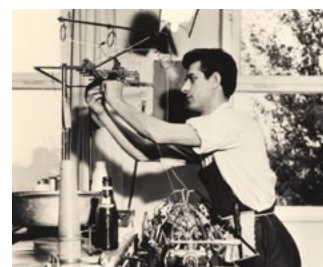


Quality Certificates

As a manufacturer of medical products, medi has a high awareness for quality. Product and system certificates prove that all legal requirements are met.

CE mark

Our medi medical devices meet all of the requirements of the European Directive 93 / 42 / EEC and of the German Medical Devices Act. In accordance with regulations, our products are subject to the conformity assessment procedures and are granted the CE mark. As a kind of “passport for industrial products”, the CE mark guarantees free trade within the European Economic Area (EEA).

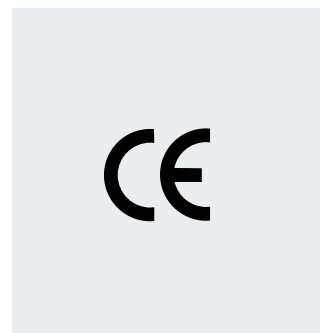

Quality label

Our medical compression hosiery is regularly checked by the Hohenstein Research Institute according to the quality and test provisions of the RAL-GZ 387. As confirmation that our products meet the quality conditions, you will find the quality label printed on the packaging.



Textile trust quality seal

All compression stockings and garments, whether circular-knit or flat-knit, thrombosis prophylaxis stockings and supports are regularly examined and rated according to strict standards by independent test centres for any toxic substances potentially damaging to human health. Granting of the Oeko-Tex Standard 100 certificate guarantees: tested quality for the protection of your health.



DIN EN ISO 9001 : 2000 / DIN EN 13485

medi maintains the quality management system DIN EN 13485. Since 1996, we have demonstrated our ISO 9000 status to our customers with a certificate. From this time, medi has exploited the opportunities offered by this standard to continually improve products from the business units Phlebology, Orthopaedics, Wound Care and Hospital in order to organise safer and more efficient processes and procedures. We define customer satisfaction as a decisive measure of quality.



Hohenstein testing device – HOSY

Pressure is the most important feature of a medical compression stocking.



The mediven® observational study - facts and figures

The study proves: High acceptance of mediven® medical compression stockings^{1,2}

High acceptance

Up to **98 %**
of patients

had no symptoms while wearing the
medical compression stockings

Wearing comfort

More than **70 %**
of patients

considered the medical compression
stockings to be comfortable

Improving symptoms

In up to **53.2 %**
of patients

the medical compression stockings
improved the subjective symptoms

Good compliance

Up to **75.6 %**
of patients

wore the medical compression stockings
on a daily basis

In **17.7 %**
of patients

the objective clinical findings improved
according to the CEAP class*

Up to **68.4 %**
of patients


wore the medical compression stockings
from morning until evening

¹ Schwahn-Schreiber et al. Phlebologie 2016;45(1): 15 – 24.

² Schwahn-Schreiber et al. Phlebologie 2016;45(4):207 – 214.

* CEAP: classification of chronic venous disorders

The gap between the medical care situation and patient-specific criteria

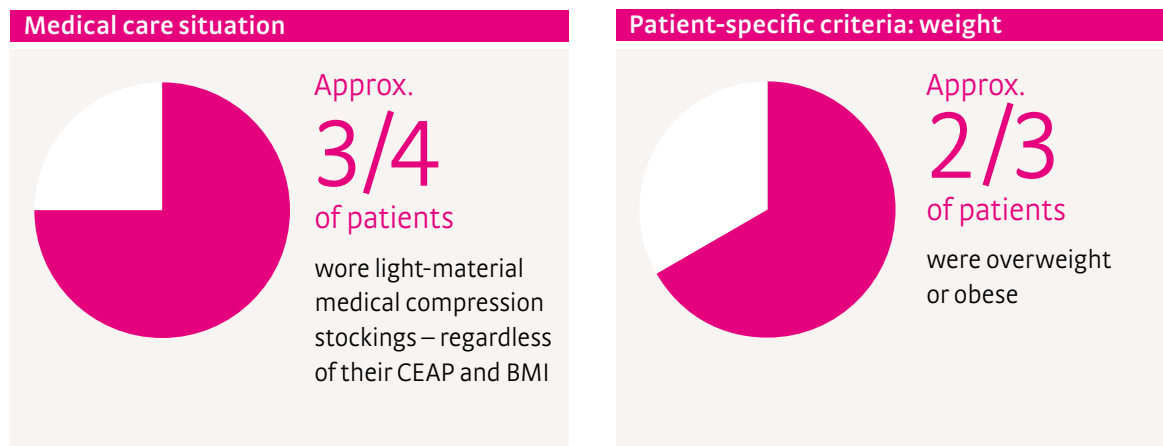

The term “material” encompasses various textile characteristics of a medical compression stocking and is defined by strength, rigidity and elasticity of the knitted fabric.

The mediven observational study has shown patient-specific criteria are rarely taken into account when prescribing medical compression stockings. Although two thirds of the patients were overweight or obese, 73.8 percent of compression stockings prescribed were made from light material. In people with higher weight frequent constrictions occurred due to wearing medical compression stockings of light material.

2 steps to the perfect, individualised mediven compression stocking:








Step 1: Consider the indication / severity of the disease

Step 2: Consider patient-specific criteria

Individual patient care simply with mediven®

Two steps to the perfect, individualised mediven compression stocking!

Step 1: Consider the indication / severity of the disease
(e.g. varicosis, thrombosis, chronic venous insufficiency)

Indication			Product (CCL)	mediven sheer & soft (10 – 15 mmHg, 15 – 20 mmHg)	mediven elegance (1, 2)	
Varicose veins	Primary or secondary varicosis			●	●	
	Varicose veins in pregnancy			●	●	
	Supporting sclerotherapy			●	●	
	After varicose vein surgery			●	●	
Chronic venous disorders according to CEAP	C0s		No visible or palpable signs of venous disease	●	●	
	C1		Telangiectasies or reticular veins	●	●	
	C2		Varicose veins	●	●	
	C3		Venous oedema			
	C4		Changes in skin and subcutaneous tissue C4a Pigmentation or eczema C4b Lipodermatosclerosis or atrophie blanche			
	C5		Healed venous ulcer			
	C6		Active venous ulcer			
Thromboembolism	Thrombophlebitis (superficial) or condition after healed phlebitis			●	●	
	Thrombosis prophylaxis in mobile patients			●	●	
	Deep venous thrombosis, condition after thrombosis, post-thrombotic syndrome					

Note:

- It is important to consider the different materials of medical compression stockings (MCS): light, medium, strong material. The term 'material' includes different textile MCS characteristics (strength, rigidity and elasticity). The use of the three materials depends on the patients' characteristics (e.g. body weight, oedema, severity of the venous disorder, connective tissue): Keep in mind: The higher the body weight and / or the stronger the venous oedema and / or the more severe the venous disorder and / or the softer the connective tissue the firmer the MCS material
- MCS of compression class I (CCL I) might be considered in elderly patients or in patients with an initial treatment, since it is well known, that the donning and doffing process of MCS is easier with CCL I compared to higher compression classes resulting in a better compliance.







	mediven for men (1, 2)	mediven comfort (1, 2)	mediven plus (1, 2, 3)	Duomed (1, 2)	mediven active (1, 2)	mediven forte (2, 3)	circaid juxtalite (20–50 mmHg)
	●	●	●	●	●	●	●
		●	●	●		●	
		●	●	●		●	●
		●	●	●		●	
	●	●	●	●	●	●	
	●	●	●	●	●	●	●
	●	●	●	●	●	●	●
		●	●	●	●	●	●
			●			●	●
							●
	●	●	●	●	●	●	
	●	●	●	●	●	●	
		●	●	●	●	●	


Step 2: Consider patient-specific criteria

Recommendation – mediven® care compass:

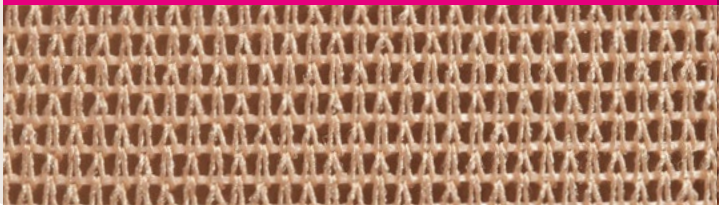- The higher the weight
 - The stronger the oedema (venous oedema)
 - The more severe the disease
 - The softer the connective tissue
- The stronger the stocking material should be.
 → Already meeting a single criterion requires a stronger material.
 → Please note! For lymphoedema or lipoedema, use mediven flat knit.

Case example: Obese, mild disease and light oedema → strong material

<div>Material</div> <div>Criteria</div>	<div>Light (e.g., mediven elegance)</div> 
1. Weight	
2. Oedema (venous oedema)	
3. Severity of the disease	
Recommended mediven® compression stockings:	mediven sheer & soft
	mediven elegance
	mediven for men
	
	
	

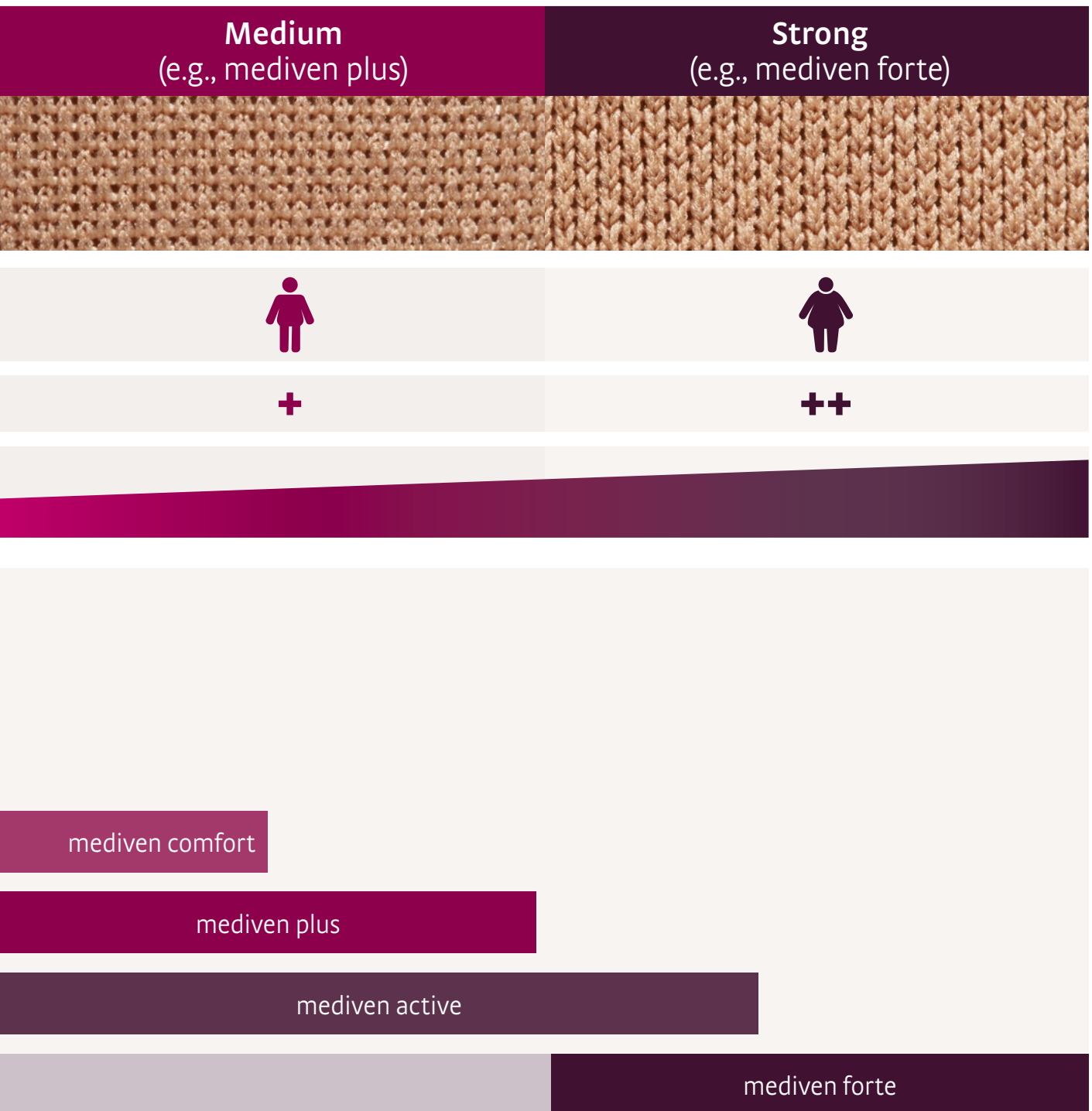

mediven® medical compression stockings

mediven round-knitted compression stockings are the cornerstone of the conservative treatment for patients with venous disorder. They are the basic form of therapy for patients with venous disorders.

In combination with a healthy lifestyle and physical activity, medical compression stockings improve the venous signs and symptoms. Your customers will soon notice that mediven compression stockings are good for them and their well-being, and that they are improving their quality of life.







There are many reasons for deciding for mediven®

These powerful product features speak for themselves.



Soft Elastic

Soft Elastic


The new knitting technique increases wearing comfort with a greater feeling of freedom for the toes and less pressure in the area of the forefoot with a closed toe.



Variety

Special measurements to fit personal needs

Individual made-to-measure garments for severe indications, unusual limb shapes or for special characteristics.

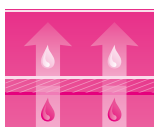

Clima Comfort

Clima Comfort*

Reduces perspiration and prevents the skin from drying out. This is achieved by the patented knit which ensures rapid moisture transportation. This allows temperature moderation throughout the stocking.*



Clima Fresh

Clima Fresh

The technology integrated in the medi high-tech yarn promises you freshness and hygiene all day long. The natural odour neutrality remains permanently.



Perfect Fit

Perfect Fit

Active breathing, elastic materials and a high quality finish combine to ensure patient comfort.

Textile trust

Guaranteed hypo-allergenic and latex-free



All mediven stockings are latex-free.

Care instructions

- Wash your compression stocking daily.
- mediven compression stockings can be washed in washing machines at 40° C on a delicate programme.
- Only use gentle detergents for washing. Do not use fabric softeners as these can block the pores and cause matting of the fibres.
- Never dry compression stockings in the sun or on heaters.
- Most mediven models are suitable for drying in the gentle programme of driers.
- Please do not have compression stockings dry cleaned, do not iron or treat with chlorine.
- Use the mediven laundry net to wash your compression stockings with extra care.

*Clima Comfort

Comfort, scientifically tested by the French Institute IFTH. The test result shows that the Clima Comfort technology from medi guarantees climate exchange between the skin and the stocking. medi achieved an outstanding total score of 11 out of 12 points.



mediven sheer & soft®

medi's most translucent compression stocking.

Suitable for

- women
- normal weight
- no venous oedema
- firm connective tissue

Your benefit

- Concrete sales argument:
Tested consumer preference in the categories sheerness, softness, moisture regulation, durability, and comfort.
For more information see separate brochure 98E72
- Offer your customers a fashionable solution for the treatment of light venous diseases.

Benefits for your customers

- Extremely sheer, not distinguishable from fashion stockings
- Soft and gentle on the skin
- Available with open toe

Additional benefits

- The skin feels comfortable all year round with Clima Comfort
- Clima Fresh – always fresh feet
- Extremely skin-friendly – tested quality to Oeko-Tex Standard 100

Material

light

Compression

15 – 20 mmHg,
20 – 30 mmHg

Product benefits

Sheer stocking for light venous diseases.



Clima Comfort



Clima Fresh




Colours

The article is available in the following colours.
S = Standard



Natural

S

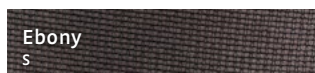

Ebony

S

mediven sheer & soft®

medi's most translucent compression stocking.

Try a major advance in compression therapy. mediven sheer & soft takes comfort to a whole new level. Yet these compression stockings are so sheer that only you will know that they are medical compression stockings.

Indications*

- Varicose veins
 - Primary or secondary varicosis
 - Varicose veins in pregnancy
 - Supporting sclerotherapy
 - After varicose vein surgery
- Chronic venous disorders: C0s to C2 according to CEAP
- Thrombosis prophylaxis in mobile patients
- Thrombophlebitis (superficial) or condition after healed phlebitis

* The choice of the optimum compression sock depends on the indication and the patient's individual needs.



Attractive topband









Knee length stocking (AD)



Panty (AT)



Open toe – detail

mediven sheer & soft				
Styles	AD	AD pte.	AG	AG pte.
				
	calf	calf petite	thigh	thigh petite
15 – 20 mmHg	O140x11-7	O147x11-7	O161x11-7	O151x11-7
20 – 30 mmHg	O240x11-7	O247x11-7	O261x11-7	O251x11-7
Styles	AT	AT pte.		
				
	panty	panty petite		
15 – 20 mmHg	O110x11-7	O117x11-7		
20 – 30 mmHg	O210x11-7	O217x11-7		

See the last page for the article numbers


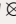
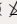
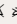
Standard styles

Shoe size 35 – 42

Material composition

Elastane (EL)	Polyamide (PA)
44 %	56 %

Washing instructions

Wash-and-wear    
Suitable for washing machine and drier

Measuring mediven sheer & soft

Size	Circumference in cm						
	I	II	III	IV	V	VI	VII
cG	42 – 57	45 – 61,5	49 – 66,5	52 – 73	56 – 79	60 – 84,5	64 – 87,5
cD	26 – 33,5	28 – 37	30 – 40	32 – 43	33,5 – 46,5	35,5 – 49,5	37,5 – 52,5
cC	28 – 35,5	30 – 39	32,5 – 42	35 – 45	37 – 48,5	39,5 – 51,5	41,5 – 54,5
cB	18 – 20	20,5 – 22,5	23 – 25	25,5 – 27,5	28 – 30	30,5 – 33	33,5 – 36
Length							
Calf stocking / panty petite:	34 – 38 cm			Thigh stocking / panty petite:			62 – 71 cm
Calf stocking / panty standard:	39 – 44 cm			Thigh stocking / panty standard:			72 – 83 cm



mediven elegance®

All you see is beauty. Effective compression has never looked so good.

Suitable for

- women
- normal weight
- no venous oedema
- firm connective tissue

Your benefits

- Win over new customers with the combination of fashion and medical efficacy
- Enhance your image as a fashion adviser in compression therapy

Benefits for your customers

- Elastic knitting yarn ensures a lower sense of pressure with the full medical compression effect
- Perfect fit and outstanding wearing comfort (1st place in the ECOM* study)
- The very latest choice of colours for the fashion conscious
- With medi compression, your customers are certain to feel comfortable in their compression garments
- Attractive topband motifs for thigh-length stockings

Additional benefits

- Dermatologically tested – suitable for sensitive / atopic skin
- Most popular compression garment of all products tested in a study with end consumers**
- Soft Elastic guarantees more freedom for the toes
- The skin feels comfortable all year round with Clima Comfort (tested by IFTH ***)
- Clima Fresh – always fresh feet
- Extremely skin-friendly – tested quality to Oeko-Tex Standard 100

* cf. Elste F. (Deutsches Zentrum für Medizinmarketing, Duale Hochschule Baden-Württemberg), ECOM Studie, Frankfurt am Main, 2012.

** cf. Ben Ammar, F. (DermScan): Clinical evaluation of the skin compatibility of a medical device - application testing realized under dermatological control, Tunis 2012/2013.

*** cf. Institut Français du textile et de l'habillement (IFTH), Clima Comfort Test, Villeneuve-d'Ascq, 2010.

Available with sparkling Swarovski® crystals.*****



*****Swarovski® is a registered trademark of Swarovski AG.

Material

light

Compression

CCL 1, 2

Product benefits

The elegant stocking in many fashionable colours and styles.



cf. Ben Ammar, F. (DermScan): Clinical evaluation of the skin compatibility of a medical device - application testing realized under dermatological control, Tunis 2012/2013.



Clima Comfort



Clima Fresh



Perfect Fit



Soft Elastic



Variety



Colours

The article is available in the following colours.

S = Standard

M = Made-to-measure

Trend colours are available.

Black S+M	Rosé S+M
Anthracite S+M	Bronze S+M
Navy S+M	Beige S+M
Cashmere S+M	medi Magenta**** S+M
Caramel S+M	White**** S+M

****Delivery period up to 10 working days.

mediven elegance[®]

All you see is beauty. Effective compression has never looked so good.


Indications*

- Varicose veins
 - Primary or secondary varicosis
 - Varicose veins in pregnancy
 - Supporting sclerotherapy
 - After varicose vein surgery
- Chronic venous disorders: C0s to C2 according to CEAP
- Thrombosis prophylaxis in mobile patients
- Thrombophlebitis (superficial) or condition after healed phlebitis

* The choice of the optimum compression sock depends on the indication and the patient's individual needs.



Thigh length stocking (AG) with plain silicone topband rose

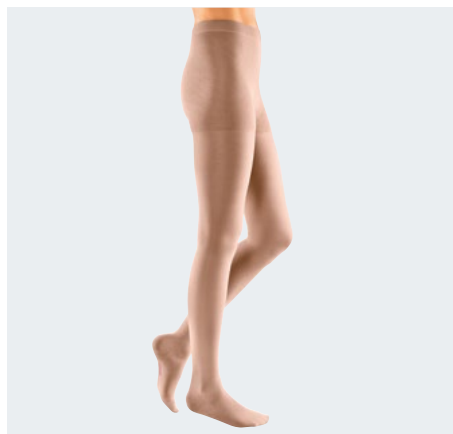











Panty (AT)



Silicone micro dot topband sensitive



Knee length stocking (AD) with soft cuff

mediven elegance					
Styles	AD	AD pte.	AG	AG pte.	AG / FBR
					
	calf narrow silicone topband	calf petite	thigh	thigh petite	thigh with topband
Class 1	0140x11-7	0147x11-7	0160x11-7	0150x11-7	016Hx11-7
Class 2	0240x11-7	0247x11-7	0260x11-7	0250x11-7	026Hx11-7
Styles	AG pte. / FBR	AT	AT pte.	AT / U	AT / U pte.
					
	thigh petite with topband	panty	panty petite	maternity panty	maternity panty petite
Class 1	015Hx11-7	0110x11-7	0117x11-7	0120x11-7	0127x11-7
Class 2	025Hx11-7	0210x11-7	0217x11-7	0220x11-7	0227x11-7

See the last page for the article numbers

Standard styles	
Standard	Sizes I – VII, Length normal and petite Topband normal and wide
Made-to-measure	Available
Special versions	Available, see page 50
Border	AD tricot border, narrow silicone dot topband AF and AG tricot border Silicone dot topband, Silicone dot topband motif, Plain silicone topband rose, Silicone micro dot topband sensitive All topbands are standard styles.
Gusset	AT comfort gusset, AT / U with closed panty top
Foot part	With open or closed toe Universal foot size, shoe sizes 36 – 43

Material composition		
	Elastane (EL)	Polyamide (PA)
CCL 1	34 %	66 %
CCL 2	39 %	61 %

Washing instructions
Wash-and-wear     
Suitable for washing machine and drier



mediven® for men

The elegant compression sock.

Suitable for

- men
- normal weight
- no / light venous oedema
- also for soft connective tissue

Your benefits

- Winner of the iF product design award 2011 offers an additional sales benefit
- Customers are satisfied and will keep coming back thanks to the perfect fit and easy donning and doffing
- Fulfil a broad spectrum of customers' wishes with the classic, modern choice of colours – anthracite, navy, brown and black

Benefits for your customers

- Power and constant compression all day long
- Simply feel great with the perfect fit and the ease of handling due to the anatomically shaped toe with the practical right and left markings
- Bang up to date with the popular 'Business Rib' design and the stylish choice of colours

Additional benefits

- Extremely soft, extra wide comfort cuffs for the best possible feel and fit
- The skin feels comfortable all year round with Clima Comfort (tested by IFTH)
- Clima Fresh over the whole leg for a feeling of freshness around the clock
- Extremely skin-friendly – tested quality to Oeko-Tex Standard 100

Material

light

Compression

CCL 1, 2

Product benefits

The men's sock in fashionable 'Business Rib' design.



cf. Ben Ammar, F. (DermScan): Clinical evaluation of the skin compatibility of a medical device - application testing realized under dermatological control, Tunis 2012/2013.



Clima Comfort



Clima Fresh



Perfect Fit



product
design
award

2011



Colours

The article is available in the following colours.

S = Standard

Black

S

Anthracite

S

Navy

S

Brown

S

mediven® for men

The elegant compression sock.

Indications*

- Primary or secondary varicosis
- Chronic venous disorders: C0s to C3 according to CEAP
- Thrombosis prophylaxis in mobile patients
- Thrombophlebitis (superficial) or condition after healed phlebitis

* The choice of the optimum compression sock depends on the indication and the patient's individual needs.





Knee length stocking (AD)



Extra wide comfort border




Anatomically shaped foot with right and left markings

mediven for men			
Styles		AD	AD pte.
			
		calf	calf petite
Class 1	Article No.	3A40x11-7	3A47x11-7
Class 2	Article No.	3B40x11-7	3B47x11-7

See the last page for the article numbers

Standard styles	
Standard	Sizes I – VII, Length standard and petite
Special versions	Not available
Border	AD tricot border
Foot part	Closed toe Shoe sizes 39 – 46

Material composition		
	Elastane (EL)	Polyamide (PA)
CCL 1	23 %	77 %
CCL 2	29 %	71 %

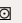Washing instructions	
Wash-and-wear     	
Suitable for washing machine and drier	



mediven® comfort

All you feel is softness. The cuddly side of compression.

Suitable for

- women
- normal weight / overweight*
- light to severe venous oedema*
- firm connective tissue

Your benefits

- Increased customer compliance with the extremely soft knit and particularly easy donning and doffing
- Extremely pleasant wearing comfort as a potent sales argument
- Even more customer satisfaction due to the double-knit comfort sole

Benefits for your customers

- Optimum comfort due to the extremely soft fabric
- Easy handling for the daily donning and doffing
- Contains lanolin** for effective skin care
- Perfect silhouette with the moulded comfort panty top

Additional benefits

- The elastic comfort gusset offers additional comfort
- The skin feels comfortable all year round with Clima Comfort (tested by IFTH)
- Clima Fresh – always fresh feet
- Extremely skin-friendly – tested quality to Oeko-Tex Standard 100

* Use mediven comfort for serious edema only in patients with normal weight. With overweight and more serious edema, medi recommends stronger material.

** Natural substance for treating dry and rough skin, has many applications, for example, in baby and child care. Caring protective action remains even after 100 washes.



Available with sparkling Swarovski® crystals.***



***Swarovski® is a registered trademark of Swarovski AG.

Material

medium

Compression


CCL 1, 2

Product benefits

Comfortable handling by easy donning and doffing.



cf. Ben Ammar, F. (DermScan): Clinical evaluation of the skin compatibility of a medical device - application testing realized under dermatological control, Tunis 2012/2013.

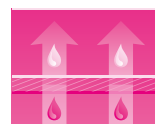

Clima Comfort



Clima Fresh



Perfect Fit



Variety



Colours

The article is available in the following colours.

S = Standard

M = Made-to-measure

Trend colours are available.

Black
S+M

Anthracite
S+M

Navy
S+M

Cashmere
S+M

Caramel
S+M

Rosé***
S+M

Bronze***
S+M

Beige***
S+M

***Delivery period up to 10 working days.

mediven® comfort

All you feel is softness. The cuddly side of compression.

Indications*

- Varicose veins
 - Primary or secondary varicosis
 - Varicose veins in pregnancy
 - Supporting sclerotherapy
 - After varicose vein surgery
- Chronic venous disorders: C0s to C4 according to CEAP
- Thrombosis prophylaxis in mobile patients
- Thrombophlebitis (superficial) or condition after healed phlebitis
- Deep venous thrombosis, condition after thrombosis, post-thrombotic syndrome

* The choice of the optimum compression sock depends on the indication and the patient's individual needs.















Comfort panty top



Double-knitted comfort sole



Silicone dot topband

mediven comfort						
open toe						
Styles	AD	AD pte.	AG / NoB	AG pte. / NoB	AT	AT pte.
						
	calf	calf petite	thigh with topband	thigh petite with topband	panty	panty petite
Class 1	Z540x01-7	Z547x01-7	Z569x01-7	Z559x01-7	Z510x01-7	Z517x01-7
Class 2	Z640x01-7	Z647x01-7	Z669x01-7	Z659x01-7	Z610x01-7	Z617x01-7
closed toe						
Styles	AD	AD pte.	AG / NoB	AG pte. / NoB	AT	AT pte.
						
	calf	calf petite	thigh with topband	thigh petite with topband	panty	panty petite
Class 1	Z540x11-7	Z540x11-7	Z569x11-7	Z559x11-7	Z510x11-7	Z517x11-7
Class 2	Z640x11-7	Z640x11-7	Z669x11-7	Z659x11-7	Z610x11-7	Z617x11-7

See the last page for the article numbers

Standard styles	
Standard	Sizes I – VII Length standard and petite Topband standard and wide
Made-to-measure	Available
Special versions	Available, see page 50
Border	AD tricot border, narrow silicone dot topband AG Silicone dot topband, Silicone dot topband motif, Plain silicone topband rose, Silicone micro dot topband sensitive All topbands are standard styles.
Gusset	AT comfort gusset
Foot part	With open or closed toe Universal foot size, shoe sizes 36 – 43

Material composition		
	Elastane (EL)	Polyamide (PA)
CCL 1	32 %	68 %
CCL 2	35 %	65 %

Washing instructions
Wash-and-wear     
Suitable for washing machine and drier



mediven plus®

All you experience is reliability. Compression has never proven so trustworthy.

Suitable for

- women / men
- normal weight / overweight
- light to heavy venous oedema
- also for soft connective tissue

Your benefits

- Safe and reliable care for a large target group and a large number of indications
- Suitable for HIM and HER due to the opaque look and numerous variants
- Satisfaction of individual customer needs with a wide range of variants, ideal for sensitive and large thighs

Benefits for your customers

- Reliable fit due to the plus-heel with Y-shape, perfectly fitting toe section and broader cuff
- As versatile as your patients' needs
- Excellent durability due to the hard-wearing material

Additional benefits

- Maxi panty top as standard
- Dermatologically tested and proven to be comfortable even for those who are prone to allergic reactions or have sensitive skin
- Antibacterial effect on the whole leg with Clima Fresh
- Four topbands for individual needs
- Extremely skin-friendly – tested quality to Oeko-Tex Standard 100



Available with sparkling Swarovski® crystals.**



**Swarovski® is a registered trademark of Swarovski AG.

Material

medium

Compression

CCL 1, 2, 3

Product benefits

The reliable stocking for every day.



cf. Ben Ammar, F. (DermScan): Clinical evaluation of the skin compatibility of a medical device – application testing realized under dermatological control, Tunis 2012/2013.



Clima Comfort



Clima Fresh



Perfect Fit



Variety



Colours

The article is available in the following colours.

S = Standard

M = Made-to-measure

Trend colours are available.



*Delivery period up to 10 working days.

mediven plus®

All you experience is reliability. Compression has never proven so trustworthy.

Indications*

- Varicose veins
 - Primary or secondary varicosis
 - Varicose veins in pregnancy
 - Supporting sclerotherapy
 - After varicose vein surgery
- Chronic venous disorders: C0s to C5 according to CEAP
- Thrombosis prophylaxis in mobile patients
- Thrombophlebitis (superficial) or condition after healed phlebitis
- Deep venous thrombosis, condition after thrombosis, post-thrombotic syndrome

* The choice of the optimum compression sock depends on the indication and the patient's individual needs.





















Knee length stocking (AD)



Plus-heel and sole reinforcement



Maxi panty top as standard

mediven plus								
Styles	AD	AD pte.	AG / NoB	AG / NoB pte.	AT	AT + maxi panty top	AT pte.	
								
	calf	calf petite	thigh with topband	thigh petite with topband	panty	panty with maxi panty top	panty petite	
Class 1	4A40x01-7	4A47x01-7	4A61x01-7	4A51x01-7	4A10x01-7	4AZ0x01-7	4A17x01-7	
Class 2	4B40x01-7	4B47x01-7	4B61x01-7	4B51x01-7	4B10x01-7	4BZ0x01-7	4B17x01-7	
Class 3	4C40x01-7	4C47x01-7	4C61x01-7	4C51x01-7	4C10x01-7	4CZ0x01-7	4C17x01-7	
Styles	ATE		ATE pte.		AT / U	AT / U pte.	CT	BT
								
	one legged panty		one legged panty petite		maternity panty	maternity panty petite	capri shorts	leggings
	right	left	right	left				
Class 1	4AE3x01-7	4AE2x01-7	4AV3x01-7	4AV2x01-7	4A20x01-7	4A27x01-7	made-to-measure	made-to-measure
Class 2	4BE3x01-7	4BE2x01-7	4BV3x01-7	4BV2x01-7	4B20x01-7	4B27x01-7		
Class 3	4CE3x01-7	4CE2x01-7	4CV3x01-7	4CV2x01-7	4C20x01-7	4C27x01-7		
Styles	ET	AT / H	AT / H pte.	AG	AG pte.			
								
	cycling short	men's leotard	men's leotard petite	thigh with waist attachment	thigh with waist attachment petite			
				right	left	right	left	
Class 1	made-to-measure	4A30x01-7	4A37x01-7	4A63x01-7	4A62x01-7	4A53x01-7	4A52x01-7	
Class 2		4B30x01-7	4B37x01-7	4B63x01-7	4B62x01-7	4B53x01-7	4B52x01-7	
Class 3		4C30x01-7	4C37x01-7	4C63x01-7	4C62x01-7	4C53x01-7	4C52x01-7	

Standard styles



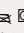

Standard	Sizes I – VII, Length normal and petite Topband normal and wide
Made-to-measure	Available
Special versions	Available, see page 50
Border	AD tricot border, narrow silicone dot topband AF and AG Silicone dot topband, Silicone dot topband motif, Plain silicone topband rose, Silicone micro dot topband sensitive All topbands are standard styles. Waist attachment
Gusset	AT, sizes I – III groin gusset Sizes IV – VII narrow, gusset at the back up to the waist, maxi panty top AT / U with closed panty top AT / H opening horizontal
Foot part	With open or closed toe Universal foot size, shoe sizes 36 – 43

See the last page for the article numbers

Material composition

	Elastane (EL)	Polyamide (PA)
CCL 1	25 %	75 %
CCL 2	31 %	69 %
CCL 3	37 %	63 %

Washing instructions

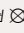Wash-and-wear    
Suitable for washing machine and drier



mediven active®

The hard wearing compression sock.

Suitable for

- men
- normal weight / overweight
- light to heavy venous oedema
- also for soft connective tissue

Your benefits

- Convincing the new target group of physically active men who look for inconspicuous therapy socks
- Additional sales benefit due to the unique knitting technique
- Best rating in categories like inconspicuous look, soft touch, wearing comfort in the iCONSULT study*
- Winner of the iF product design award 2013 offers an additional sales benefit

Benefits for your customers

- Relief of pressure due to a seam you can't feel at the toes
- Suitable for everyday use due to its robust and hard wearing material
- Incredibly soft with its sporty comfort sole as well as the broad cuff

Additional benefits

- Strong and low heel for additional comfort and a fit like a second skin
- The skin feels comfortable all year round with Clima Comfort (tested by IFTH)
- Extremely skin-friendly – tested quality to Oeko-Tex Standard 100

* cf. iCONSULT Forschung & Consulting GmbH, Usage & Attitude-Studie Venenbeschwerden. München, 2011.

Material

strong

Compression

CCL 1, 2

Product benefits

The indestructible and comfortable men's sock.



cf. Ben Ammar, F. (DermScan): Clinical evaluation of the skin compatibility of a medical device - application testing realized under dermatological control, Tunis 2012/2013.



Clima Comfort



Perfect Fit



product
design award

2013




Colours

The article is available in the following colours.
S = Standard




Black
S+M

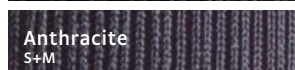

Anthracite
S+M



Navy
S+M

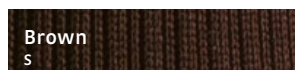

Brown
S



White
S

mediven active®

The hard wearing compression sock.

Indications*

- Primary or secondary varicosis
- Chronic venous disorders: C0s to C4 according to CEAP
- Thrombosis prophylaxis in mobile patients
- Thrombophlebitis (superficial) or condition after healed phlebitis
- Deep venous thrombosis, condition after thrombosis, post-thrombotic syndrome

* The choice of the optimum compression sock depends on the indication and the patient's individual needs.





Knee length stocking (AD)



Seam you can't feel at the toe





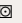
Comfort sole

mediven active			
Styles		AD	AD pte.
			
		calf	calf petite
Class 1	Article No.	3540x11-7	3547x11-7
Class 2	Article No.	3640x11-7	3647x11-7

See the last page for the article numbers

Standard styles	
Standard	Sizes I – VII, Length standard and petite
Special versions	Not available
Border	AD tricot border
Foot part	Closed toe Shoe sizes 38 – 48

Material composition		
	Elastane (EL)	Polyamide (PA)
CCL 1	10 %	90 %
CCL 2	11 %	89 %

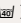Washing instructions
Wash-and-wear     
Suitable for washing machine and drier



mediven® forte

All you feel is protection. Pleasant compression has never been so powerful.

Suitable for

- women / men
- normal weight to obese
- light to heavy venous oedema
- also for soft connective tissue

Your benefits

- Safe and reliable care, even for severe indications
- Reliable therapy for your customers

Benefits for your customers

- Maximum pressure stability (high working pressure for a deep functional effect)
- Extremely hard-wearing seamless material for optimum wearing comfort

Additional benefits

- Extremely soft, extra wide comfort cuffs for the best possible feel
- Clima Fresh over the whole leg for a feeling of freshness around the clock
- Extremely skin-friendly – tested quality to Oeko-Tex Standard 100



Available with sparkling Swarovski® crystals.**



**Swarovski® is a registered trademark of Swarovski AG.

Material

strong

Compression

CCL 2, 3

Product benefits

The strong stocking for severe indications.



cf. Ben Ammar, F. (DermScan). Clinical evaluation of the skin compatibility of a medical device – application testing realized under dermatological control, Tunis 2012/2013.



Clima Fresh



Variety



Perfect Fit



Colours

The article is available in the following colours.

S = Standard

M = Made-to-measure

Trend colours are available.

Black

S+M

Anthracite

S+M

Navy

S+M

Cashmere

S+M

Caramel

S+M

Rosé*

S+M

Bronze*

S+M

Beige*

S+M

*Delivery period up to 10 working days.

mediven® forte

All you feel is protection. Pleasant compression has never been so powerful.

Indications*

- Varicose veins
 - Primary or secondary varicosis
 - Varicose veins in pregnancy
 - Supporting sclerotherapy
 - After varicose vein surgery
- Chronic venous disorders: C0s to C5 according to CEAP
- Thrombosis prophylaxis in mobile patients
- Thrombophlebitis (superficial) or condition after healed phlebitis
- Deep venous thrombosis, condition after thrombosis, post-thrombotic syndrome

* The choice of the optimum compression sock depends on the indication and the patient's individual needs.


















Knee length stocking (AD)



Thigh length stocking (AG) with open toe







Open toe – detail

mediven forte									
Styles	AD	AD pte.	AG / NoB	AG pte./NoB	AG	AG pte.			
									
	calf	calf petite	thigh	thigh petite	thigh with waist attachment	thigh petite with waist attachment			
					right	left	right	left	
Class 2	F240x01-7	F247x01-7	F260x01-7	F250x01-7	F263x01-7	F262x01-7	F253x01-7	F252x01-7	
Class 3	F340x01-7	F347x01-7	F360x01-7	F350x01-7	F363x01-7	F362x01-7	F353x01-7	F352x01-7	
Styles	AT	ATE	AT pte.	AT / U	BT	CT	ET	AT / H	AT / H pte.
									
	panty	one legged panty	panty petite	maternity panty	leggings	capri shorts	cycling short	men's leotard	men's leotard petite
Class 2	F210x01-7	made-to-measure	F217x01-7	made-to-measure	made-to-measure	made-to-measure	made-to-measure	made-to-measure	made-to-measure
Class 3	F310x01-7	made-to-measure	F317x01-7	made-to-measure	made-to-measure	made-to-measure	made-to-measure	made-to-measure	made-to-measure

See the last page for the article numbers

Standard styles	
Standard	Sizes I – VII, Length normal and petite, Topband normal and wide
Made-to-measure	Available
Special version	Available, see page 50
Border	AD tricot border, narrow silicone dot topband AF and AG tricot border Silicone dot topband, Silicone dot topband motif, Plain silicone topband rose, Silicone micro dot topband sensitive All topbands are standard styles.
Gusset	AT, sizes I – III groin gusset Sizes IV – VII narrow gusset at the back up to the waist, AT / U with closed panty top AT / H opening horizontal
Foot part	With open or closed toe Choice of 2 foot lengths Short foot sizes 36 – 38, normal foot sizes 38 – 43

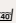Material composition		
	Elastane (EL)	Polyamide (PA)
CCL 2	21 %	79 %
CCL 3	26 %	74 %

Washing instructions
Wash-and-wear    
Suitable for washing machine and drier



mediven[®] Special models in the round-knitted range

Special versions are all variants of the articles that are available above and beyond the standard range of mediven articles.

This overview will help you choose and offer the right add-on in order to provide the best solution for every unique patient and make it easier for you to hold active sales talks. Special versions of mediven take into account the individuality of patients with venous disease and ensure pleasant and comfortable wear combined with easier treatment. Compliance means high patient loyalty to treatment!

Increase your chance of successful repeat sales and thus long-term customer loyalty.



Standard models

Foot tips

Open toe

Suitable for

- customers who are prone to perspiration
- customers who often put on an additional stocking



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Standard	Standard	Standard	Standard
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge

Closed toe

Suitable for

- single foot size: mediven elegance, mediven plus and mediven comfort
foot size 36 – 43
- two foot sizes for mediven forte
small foot up to 23 cm
large foot from 24 cm



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Standard	Standard	Standard	Standard
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge

Special models

Foot tips

Soft toe

Suitable for

- customers who are hypersensitive in the toe region (e.g. hammer toe).
The soft foot tip does not press on the toes.

Note: Due to the various knit materials, it can come in various colours.



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Extra charge	Extra charge	Extra charge	Extra charge
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge

Plush foot

Available in AD, AG and AT varieties

Suitable for

- customers with pressure-sensitive feet
- customers with cold feet

available until
December 31, 2019



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	–	–	Extra charge	–
Made-to-measure	–	–	Extra charge	–
In case of modification	–	–	–	–

Hallux valgus toe section

Suitable for

- customers who require freedom of movement for their toes
- simultaneously relieving pressure from the “critical” bunion area



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Extra charge	Extra charge	Extra charge	Extra charge
In case of modification	–	–	–	–

Anatomical toe

Suitable for

- customers who are hypersensitive in the toe region
- particularly well suited to narrow shoes (e.g. business shoes)

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Extra charge	Extra charge	Extra charge	Extra charge
In case of modification	–	–	–	–



Jersey foot tips can be lengthened or shortened according to specifications

Suitable for

- individual supply according to customer specific measurements

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge



Jersey foot tips lengthened according to specifications



Jersey foot tips shortened according to specifications

Jersey border at the foot lengthened according to specifications

Suitable for

- individual supply according to customer specific measurements

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge



Foot length shortened according to specifications

Suitable for

- individual supply according to customer specific measurements

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge



Pantyhose with compressive panty top

Suitable for

- large panty section circumference, lax connective tissues, pendulous abdomen, weak abdominal muscles, diastases
- pelvic vein thrombosis, lymphoedema and lipoedema, lipolymphoedema



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	–	–
Made-to-measure	Extra charge	Extra charge	–	Extra charge


Panty top longer / shorter

Indications longer

- Large bottom
- Large abdomen

Indications shorter

- Small bottom

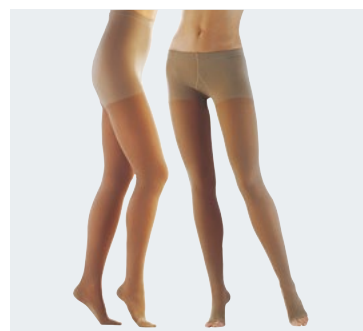

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	–	–
Made-to-measure	No extra charge	No extra charge	–	No extra charge
In case of modification	Extra charge	Extra charge	–	Extra charge

Panty top front / back, longer / shorter


Suitable for front longer or back shorter

- small bottom, large abdomen

Suitable for front shorter or back longer

- large bottom, small abdomen

Note: Max. 9 cm shorter or longer

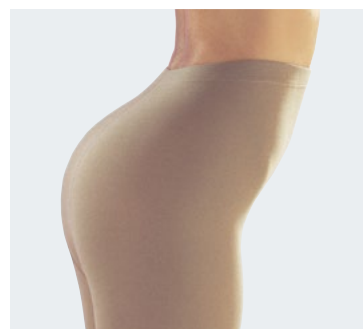

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	–	–
Made-to-measure	No extra charge	No extra charge	–	No extra charge
In case of modification	Extra charge	Extra charge	–	Extra charge

Pantyhose with open crotch

Suitable for

- immobile patients
- incontinent patients
- catheterized patients
- genital mycoses



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	–	–
Made-to-measure	No extra charge	No extra charge	–	No extra charge
In case of modification	Extra charge	Extra charge	–	Extra charge

Pantyhose with stoma opening

Suitable for

- stoma patients

Please note

- First fit the customer with the pantyhose, then mark the stoma opening. Then send us the pantyhose so we can create the hole. This way, the stoma opening will be in the correct position.
- Please bring a "template" for repeat sales



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	–
Made-to-measure	Extra charge	Extra charge	Extra charge	Extra charge
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge

One-leg pantyhose

- As an alternative to a waist attachment or topband
- For proximal thromboses




	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	–	Extra charge	–	–
Made-to-measure	–	Extra charge	–	Extra charge

* Extra charge rules always based on price AG with tricot border.

Waist attachment

Suitable for

- conical thighs
- very soft thighs
- allergic patients (alternative to silicone-coated topband)

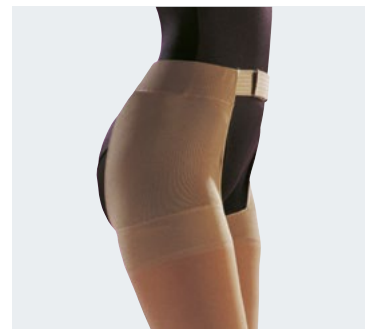

	mediven elegance	mediven plus*	mediven comfort	mediven forte*
Standard	–	Extra charge	–	Extra charge
Made-to-measure	–	Extra charge	–	Extra charge
In case of modification	–	Extra charge	–	Extra charge

* Extra charge rules always based on price AG with tricot border.

Silicone dot topband, Silicone dot topband motif, Silicone micro dot topband sensitive, Plain silicone topband rose

Available for mediven elegance, mediven comfort, mediven plus and mediven forte

Plain silicone topband rose: Tried-and-tested, reliable hold thanks to the surface coating. Elegant, feminine design. Fine, thin topband quality.

Silicone micro dot topband sensitive: Barely noticeable, fine silicone beads on the reverse, which provide reliable hold. Very good air circulation. Neutral design. Fine, thin topband quality.

Silicone dot topband motif: Tried-and-tested, reliable hold thanks to textured silicone. Good air circulation. Fashionable design. Sturdy, robust topband quality.

Silicone dot topband: Tried-and-tested, reliable hold thanks to textured silicone. Good air circulation. Neutral design. Sturdy, robust topband quality.



Plain silicone topband rose



Silicone dot topband



Silicone dot topband motif



Silicone micro dot topband sensitive

Stocking suspenders

Stocking suspenders with strap, adjustable with 2 or 4 suspenders. Can be used with all styles.

- One side in sizes:
1 Waist 80 – 90 cm, 2 Waist 95 – 105 cm, 3 Waist 110 – 120 cm
- Both sides in sizes:
1 Waist 80 – 90 cm, 2 Waist 95 – 105 cm, 3 Waist 110 – 120 cm



Yard ware (1 m, 5 m)

Can be ordered as required!

Silicone dot topband (NoB)

Silicone dot topband motif (NoM)

Plain silicone topband rose (FBR)

Silicone micro dot topband sensitive (MBs)

In all colours, composition: silicone,
polyamide, polyester, elastane



Plain silicone
topband rose (FBR)



Silicone dot
topband (NoB)



Silicone dot
topband motif
(NoM)



Silicone micro dot
topband sensitive
(MBs)

mediven[®]

Gusset variants

Gussets are knitted into the groin area of compression pantyhose. The following pages will give you an overview of the “standard gusset” that is used in all the round knit models.

In addition to this, you can choose from further gusset variants for standard sizes and made-to-measure garments for your customers, since a size in centimetres says little about the anatomy of patients with venous disease or the relationship between abdomen and bottom. Besides the pictures, a description simplifies the correct choice of gusset. The optimum compression pantyhose gives the wearer absolute comfort in the panty top.





mediven® Gusset standard variants

Standard Gusset	mediven elegance	mediven plus	mediven forte	mediven comfort
Comfort gusset	Sizes I – VII cT 60 – 125 cH 80 – 145			Sizes I – VII cT 60 – 125 cH 80 – 135
Crotch gusset Small abdomen and small bottom For women with a less pronounced abdomen and bottom		Sizes I – III cT 60 – 95 cH 80 – 110	Sizes I – III cT 60 – 95 cH 80 – 110	
Narrow gusset at the back Small abdomen and large bottom For women with a pronounced bottom, but a relatively flat abdomen		Sizes IV – VII cT 80 – 125 cH 100 – 145 Maxi panty top Sizes I – III cT 95 – 140 cH 110 – 150	Sizes IV – VII cT 80 – 125 cH 100 – 145	
Wide gusset at the back Large abdomen and very large bottom For women with a pronounced bottom but a comparatively flat abdomen		Maxi panty top Sizes IV – V cT 120 – 150 cH 135 – 160		
Extra wide gusset at the back Very large abdomen and very large bottom For women with a pronounced bottom and abdomen		Maxi panty top Sizes VI – VII cT 140 – 165 cH 145 – 170		
Maternity panty AT / U, closed Growing abdomen Suitable for pregnant women with a large, growing abdominal circumference	Sizes I – VII cT from 60 cH from 80	Sizes I – VII cT from 60 cH from 80		
Men's leotard, horizontal fly For men with fly		Sizes I – VII cT 60 – 125 cH 80 – 145		

Comfort gusset

Standard gusset





Standard model	
Models	mediven elegance, mediven comfort
Sizes	I – VII
Made-to-measure	
Models	mediven elegance, mediven comfort
Please order	"Comfort crotch"

Crotch gusset

Standard gusset

For women with a small abdomen and bottom.

Standard model	
Models	mediven plus, mediven forte
Sizes	I – III
Made-to-measure	
Models	mediven plus, mediven forte
Please order	"Small gusset in crotch only"

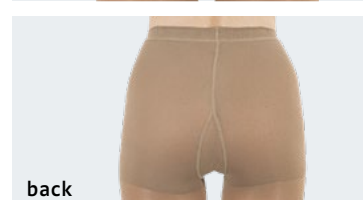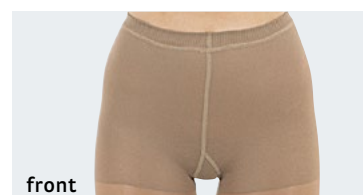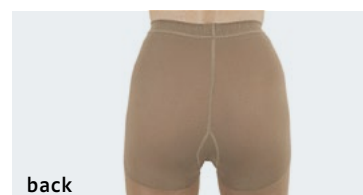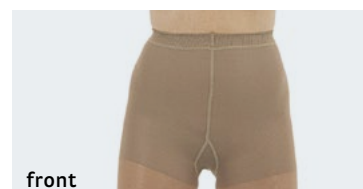

Maxi-Panty top

For women with large hip and waist circumferences.

Special model

Models	mediven plus
Sizes	I – VII
Please order	“Maxi-Panty top”

Narrow gusset, up to the waist at the back


For women with large bottoms and a relatively flat abdomen.

Made-to-measure

Models	mediven elegance, mediven plus, mediven forte
--------	---

Special model

Models	mediven elegance, Size IV – VII mediven plus, Size I – III
Please order	“Narrow gusset up to waist at back” Can also be incorporated in the AT / U. Caution: this needs a transverse seam in the anterior crotch area.

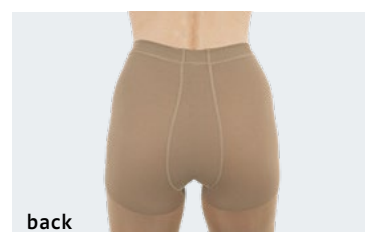

Narrow gusset, up to the waist at the front


For women with a large abdomen and a relatively flat bottom.

Made-to-measure

Models	mediven elegance, mediven plus, mediven forte
--------	---

Special model

Models	mediven elegance, Size I – III mediven plus, Size I – VII
Please order	“Narrow gusset, up to the waist at the front”

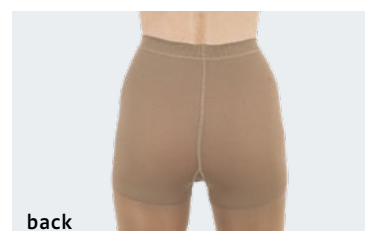

Wide gusset, up to the waist at the back

For women with a far more pronounced bottom and, in contrast, a flat abdomen.

Made-to-measure

Models mediven elegance, mediven plus, mediven forte, mediven comfort


Special model

Models mediven elegance, mediven plus, mediven comfort

Please order “Wide gusset, up to waist at back”
Can also be incorporated in the AT / U. **Caution:** this needs a transverse seam in the anterior crotch area.



front

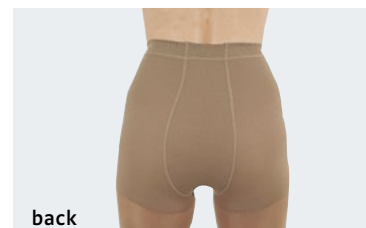

back

Wide gusset, up to the waist at the front

For women with a pronounced abdomen and a relatively flat bottom.

Made-to-measure

Models mediven elegance, mediven plus, mediven forte, mediven comfort


Special model

Models mediven elegance, mediven plus, mediven comfort

Please order “Wide gusset, up to the waist at the front”



front

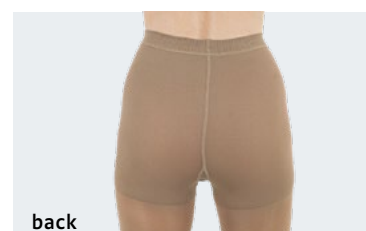

back

Variable gusset, back

For women with a narrow waist, a far more pronounced bottom and, in contrast, a flat abdomen.

Made-to-measure

Models mediven elegance, mediven plus, mediven forte


Special model

Models mediven elegance, mediven plus

Please order “Variable gusset, back”



front

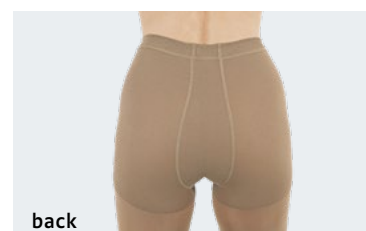

back

Variable gusset, front

For women with a narrow waist, a far more pronounced abdomen and, in contrast, a flat bottom.

Made-to-measure

Models mediven elegance, mediven plus, mediven forte


Special model

Models mediven elegance, mediven plus

Please order "Variable gusset, front"



front

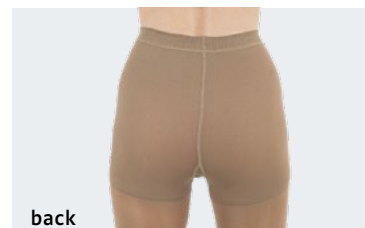

back

Continuous gusset, narrow

For women who are sensitive to pressure and have a slightly larger bottom and abdomen.

Made-to-measure

Models mediven elegance, mediven plus, mediven forte

Special model

Models mediven elegance, mediven plus

Please order "Continuous gusset, narrow"



front



back

Continuous gusset, front narrow, back wide

For women with a large abdomen and a particularly pronounced bottom.

Made-to-measure

Models mediven elegance, mediven plus, mediven forte


Special model

Models mediven elegance, mediven plus

Please order "Continuous gusset, front narrow, back wide"



front

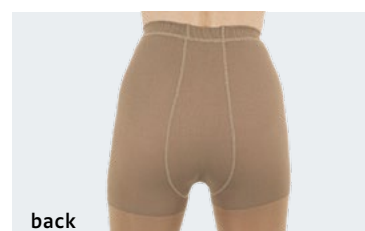

back

Continuous gusset, front wide, back narrow

For women with a large abdominal circumference and a particularly pronounced bottom.


Made-to-measure

Models mediven elegance, mediven plus, mediven forte

Special model

Models mediven elegance, mediven plus

Please order "Continuous gusset, front wide and back narrow"

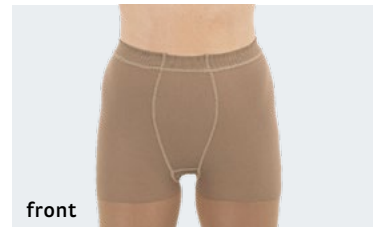

Continuous gusset, wide

For women with large abdomen and hips.



Made-to-measure

Models mediven elegance, mediven plus, mediven forte, mediven comfort

Special model

Models mediven elegance, mediven plus, mediven comfort

Please order "Continuous gusset, wide"

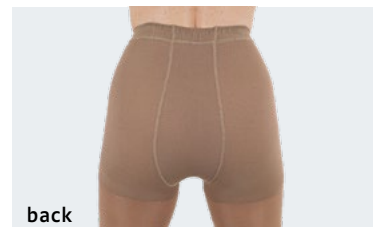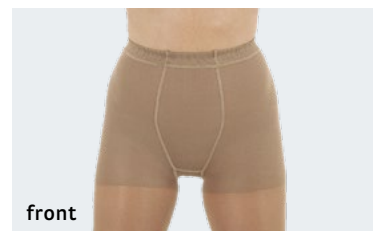


Maternity panty AT / U, closed

Standard gusset

For pregnant women with large and growing abdominal circumferences.

Standard model	
Models	mediven elegance, mediven plus
Sizes	I – VII
Made-to-measure	
Models	mediven elegance, mediven plus, mediven forte
Please order	“Maternity panty AT / U, closed” The AT / U is possible with a narrow (No. 4) or with a wide (No. 6) gusset up to the waist at the back. Caution: this needs a transverse seam in the anterior gusset area.

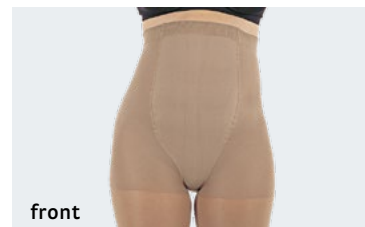

Maternity panty AT / U, open

For pregnant women with large and growing abdominal circumferences and tenderness to pressure.


Made-to-measure	
Models	mediven elegance, mediven plus, mediven forte
Special model	
Models	mediven elegance, mediven plus
Please order	“Maternity panty AT / U, open”



Men's gusset with vertical / horizontal fly

For men, with fly.

Standard model	
Models	mediven plus (standard horizontal fly)
Sizes	I – VII
Made-to-measure	
Models	mediven plus, mediven forte
Please order	“Men's crotch with horizontal fly” or “Men's crotch with vertical fly”

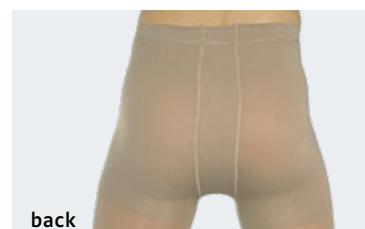



Basic care of medi

Duomed is the economic solution for a reliable compression therapy.

The cooperative partnership with medi enables you to address all customer needs and cover the complete product range of basic care, mediven quality products and premium special models. Simply obtain everything from medi through one provider.



Duomed

The economic solution for venous problems.

Suitable for

- women / men
- normal weight / over weight
- light to severe venous oedema
- also for soft connective tissue

Your benefits

- Offer to your customers an economic alternative
- You can be sure to offer appropriate medical compression

Benefits for your customers

- Combines ideal wearing properties and appropriate medical compression
- Your economic solution for a reliable compression therapy

Additional benefits

- Clima Fresh – always fresh feet
- Extremely skin-friendly – tested quality to Oeko-Tex Standard 100

Material

medium

Compression

CCL 1, 2

Product benefits

Economic wear for daily care.



Clima Fresh



Colours

The article is available in the following colours.

S = Standard

Beige

S

Sand

S

Caramel

S+M

Black

S

Duomed

The economic solution for venous problems.

Indications*

- Varicose veins
 - Primary or secondary varicosis
 - Varicose veins in pregnancy
 - Supporting sclerotherapy
 - After varicose vein surgery
- Chronic venous disorders: C0s to C4 according to CEAP
- Thrombosis prophylaxis in mobile patients
- Thrombophlebitis (superficial) or condition after healed phlebitis
- Deep venous thrombosis, condition after thrombosis, post-thrombotic syndrome

* The choice of the optimum compression sock depends on the indication and the patient's individual needs.























Knee length stocking (AD) with closed toe



Thigh length stocking (AG) with closed toe




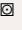


Panty (AT) with open toe

Duomed								
Styles	open toe	AD	AD pte.	AG	AG pte.		AG / SN	
								
		calf	calf petite	thigh	thigh petite		thigh with topband	
Class 1	Article No.	V140x02-6	V147x02-6	V160x02-6	V150x02-6		V161x02-6	
Class 2	Article No.	V240x02-6	V247x02-6	V260x02-6	V250x02-6		V261x02-6	
Styles	open toe	AG pte. / SN	AT	AT pte.	AG		AG pte.	
								
		thigh petite with topband	panty	panty petite	thigh with waist attachment right	thigh with waist attachment left	thigh petite with waist attachment right	thigh petite with waist attachment left
Class 1	Article No.	V151x02-6	V110x02-6	V117x02-6	V163x02-6	V162x02-6	V153x02-6	V152x02-6
Class 2	Article No.	V251x02-6	V210x02-6	V217x02-6	V263x02-6	V262x02-6	V253x02-6	V252x02-6
Styles	closed toe	AD	AD pte.	AG	AG pte.		AG / SN	
								
		calf	calf petite	thigh	thigh petite		thigh with topband	
Class 1	Article No.	V140x12-6	V147x12-6	–	–		V161x12-6	
Class 2	Article No.	V240x12-6	V247x12-6	–	–		V261x12-6	
Styles	closed toe	AG pte. / SN	AT	AT pte.				
								
		thigh petite with topband	panty	panty petite				
Class 1	Article No.	V151x12-6	V110x12-6	V117x12-6				
Class 2	Article No.	V251x12-6	V210x12-6	V217x12-6				

See the last page for the article numbers

Standard styles	
Standard sizes	S – XXL
Made -to-measure	not available
Special models	not available
Border	–
Gusset	–
Foot part	With open or closed toe

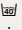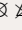Material composition		
	Elastane (EL)	Polyamide (PA)
CCL 1	37 %	63 %
CCL 2	43 %	57 %
Washing instructions		
Wash-and-wear    		
Suitable for washing machine and drier		



mediven harmony[®]

The round knitted arm sleeve is suitable for example to be used for less severe, post-traumatic oedemas or oedemas caused by inactivity.





mediven harmony®

Comfortable and effective round-knitted
armsleeve.

Benefits

- The double ellipses on the point and in the crease of the elbow offer free motion. They are designed to assist with lymphatic drainage and to reduce wrinkles to a minimum to prevent soreness.
- High containment: high wall stability with a gentle touch of supple knit.
- Unmatched well-being: the innovative fabric structure stimulates the tissue and exerts a pleasant massage effect.
- The anatomical shape of the armsleeve ensures easy and correct donning and doffing.
- The hand part of the combined armsleeve offers two flat seams and a wedge at the web of the thumb. This reduces the tension and increases compliance.
- The silicone topband is knitted to the armsleeve with a flat and unconscious seam which secures hold but not sticking because of breathability.

Additional benefits

- Perfect Fit
- The skin feels comfortable all year round with Clima Comfort
- Variety: available in 2 compression classes, 3 colours and 2 hand pieces
- Premium quality RAL standard
- Latex-free
- Machine wash gentle cycle at 40° C and cool tumble dry
- Naturally moisturizing properties of Lanolin

Compression

CCL 1, 2

Product benefits



Perfect Fit



Clima Comfort



Variety



Double ellipses



Colours

The article is available in the
following colours.
S = Standard



Sand
S



Caramel
S



Black
S*

* Black: minimum order quantities of 20
pieces per size and variant; approx. 4
weeks delivery period

mediven harmony[®]

**Comfortable and effective round-knitted
armsleeve.**

Indications*

- Shoulder arm vein thrombosis
- Postoperative and posttraumatic oedema
- Primary and secondary lymphoedema of the arm stage 1 (conditionally)

Please note: not suitable for pronounced oedema

* These indications should be seen as a recommendation. The garment model and the strength of pressure required, i.e. the compression class, depend on the diagnosis, the treatment site, the clinical findings and the patient's susceptibility to oedema. Please also consider the patient's individual requirements and the circumstances (comorbidities, mobility, BMI, state of their skin and tissues, etc.).

Hand part (glove): prevention of accumulation of lymph fluid in the hand area while wearing a medical compression armsleeve



**Narrow napped silicone
topband**



Combined armsleeve with hand part AG, sand




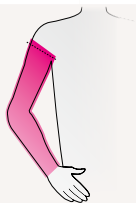
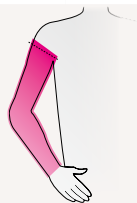
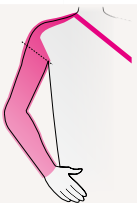


Ellipse in the elbow crease



Ellipse on the point of the elbow

Variants (all variants available as regular or extra wide top)

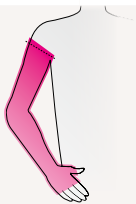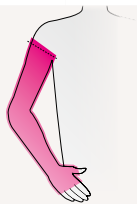	AG combined armsleeve		AG combined armsleeve with topband		AG combined armsleeve with shouldercap	
						
Standard length	Regular	CCL 1 Art.No.: 2Y10x01-8 CCL 2 Art.No.: 2Z10x01-8	Regular	CCL 1 Art.No.: 2Y11x01-8 CCL 2 Art.No.: 2Z11x01-8	Regular	CCL 1 Art.No.: 2Y15x01-8 CCL 2 Art.No.: 2Z15x01-8
	Extra wide	CCL 1 Art.No.: 2Y10x11-8 CCL 2 Art.No.: 2Z10x11-8	Extra wide	CCL 1 Art.No.: 2Y11x11-8 CCL 2 Art.No.: 2Z11x11-8	Extra wide	CCL 1 Art.No.: 2Y15x11-8 CCL 2 Art.No.: 2Z15x11-8
Short length*			Regular	CCL 1 Art. No.: 2Y41701-8 CCL 2 Art. No.: 2Z41701-8		
			Extra wide	CCL 1 Art. No.: 2Y41711-8 CCL 2 Art. No.: 2Z41711-8		
	CG		CG with topband		CG with shouldercap	
						
Standard length	Regular	CCL 1 Art.No.: 2Y00x01-8 CCL 2 Art.No.: 2Z00x01-8	Regular	CCL 1 Art.No.: 2Y01x01-8 CCL 2 Art.No.: 2Z01x01-8	Regular	CCL 1 Art.No.: 2Y05x01-8 CCL 2 Art.No.: 2Z05x01-8
	Extra wide	CCL 1 Art.No.: 2Y00x11-8 CCL 2 Art.No.: 2Z00x11-8	Extra wide	CCL 1 Art.No.: 2Y01x11-8 CCL 2 Art.No.: 2Z01x11-8	Extra wide	CCL 1 Art.No.: 2Y05x11-8 CCL 2 Art.No.: 2Z05x11-8
Short length*			Regular	CCL 1 Art. No.: 2Y31701-8 CCL 2 Art. No.: 2Z31701-8		
			Extra wide	CCL 1 Art. No.: 2Y31711-8 CCL 2 Art. No.: 2Z31711-8		

x = colour; replace with 5 for black, 7 for caramel or 8 for sand

* Short length only available in caramel

Armsleeve	Size I		Size II		Size III		Size IV	
Circumference (cm)	Regular	Extra wide	Regular	Extra wide	Regular	Extra wide	Regular	Extra wide
cG	25,5 – 28,0	28,5 – 31,0	28,0 – 30,5	30,5 – 33,5	30,0 – 33,0	33,0 – 36,0	32,0 – 35,0	35,0 – 38,5
cF	23,5 – 25,5	25,5 – 28,0	25,0 – 27,5	27,5 – 30,0	27,5 – 30,0	30,0 – 33,0	29,5 – 32,0	32,0 – 35,0
cE	23,0 – 25,0	24,0 – 26,0	24,5 – 26,5	26,5 – 29,0	26,0 – 28,5	28,5 – 31,0	27,5 – 30,0	30,0 – 33,0
cD	22,0 – 24,0	24,0 – 26,0	23,0 – 25,0	25,0 – 27,5	24,0 – 26,0	26,0 – 28,5	25,0 – 27,5	28,0 – 30,5
cC 1 = 100% **	15,0 – 16,5		16,5 – 18,0		18,0 – 19,5		19,5 – 21,0	
cC	13,5 – 14,5		14,5 – 15,5		16,0 – 17,5		17,5 – 19,0	
cA	15,0 – 18,0		16,0 – 19,0		17,0 – 20,0		19,0 – 22,0	
Armsleeve	Size V		Size VI		Size VII		Size VIII***	
Circumference (cm)	Regular	Extra wide	Regular	Extra wide	Regular	Extra wide	Regular	Extra wide
cG	33,5 – 36,5	37,0 – 40,5	36,0 – 39,5	39,5 – 43,0	38,0 – 41,5	41,5 – 45,5	40,5 – 44,5	45,0 – 49,5
cF	31,0 – 34,0	34,0 – 37,0	33,0 – 36,0	36,0 – 39,5	35,0 – 38,5	38,5 – 42,0	37,5 – 41,0	41,5 – 45,5
cE	29,0 – 31,5	31,5 – 34,5	30,5 – 33,5	33,5 – 36,5	32,5 – 35,5	35,5 – 39,0	35,0 – 38,5	38,5 – 42,0
cD	27,0 – 29,5	29,5 – 32,0	28,5 – 31,0	31,0 – 34,0	30,5 – 33,5	33,5 – 36,5	33,0 – 36,0	36,0 – 39,5
cC 1 = 100% **	21,0 – 23,0		23,0 – 25,0		25,0 – 27,5		27,5 – 30,0	
cC	19,0 – 20,5		20,5 – 22,5		22,5 – 24,5		25,0 – 27,5	
cA	20,0 – 23,0		22,0 – 25,0		24,0 – 27,0		26,0 – 29,0	
Length (cm)	Standard length		Short length		** Measurement cC1 is 5 cm above cC *** Size VIII: Minimum order quantity of 20 pieces per colour			
	41 – 47		36 – 40					

** Measurement cC1 is 5 cm above cC

*** Size VIII: Minimum order quantity of 20 pieces per colour and variant; delivery period approx. 4 weeks

Variants

AC1 hand piece
without fingers with thumb base



CCL 1 Art.No.: 2Y20x02 – 7
CCL 2 Art.No.: 2Z20x02 – 7

AC1 glove
with compressive fingers



CCL 1 Art.No.: 2Y21x02 – 7
CCL 2 Art.No.: 2Z21x02 – 7

Circumference / Hand part (cm)

Sizes	II	III	IV	V	VI	VII
cC 1 = 100 % *	15,0 – 18,0	17,0 – 20,0	19,0 – 21,5	21,0 – 23,5	23,0 – 26,0	25,5 – 30,0
cC	13,5 – 15,5	15,0 – 18,0	17,0 – 20,0	19,0 – 21,0	20,0 – 23,5	22,5 – 27,5
cB	15,5 – 20,0	17,5 – 21,0	19,5 – 23,0	20,5 – 24,0	22,5 – 26,0	24,5 – 30,0
cA	15,0 – 19,0	17,0 – 20,0	19,0 – 22,0	20,0 – 23,0	22,0 – 25,0	24,0 – 29,0

Circumference / Fingers (cm) / Sizes

Sizes		II	III	IV	V	VI	VII
Thumb	Bottom cX	7,0	7,5	8,0	8,5	9,0	10,0
	Top cZ	6,0	6,5	6,5	7,0	7,5	8,0
Index finger	Bottom cX	6,5	7,0	7,5	8,0	8,5	9,5
	Top cZ	5,0	5,5	6,0	6,0	6,5	7,0
Middle finger	Bottom cX	6,5	7,0	7,5	8,0	8,5	9,5
	Top cZ	5,0	5,5	6,0	6,0	6,5	7,0
Ring finger	Bottom cX	6,0	6,5	7,0	7,0	7,5	9,0
	Top cZ	5,0	5,0	5,5	5,5	6,0	6,5
Little finger	Bottom cX	5,5	6,0	6,0	6,5	7,0	8,0
	Top cZ	4,5	4,5	5,0	5,0	5,5	6,0



Length / Hand part (cm), all sizes

AB	3,5 – 4,5
AC	10,0 – 12,0
AC1	15,0 – 16,0

Length / Fingers (cm)


Thumb	3,5 – 4,5
Index finger	4,5 – 5,5
Middle finger	5,5 – 6,5
Ring finger	4,5 – 5,5
Little finger	3,5 – 4,5

* Measurement cC1 is 5 cm above cC

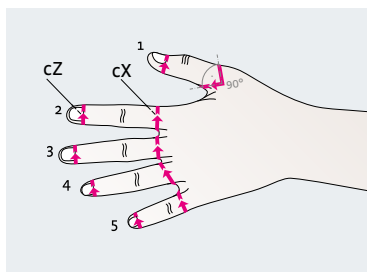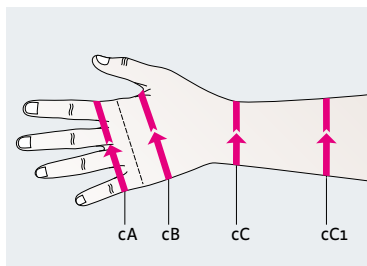

Circumference cZ is measured
at the nailbed.

Circumference cX is measured
at the base of the finger.

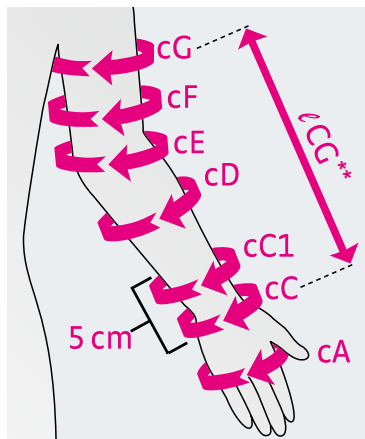

Measurement median harmony,
sizing chart see page 79.

**Length CG is measured on the inside of the arm
from the wrist to the anterior axillary fold.







CIRCAID®

Adjustable compression devices

Adjustable compression garments for the treatment of venous diseases.

circaid products enable patient engagement earlier in the treatment of venous diseases and improve compliance with compression therapy.

circaid products are:

- Cost-effective compared to bandages
- Time saving compared to bandages
- Safer and simpler treatment compared to bandages: no more wrinkles and complicated bandaging techniques
- Better outcomes for your patients: self managing techniques improve patient compliance

Empower your patients now!



Compression therapy by the specialist

**medi offers the most comprehensive line
of compression garments for individual needs.**

The inelastic range of circaid is the cost-effective alternative to short stretch and multi-layer bandage systems. circaid products guarantee a constant compression level and feature a fast and easy donning and doffing system. The ability to instantly readjust the circaid system helps to consistently apply compression to a reducing limb. This is not possible with bandaging without completely removing the layers. circaid garments promote self-management and thus an increased patient compliance as well as an improved quality of life.

Inspiration from nature.

Giraffes have venous pressures of more than 250 mmHg at their forelegs and hind legs, three times more than humans. However, giraffes do not suffer from lymphoedema or venous disorders. The answer is the skin: giraffe skin is inelastic, meaning it does not stretch. Thus, giraffes are not susceptible to lymphoedema or venous diseases even though they may be on their feet 24 hours a day. circaid has incorporated this concept into its products creating solutions for an increased patient compliance and a better quality of life.

The circaid founder's wife suffered from lymphoedema for years without finding a successful treatment. When her husband – an engineer and inventor – saw the pain his wife was in, he knew he had to find a solution. The result was the first patented circaid legging (short for circulation aid) inspired by the inelastic skin giraffes have. It literally put his wife back on her feet to resume a normal and active life. This core technology would evolve into circaid products.



Inelastic

circaid[®] features

Overview of circaid characteristics.



Built-In-Pressure

Built-In-Pressure system
To set and control the correct therapeutic level of pressure.



Breathe-O-Prene

Breathe-O-Prene[®]
Breathable, flexible and comfortable material for best wearing comfort.




Juxtaposed band

Juxtaposed band system
Ensures a perfect fit as the straps can be individually adjusted while on the limb.



Variety

Variety
Wide range of styles and sizes available.

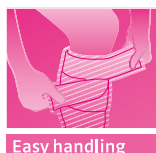

Easy handling

Easy handling
Easy donning and doffing promotes patient empowerment.



Inelastic

Inelastic
Inelastic giraffe skin – core technology of all circaid products.



Very good skin tolerability and suitability for everyday wear
• Without cotton or rubber





circaid® Built-In-Pressure system (BPS) patented technology*

circaid inelastic compression garments are engineered with the ability to be set to precise compression ranges with a graduated compression profile from the ankle up the leg.

The user-friendly design of the circaid Built-In-Pressure system (BPS) enables patients to achieve the necessary compression ranges needed for the treatment and management of chronic venous disease.

With the circaid Built-In-Pressure system (BPS) guide card patients are able to set their prescribed compression range while donning their garment and to readjust the compression range during the day as needed. Therefore, the appropriate compression range is ensured throughout the period of wear. Simultaneously, it is very simple to verify the correct range of applied pressure.

* patented within the USA



circaid® juxtalite® lower leg

Adjustable compression garment for venous diseases.

Indications

- Varicose veins
 - Primary or secondary varicosis
 - Supporting sclerotherapy
- Chronic venous disorders: C1 to C6 according to CEAP (For those patients who are not able to apply compression stockings due to a lack of mobility, or severe comorbidities)

Product features

- Graduated compression profile
- Inelastic material for high working pressure
- Garment straps can be individually readjusted while on the limb
- Anti-odour and anti-microbial fabric
- Latex-free for skin-friendly wearing comfort
- Sustained compression with defined mmHg for up to 24 hours a day if required

Benefits for your patients

- Promotes self-management:
instantly adjustable for easy application
- Promotes patients empowerment:
easy donning and doffing process
- Promotes a better quality of life:
improves hygiene
- Promotes normal mobility and activity

Your benefit

- Offering an alternative therapy concept for those patients who are not able to apply compression stockings


Variants

- Available in 8 sizes (S – XXL, M / L / XL extra wide) and 2 lengths (short, long)

Compression

20 – 50 mmHg

Product benefits

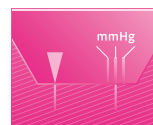

Built-In-Pressure



Breathe-O-Prene



Juxtaposed band



Variety



Easy handling



Inelastic

Colours

The article is available in the following colours.
S = Standard

Beige
S

circaid[®] juxtalite[®]

Adjustable compression garment for venous diseases.

Your order includes

- inelastic adjustable compression garment
- circaid undersleeve lower leg lycra
- pair of circaid compression anklets
- circaid Built-In-Pressure system (BPS) guide card



circaid juxtalite

circaid[®] juxtalite[®] ankle foot wrap

Additional compression for the foot.

Product features

- Provides additional compression to foot and ankle
- Instantly adjustable
- Can be trimmed to provide the perfect fit
- Available in 2 sizes (S, M)



circaid juxtalite ankle foot wrap

circaid[®] power added compression band




Additional, customisable compression option for the foot.

Product features


- Provides additional compression to the arch of the foot and around the malleoli
- Used in combination with the circaid compression anklet
- Can be trimmed to fit almost every foot shape
- Available in standard and large



circaid power added compression band
(pac band)

circaid juxtalite			
Styles	AD	AD	AB
			
	short	long	ankle foot wrap
Size / Article No.	S	JU15702	JU15002
	M	JU15703	JU15003
	L	JU15704	JU15004
	XL	JU15705	JU15005
	XXL	JU15706	JU15006
	M extra wide	JU15713	JU15013
	L extra wide	JU15714	JU15014
	XL extra wide	JU15715	JU15015

Measurement chart circaid® juxtalite®

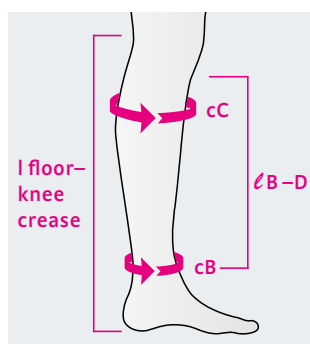


Circumferences in cm				
	S	M	M – X	L
cC	26 – 36	33 – 44	44 – 54	41 – 51
cB	19 – 22	22 – 27	22 – 27	25 – 32
	L – X	XL	XL – X	XXL
cC	51 – 61	37 – 48	48 – 61	53 – 64
cB	25 – 32	30 – 37	30 – 37	35 – 42

Actual garment length in cm		
	Short	Long
B – D	28	33
Sizing measurement		
floor – knee crease	< 44	> 44


All garments contain the circaid Built-In-Pressure system (BPS).

Measurement chart circaid® juxtalite® ankle foot wrap

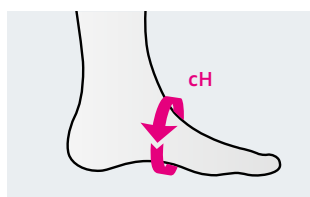

Circumferences in cm		
	S	M
cH	19 – 26	24 – 31

Measurement chart circaid® power added compression band

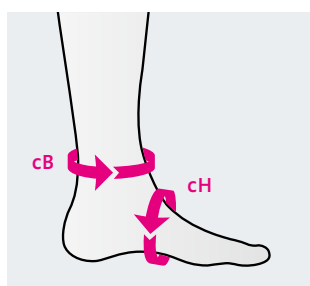

Circumferences in cm		
	Standard	Large
cB	19 – 27	25 – 50
cH	19 – 36	22 – 41
Article No.	JU5P001	JU5P002





Over the counter products

medi also offers numerous products, which patients can purchase without a prescription at a specialist shop. For example medi's travel socks (medi travel men and medi travel women) can help reduce the risk of thrombosis on longer trips. They ensure light, relaxed legs and give the traveler a feeling of safety. Moreover, for people with sensitive skin, medi offers a special skin care series to complement compression therapy. Rich ingredients pamper the skin and help it regenerate. Learn more about medi's over the counter products on the following pages.



medi travel[®] men

Compression garments on the journey can reduce the risk of traveller's thrombosis.*

Your benefits

- Increase your cross-selling by offering your customer a travel sock for prevention
- Complete your product range for the prevention of traveller's thrombosis and gain new customers
- Offer a product that has been specially designed to match the male target group and benefit from the access to unexploited market potential

Benefits for your customers

- Graduated compression can reduce the risk of thrombosis during travelling*
- Antistatic material with a reinforced sole for excellent comfort
- Attractive ribbed look makes medi travel men appear just like a regular man's sock

Additional benefits

- The skin feels comfortable all year round with Clima Comfort (tested by IFTH)
- With active breathing and elastic materials, Perfect Fit ensures a pleasant wearing comfort and an optimum fit
- Extremely skin-friendly – tested quality to Oeko-Tex Standard 100

Product benefits



Clima Comfort



Perfect Fit



medi travel complies with all applicable EU Directives. All the prescribed conformity assessment procedures have been completed.

Colours

The article is available in the following colours.

S = Standard

Black
S

Anthracite
S

* Hsieh HF, Lee FP. Graduated compression stockings as prophylaxis for flight-related venous thrombosis: systematic literature review. J Adv Nurs 2005;51(1):83-98.

medi travel[®] men

Compression garments on the journey can reduce the risk of traveller's thrombosis.*

The lack of physical exercise during a journey exerts constant high pressure on the venous system. The venous valves no longer close perfectly and the blood circulation slows down. The legs become heavy and tired – in the worst case, traveller's thrombosis may develop. medi travel socks have been specially developed to keep the veins in shape during long periods of sitting or standing. They support the natural blood circulation, so the blood flows back to the heart against gravity instead of pooling in the legs.





medi travel men Black



medi travel men Anthracite


* Hsieh HF, Lee FP. Graduated compression stockings as prophylaxis for flight-related venous thrombosis: systematic literature review. J Adv Nurs 2005;51(1):83-98.


medi travel men		
Styles	AD	AD pte.
		
	calf	calf petite
Article No.	Black / T140512-6	Black / T147512-6
Article No.	Anthracite / T140A12-6	Anthracite / T147A12-6

Standard styles	
Length	2 lengths: 43 – 49 cm, 37 – 42 cm petite
Foot part	Closed toe
Shoe size	38 – 48
Standard sizes	S – XXL


Measuring chart		
Size	Circumference in cm	Circumference in inches
S	18 – 20	7 ¼ – 8
M	20 – 23	8 – 9
L	23 – 26	9 – 10 ¼
XL	26 – 29	10 ¼ – 11 ½
XXL	29 – 32	11 ½ – 12 ½
ankle – b		

Material composition	
Elastane (EL)	Polyamide (PA)
10 %	90 %

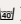Washing instructions
Wash-and-wear     
Suitable for washing machine and drier

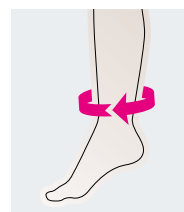

Ankle circumference

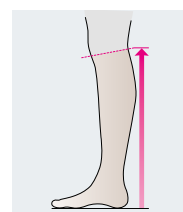

Length



medi travel[®] women

Compression garments on the journey can reduce the risk of traveller's thrombosis.*

Your benefits

- Increase your cross-selling by offering your customer a travel sock for prevention
- Complete your product range for the prevention of traveller's thrombosis and gain new customers
- Offer a product that has been specially designed to match the female target group

Benefits for your customers

- Graduated compression can reduce the risk of thrombosis during travelling*
- High medical effect for feeling secure
- medi travel women have a modern design and look just like semitransparent sheer stockings

Additional benefits

- The skin feels comfortable all year round with Clima Comfort (tested by IFTH)
- Clima Fresh with antibacterial effects prevents odour formation
- Extremely skin-friendly – tested quality to Oeko-Tex Standard 100

Product benefits



Clima Comfort



Clima Fresh



Perfect Fit



medi travel complies with all applicable EU Directives. All the prescribed conformity assessment procedures have been completed.

Colours

The article is available in the following colours.

S = Standard

Black
S

Caramel
S

* Hsieh HF, Lee FP. Graduated compression stockings as prophylaxis for flight-related venous thrombosis: systematic literature review. J Adv Nurs 2005;51(1):83-98.

medi travel[®]

women

Compression garments on the journey can reduce the risk of traveller's thrombosis.*

The lack of physical exercise during a journey exerts constant high pressure on the venous system. The venous valves no longer close perfectly and the blood circulation slows down. The legs become heavy and tired – in the worst case, traveller's thrombosis may develop. medi travel socks have been specially developed to keep the veins in shape during long periods of sitting or standing. They support the natural blood circulation, so the blood flows back to the heart against gravity instead of pooling in the legs.





medi travel women Black



medi travel women Caramel

* Hsieh HF, Lee FP. Graduated compression stockings as prophylaxis for flight-related venous thrombosis: systematic literature review. J Adv Nurs 2005;51(1):83-98.

medi travel women		
Styles	AD	AD pte.
		
	calf	calf petite
Article No.	Black / T140522-6	Black / T147522-6
Article No.	Caramel / T140122-6	Caramel / T147122-6

Standard styles

Length	2 lengths: 38 – 44 cm; 34 – 38 cm petite
Foot part	Closed toe
Shoe size	36 – 43
Standard sizes	S – XXL




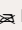
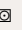
Measuring chart


Size	Circumference in cm	Circumference in inches
S	18 – 20	7 1/4 – 8
M	20 – 23	8 – 9
L	23 – 26	9 – 10 1/4
XL	26 – 29	10 1/4 – 11 1/2
XXL	29 – 32	11 1/2 – 12 1/2
ankle – b		

Material composition


Elastane (EL)	Polyamide (PA)
25 %	75 %

Washing instructions

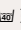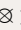Wash-and-wear     
Suitable for washing machine and drier

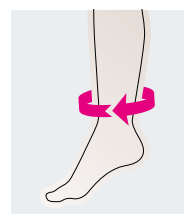

Ankle
circumference

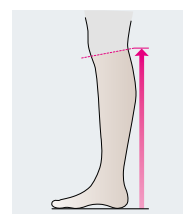

Length





Commercial products

Sometimes it is difficult for patients to put on their compression stockings. medi offers a large selection of practical donning and doffing aids. For patients with sensitive skin there is also a special skin care series by medi. The rich ingredients pamper the skin and help it regenerate.

medi

Donning and doffing aids

medi Butler

Your benefits:

- Nine Butlers: the right solution for several customer requirements
- Increase add-on sales
- Time saving with uncomplicated donning and doffing

Benefits for your customers:


- Quick and simple donning of compression stockings
- Less effort required
- Wide choice for the most individual demands

medi Export Butler with long handles

The ideal donning aid for all compression stockings wearers who can comfortably don their stockings without assistance. The long handle ensures maximum stability when putting on compression stockings and pantyhose.

The handle length is 32 cm. The stirrup width is 11 cm.

Article No. 6900041000

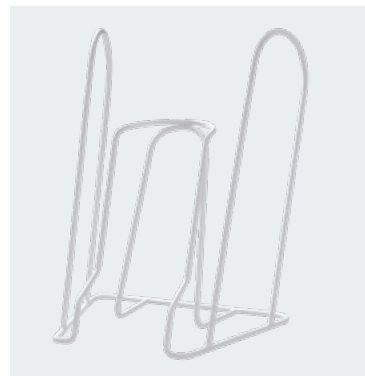

medi Butler with long handles

The handle length is 35 cm. The stirrup width is 11 cm.

Article No. 6900052000




medi Butler with variable handles

The donning aid for stocking wearers with considerably restricted freedom of movement.

The handle length can be adjusted between 37 and 58 cm.

The stirrup width is 11 cm.

Article No. 6900051000

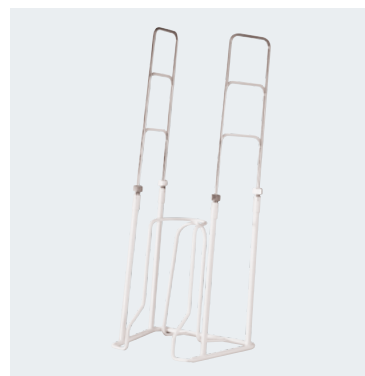

medi Big (XXL) Butler

The stirrup of the medi Big Butler has a larger diameter. This makes allowance for larger circumferences which are often associated with less freedom of movement. The handle length is 44 cm. The stirrup width is 15 cm.

Article No. 6900102000



medi Butler for pantyhose

The medi pantyhose Butler eases donning of compression pantyhose. The handle length is 48 cm. The stirrup width is 11 cm.

Article No. 6900055000



medi Hospitalbutler

Adjustable diameter with hard paint finish coating. The stirrup width is 11cm.

Article No. 6900062000



medi Arm Butler

The medi Arm Butler eases donning of compression armsleeves.

Article No. 6900057000



medi 2in1

- Your compression stocking glides up over your heel effortlessly
- For all garment lengths
- For open and closed toes
- Hard-wearing and tear-resistant
- Light and folds up – very compact and portable


Article No.	Description
6900013000	medi 2in1



medi slide

- Smooth material with a special coating for an easy handling
- Minimize physical strain and enhance conveniences
- Is suitable for stockings with open toe
- Reduces friction


Article No.	Description
6903012000	Small < 41 cm (size of shoe)
6903014000	Large > 41 cm (size of shoe)

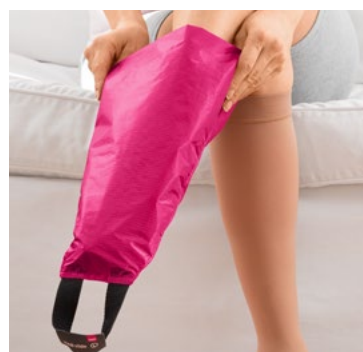

medi Butler Off

The medi Butler Off helps you take off your compression stocking without tiresome bending and without having to pull hard. To take the stocking off, simply slide the medi Butler Off into the compression stocking and push it off downwards over the heel. This avoids damaging the skin and the knit of the compression stocking (not recommended for mediven forte and flat-knitted stockings).

Article No.	Description
6900362000	medi Butler Off

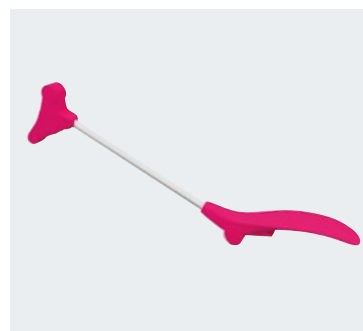

Dressing aids

Easy-on-slipper, one size fits all.

Article No.	Description
6900022000	Dressing aids (unit with 40 pieces)



Easy-on-slipper

Rubber gloves

- Sizes: small, medium, large and x-large
- Material: 100 % Latex

Article No.	Description
6901022000	rubber gloves S 6,5-7 1 pair
6901023000	rubber gloves M 7,5-8 1 pair
6901024000	rubber gloves L 8,5-9 1 pair
6901025000	rubber gloves XL 9,5-10 1 pair
6901122000	rubber gloves S 6,5-7 12 pairs / carton
6901123000	rubber gloves M 7,5-8 12 pairs / carton
6901124000	rubber gloves L 8,5-9 12 pairs / carton
6901125000	rubber gloves XL 9,5-10 12 pairs / carton



Rubber gloves

medi textile gloves latex free

- These gloves facilitate putting on, arranging and positioning compression garments on legs and arms.
- Our special coating makes it easy to get a hold on stockings.
- Suitable for use for persons who are allergic or intolerant to latex or natural rubber.
- Material: 55% polyester, 45% nitrile

Article No.	Description
6911102000	medi textile gloves latex free small 1 pair
6911103000	medi textile gloves latex free medium 1 pair
6911104000	medi textile gloves latex free large 1 pair
6911112000	medi textile gloves latex free small 12 pairs / carton
6911113000	medi textile gloves latex free medium 12 pairs / carton
6911114000	medi textile gloves latex free large 12 pairs / carton



medi textile gloves latex free

medi textile gloves with latex

- These gloves facilitate putting on, arranging and positioning compression garments on legs and arms.
- Material: 40% polyester, 60% latex

Article No.	Description
6903022000	medi textile gloves small 1 pair
6903023000	medi textile gloves medium 1 pair
6903024000	medi textile gloves large 1 pair
6903122000	medi textile gloves small 12 pairs / carton
6903123000	medi textile gloves medium 12 pairs / carton
6903124000	medi textile gloves large 12 pairs / carton



medi textile gloves



medi skin care

Skin care classics from medi
medi day, medi night, medi fresh and medi soft.

Your benefits

- Increase add-on sales / present as a full-line provider
 - Confidence that the ingredients are perfectly matched to the compression garments
 - Increase customer satisfaction by supplying ideal care for vein-stressed skin
 - Decreases wear and tear on the compression garments by encouraging smooth soft skin
-

Properties

Compression garments are the central component of venous therapy. Skin care is an important complement to compression garments.

An important tip for patients with allergies

All products meet the highest demands and are produced free of parabens (preservatives) and wool fat alcohols, and are latex-free.

The medi care products do not have a use-by or expiry date. They are marked with an "opened tub" and a month. This means that as soon as the tube has been opened and the contents are being used, the product should be used within 6, 9, or 12 months, as with shower gels, shampoos, etc.

medi skin care

The development of a venous disease increases the burden on the skin. The stretched wall of the vein leaks fluids into the surrounding tissue. The increased loss of water from the surface of the skin makes it more prone to injury. In order to enhance the success of compression therapy, the skin care products have been developed to meet the specific requirements of an affected leg.

medi day – cooling gel with horse chestnut

Benefits

- Refreshing and cooling effect
- Well-nourished, smooth skin
- Reduces the feeling of tightness
- Supports user comfort of medical compression stockings

Use medi day directly before putting on the compression garments and let the day begin!

Article No.	Description
P400025000	medi day with 50 ml (unit with 6 pieces)
P400026000	medi day with 150 ml (unit with 6 pieces)
P400027000	medi day with 500 ml (1 piece)



medi day



medi night – skincare cream with regenerative witch hazel

Benefits

- Soothes the skin and also supports regeneration

Apply medi night after putting on the compression stockings – for a restful night.

Article No.	Description
P500025000	medi night with 50 ml (unit with 6 pieces)
P500026000	medi night with 150 ml (unit with 6 pieces)



medi night



*You can find out more information about the test carried out at www.dermatest-guarantee.de

**medi fresh –
refreshing spray with menthol**

Benefits

- Feel refreshed straight away
- Can be used several times a day
- Gentle to the stocking fabric
- Easy to use everyday

Spray medi fresh any time on the stockings and enjoy its soothing effect.

100 ml bottle

Article No.	Description
P900026000	medi fresh with 100 ml (unit with 6 pieces)

**medi soft –
gentle cream foam**

Benefits

- Well-nourished, smooth skin
- Reduces the feeling of tightness
- Supports user comfort of medical compression stockings
- Provides additional support for fabrics and adhesive tapes

Treat your skin in the mornings and evenings to some extra moisture.

125 ml

Article No.	Description
P800006000	medi soft 125 ml (unit with 6 pieces)



medi fresh



medi soft



*You can find out more information about the test carried out at www.dermatest-guarantee.de

medi textile care

Gentle care for hand washing your garments.

medi clean

For the easy care of your compression garments – anytime, anywhere.

The special washing liquid is formulated to gently clean your compression stockings.

200 ml tube

Article No.	Description
P920026000	medi clean with 200 ml (unit with 6 pieces)



medi clean

medi spot ex

The clever solution for stain-free compression garments.

Use before washing, the spot removal stick medi spot ex removes shoe marks from the toes or the heel of the fabric.

15 ml roller

Article No.	Description
P930025000	medi spot ex with 15 ml (unit with 6 pieces)



medi spot ex

*You can find out more information about the test carried out at www.dermatest-guarantee.de

medi fix

Adhesive lotion for compression stockings and sleeves with tricot border for a secure fit.

50 ml Soft-Roller

Article No.	Description
1023025000	medi fix with 50 ml (unit with 6 pieces)



medi fix



medi laundry net


Gentle washing

Gentle care helps compression stockings maintain their comfort and their medical efficacy. The medi laundry net is the ideal aid for caring for compression stockings. The net structure protects the fabric against damage during the daily wash, but does not hinder the removal of dirt particles or soap residues.

Article No.	Description
1020585000	medi laundry net (unit with 15 pieces)



medi laundry net

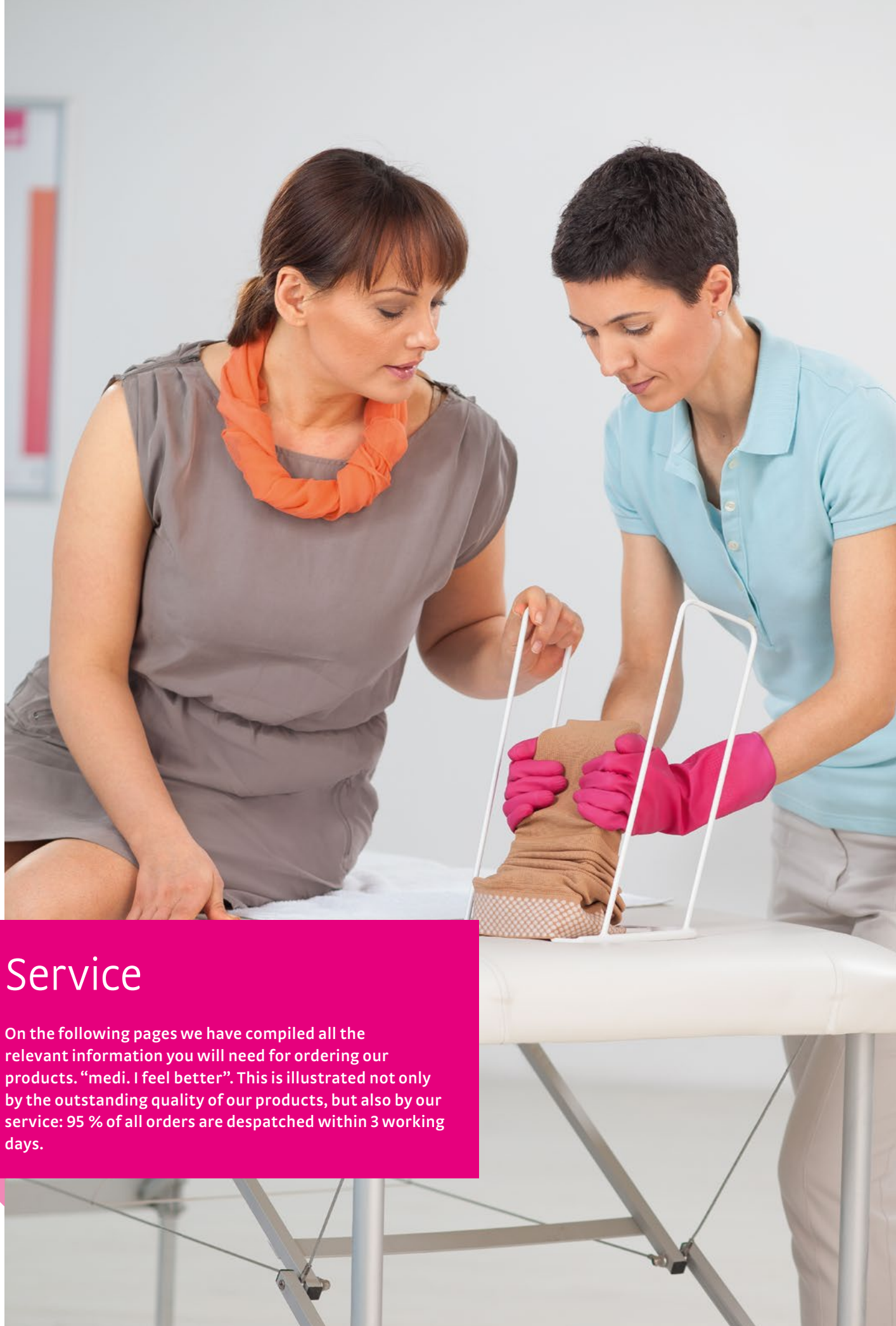

Service

On the following pages we have compiled all the relevant information you will need for ordering our products. “medi. I feel better”. This is illustrated not only by the outstanding quality of our products, but also by our service: 95 % of all orders are despatched within 3 working days.



mediven® overview

	mediven sheer & soft	mediven elegance	mediven for men	mediven comfort	mediven plus	mediven active	mediven forte	Duomed
	Translucent	Highly fashionable	Elegant	Comfortable	All-rounder	Hard wearing	Robust	Economical
CCL	15 – 20 mmHg, 20 – 30 mmHg	1, 2	1, 2	1, 2	1, 2, 3	1,2	2, 3	1, 2
Length (regular and petite)	calf, thigh, panty	calf, thigh, panty maternity panty	calf	calf, thigh, panty	calf, thigh, panty, maxi panty top, maternity panty, men's leotard	calf	calf, thigh, panty, maternity panty, men's leotard	calf, thigh, panty
Styles available	standard only	standard and made-to- measure	standard only	standard and made-to- measure	standard and made-to- measure	standard only	standard and made-to- measure	standard only
Specials		yes	no	yes	yes	no	yes	no
Toe	open and closed	open and closed	closed	open and closed	open and closed	closed	open and closed	open and closed
Colours	Ebony	Black	Black	Black	Black	Black	Black	Beige
	Natural	Anthracite	Anthracite	Anthracite	Anthracite	Anthracite	Anthracite	Sand
		Navy	Navy	Navy	Navy	Navy	Navy	Caramel
		Cashmere	Brown	Cashmere	Cashmere	Brown	Cashmere	Black
		Caramel		Caramel	Caramel	White	Caramel	
		Rosé		Rosé*	Rosé		Rosé*	
		Bronze		Bronze*	Bronze		Bronze*	
		Beige		Beige*	Beige		Beige*	
		medi Magenta*			medi Magenta*			
		White*			White*			

*Delivery period up to 10 working days.

	medi travel men	medi travel women
Length (regular and petite)	calf	calf
Styles available	standard only	standard only
Toe	closed	closed
Colours	Black	Black
	Anthracite	Caramel

mediven elegance®, mediven® for men, mediven active®,
mediven® comfort, mediven plus®


Circumferences							
Sizes	I	II	III	IV	V	VI	VII
cG thigh with silicone topband	43 – 48	45 – 52	49 – 56	53 – 60	56 – 64	60 – 68	64 – 72
cG thigh with silicone topband extra wide	49 – 57	53 – 62	57 – 67	61 – 72	65 – 77	69 – 81	73 – 85
cG	43 – 57	45 – 62	49 – 67	53 – 72	56 – 77	60 – 81	64 – 85
cF	39 – 52	41 – 56	44 – 60	47 – 65	50 – 69	53 – 73	57 – 78
cE	30 – 37	33 – 40	35 – 43	37 – 45	39 – 48	41 – 51	42 – 53
cD	27 – 33	29 – 36	32 – 39	34 – 42	36 – 45	38 – 48	40 – 50
cC	28 – 34	30 – 37	33 – 40	35 – 43	37 – 46	39 – 49	41 – 51
cB ₁	23 – 27	24 – 29	26 – 32	29 – 35	31 – 37	33 – 39	35 – 41
cB	18 – 20	20 – 22	22 – 24	24 – 26	26 – 28	28 – 30	30 – 32
cY	26 – 31	28 – 33	29 – 35	31 – 37	32 – 38	33 – 40	34 – 42
cA	17 – 22	19 – 24	21 – 26	23 – 29	25 – 32	27 – 34	28 – 35

Length	
ℓG regular	72 – 83
ℓG petite	62 – 71
ℓD regular	39 – 44
ℓD petite	34 – 38

Length mediven for men	
ℓD regular	39 – 44
ℓD petite	34 – 38
Length mediven active	
ℓD regular	43 – 49
ℓD petite	37 – 42

mediven® forte

Circumferences							
Sizes	I	II	III	IV	V	VI	VII
cG thigh with silicone topband	44 – 47	46 – 50	50 – 54	53 – 58	56 – 61	60 – 65	64 – 70
cG thigh with silicone topband extra wide	48 – 53	51 – 57	55 – 61	59 – 65	62 – 70	66 – 74	71 – 80
cG	44 – 53	46 – 57	50 – 61	53 – 65	56 – 70	60 – 74	64 – 80
cF	40 – 47	42 – 51	45 – 55	48 – 58	51 – 62	54 – 67	58 – 72
cE	30 – 37	33 – 40	36 – 42	38 – 44	40 – 46	42 – 48	44 – 50
cD	28 – 33	30 – 35	32 – 37	34 – 39	36 – 41	38 – 43	40 – 45
cC	29 – 34	31 – 36	33 – 38	35 – 40	37 – 42	39 – 44	41 – 46
cB ₁	24 – 26	26 – 28	28 – 31	30 – 33	32 – 35	34 – 37	36 – 39
cB	18 – 20	20 – 22	22 – 24	24 – 26	26 – 28	28 – 30	30 – 32
cY	28 – 30	29 – 32	31 – 34	33 – 36	34 – 38	36 – 40	37 – 41
cA	18 – 21	20 – 23	22 – 25	24 – 27	26 – 29	28 – 31	30 – 33

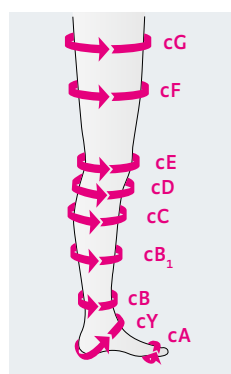

Length	
ℓG regular	72 – 83
ℓG petite	62 – 71
ℓD regular	38 – 43
ℓD petite	33 – 37

Foot	
Two foot sizes available:	
Short foot sizes 36 – 38	
Normal foot sizes 38 – 43	


Duomed

Sizes	cB Ankle measurement	cC Calf measurement	cG Thigh measurement	Measurements for silicone topband
S	19 – 21	28 – 34	42 – 57	47 – 54
M	22 – 24	32 – 38	48 – 64	52 – 60
L	25 – 27	36 – 42	54 – 71	56 – 65
XL	28 – 30	40 – 46	60 – 78	62 – 72
XXL	30 – 32	42 – 50	65 – 85	65 – 76

Compression Classes	Indications
1 mild compression	For slight varicosis without any pronounced tendency to oedema and for early varicosis in pregnancy. With special reasons, e.g. loss of strength in the hands, arthrosis, rheumatism, diabetes. Compression in mmHg: 18.0 – 21.0 Compression in kPa: 2.4 – 2.8
2 moderate compression	For more serious symptoms, pronounced varicosis with a tendency to oedema, post-traumatic swelling, after resolution of simple ulcers, after superficial thrombophlebitis, after sclerosis and surgery of varicoses to stabilise successful treatment and for more pronounced varicosis in pregnancy. Compression in mmHg: 23.0 – 32.0 Compression in kPa: 3.1 – 4.3
3 strong compression	For all complications following constitutional and post-thrombotic venous insufficiency, pronounced tendency to oedema, secondary varicosis, atrophie blanche, dermatosclerosis, and after resolution of serious, in particular recurrent, ulcers. Compression in mmHg: 34.0 – 46.0 Compression in kPa: 4.5 – 6.1

Compression class (CCL)	Pressure (mmHg)
1	18 – 21
2	23 – 32
3	34 – 46
4	49 and above

This table represents the RAL-GZG standard. There also exist other standards (e. g. France : AFNOR, Great Britain : BSI). The selection of a particular compression class is at the discretion of the doctor.

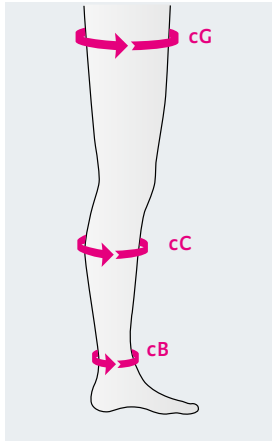


Length

ℓG regular	72 – 83
ℓG petite	62 – 71
ℓD regular	39 – 44
ℓD petite	34 – 38

Toe

For all closed toe stockings:
only one highly elastic foot size

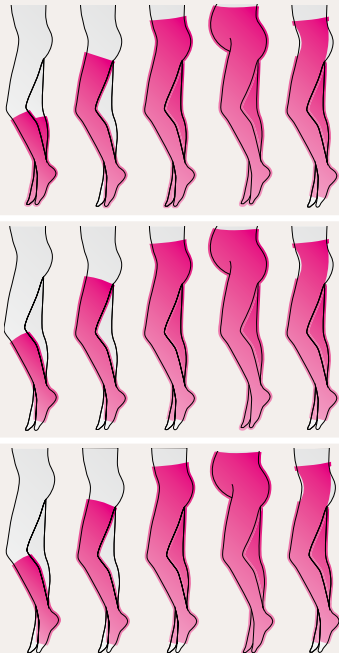

Contraindications


According to the guidelines of the German Society for Phlebology:

- Absolute contraindications: advanced peripheral arterial occlusive diseases, decompensated heart failure, septic phlebitis, phlegmasia coerulea dolens.
- Relative contraindications: weeping dermatoses, intolerability of compression stocking material, sensory disturbances of the extremities, advanced peripheral neuropathy (e. g. diabetes mellitus), primary chronic polyarthritis.

medi assumes no liability as a result of any contraindicated use of this product.

Correct fitting of round knit medical compression stockings


Note: please take the measurements on the inside of the leg except for ℓG

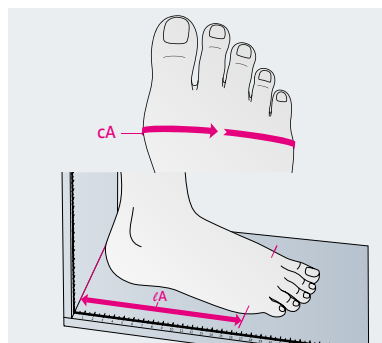

cA, lA, lZ


Take circumferential measurement **cA** just behind the joint at the base of the toes where the little toe begins.

The length of the foot for open toe stockings is determined by the length measurement **lA** which is measured from the ridge of the heel to measurement point **cA**. **The length of the foot for closed toe stockings** is measured from the ridge of the foot to the tips of the toes. (**lZ-measurement**)

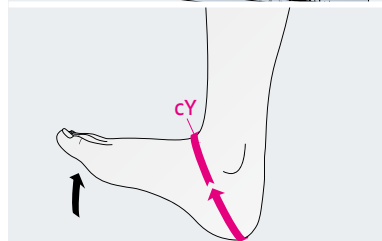


cY

Take the circumferential measurement **cY**, around the heel and the instep, with the foot in maximum dorsal extension.

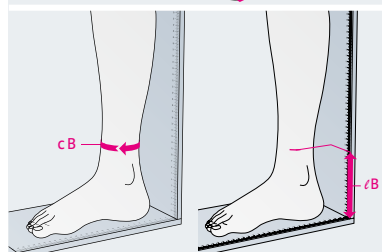


cB, lB

Take the circumferential measurement **cB** and length measurement **lB** at the narrowest circumference of the leg below the knee, directly above the ankle. This measurement above the ankle bones is especially important for selecting the stocking size, because the pressure here will be 100%.

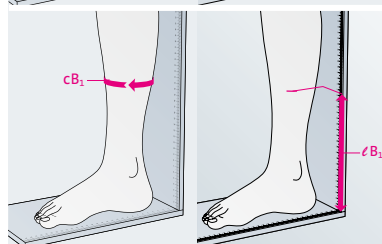


cB¹, lB¹

The circumferential measurement **cB¹** and the length measurement **lB¹** are at the point where the calf begins, or at the transition point to the Achilles tendon.

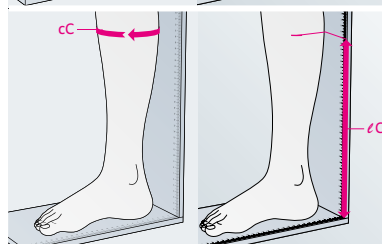


cC, lC

Take the **cC** and **lC** length measurements at the widest circumference of the calf.

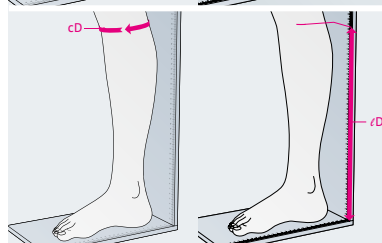


cD, lD

Take the circumference **cD** and length **lD** measurements about two fingerbreadths below the hollow of the knee at the level of the head of the fibula.

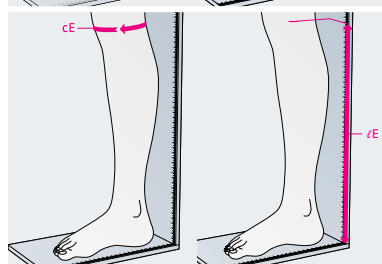


cE, lE

The circumferential measurement **cE** is the circumference of the knee measured through the hollow of the knee in the middle of the kneecap. Take the length measurement **lE** when the leg is straight.

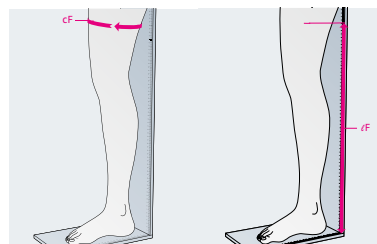


cF, lF

Circumferential measurement **cF** and length measurement **lF** are located where the inner thigh muscle begins or in the middle of the thigh.

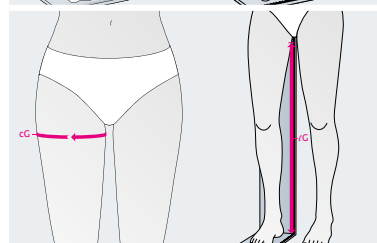


cG, lG

Take the **cG** and length measurement **lG** about 3 cm below the crotch.

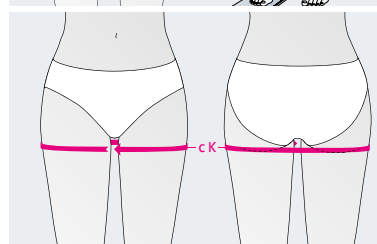


cK

Take the circumferential measurement **cK** at the height of the g measurement, around both legs.

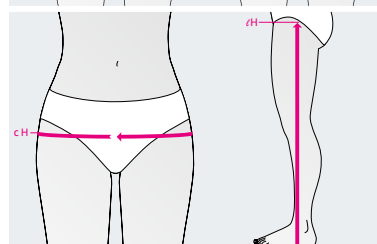


cH, lH

Take the circumferential measurement **cH** at hip level at the widest point of the buttocks.

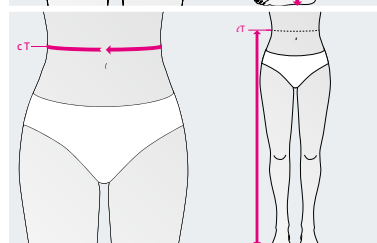


cT, lT

Take the circumferential measurement **cT** round the waist.

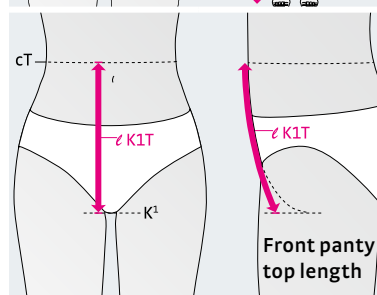

Front panty top length
Measure from the crotch to the waist. It is however more comfortable for the patient if you go 3cm to the side of the crotch and measure the panty top length from there.

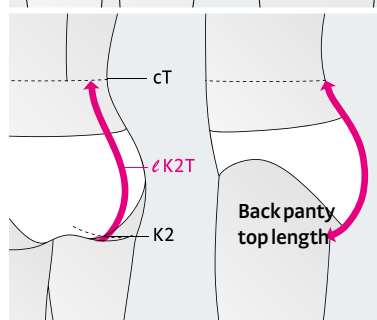

Back panty top length
Measure from the gluteal fold to the waist. If the patient requests a longer or shorter panty top, you must measure to the length the patient requests. The "waist" point is only the standard length.

Round knit – Order for round knit Made-to-measure compression stockings / pantyhose

medi

Fax order to medi · Telefax +49 921 912-781

Patient Name

Date

Referee Name

Quantity ☐ piece(s) ☐ pair(s) ☐ pants

Delivery Address

Foot Requirements

- ☐ Foot length open toe (ℓA)
(heel to ball eA): _____ cm
- ☐ Foot length closed toe (ℓZ)
(heel to toe): _____ cm
- ☐ Soft toe

Colour

Topbands mediven elegance, mediven comfort, mediven plus and mediven forte

- ☐ Plain silicone topband rose (FBR)
- ☐ Silicone dot topband (NoB)
- ☐ AD: Narrow silicone dot topband
(sNoB) 2,5 cm
- ☐ Silicone dot topband motif
(NoM)
- ☐ Silicone micro dot topband
sensitive (MBs)

Waist attachment

only plus and forte

- ☐ left ☐ right
Heel ℓT: _____ cm

Waist
circumference eT: _____ cm

Special requests

Circumferences left leg / cm		Circumferences right leg / cm	Length panty section cm	
			please specify length of panty section	
			front	
			back	
			eT	ℓT
			eH	ℓH
			Lengths leg	
			LT	RT
eG		eG	ℓG	
eF		eF	ℓF	
eE		eE	ℓE	
eD		eD	ℓD	
eC		eC	ℓC	
eB ¹		eB ¹	ℓB ¹	
eB		eB	ℓB	
eY		eY		
eA		eA		
ℓA		ℓA	ℓA	
ℓZ		ℓZ	ℓZ	

Models	CCL			Lengths					
	1	2	3	AD	AF	AG	AT	AT / U	AT / H
Please mark with a cross:									
mediven elegance									
mediven comfort									
mediven plus									
mediven forte									

All measurements are body measurements! The person placing the order is liable for incorrect measurements. Made-to-measure garments cannot be returned.
All made-to-measure orders must be placed in writing.

medi. I feel better.



Definition of article numbers phlebology

Please choose your size on the last (7th) digit of the article number from 2 to 6 (travel men, travel women, Duomed) or from 1 to 7 (all other qualities). Please choose open or closed toe (6th digit: 0 = open toe, 1 = closed toe) for all qualities except mediven plus, mediven for men, mediven active, medi travel men / women. Please choose your colour (5th digit) instead of x.

mediven sheer & soft

0 = Natural

5 = Ebony

mediven elegance

C = Cashmere

K = Rosé*

1 = Caramel

Z = Bronze*

0 = Beige

3 = Navy

A = Anthracite

5 = Black

M = medi Magenta

7 = White

* Please choose open or closed toe (6th digit: 7= open toe, 8 = closed toe).

mediven for men / mediven active

3 = Navy

6 = Brown

A = Anthracite

5 = Black

7 = White (only mediven active)

mediven comfort

C = Cashmere

K = Rosé

1 = Caramel

Z = Bronze

0 = Beige

3 = Navy

A = Anthracite

5 = Black

mediven plus

C = Cashmere

K = Rosé

1 = Caramel

Z = Bronze

0 = Beige

3 = Navy

A = Anthracite

5 = Black

M = medi Magenta

7 = White

mediven forte

C = Cashmere

K = Rosé

1 = Caramel

X = Bronze

0 = Beige

3 = Navy

Z = Anthracite

5 = Black

Duomed

0 = Beige

5 = Black

medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth / Germany
T +49 921 912-0
F +49 921 912-781
export@medi.de
www.medi.de/en

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61 2 9890 8696
F +61 2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43-512 57 95 15
F +43-512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T: +32-11 24 25 60
F: +32-11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Cristovan de Vita, n.º 260, Un. 30,
Centro Logístico Raposo Tavares,
Vila Camargo
Vargem Grande Paulista - SP
CEP 06730-000
Brasil
T: +55-11-3500 8005
sac@medibrasil.com
www.medibrasil.com

medi Canada Inc / médi Canada Inc
597, Rue Duvernay, Verchères
QC Canada J0L 2R0
T: +1 450-583-3317 / +1 800 361 3153
F: +1 888-583-6827
service@medicanada.ca
www.medicanada.ca

MAXIS a.s.,
medi group company
Slezská 2127/13
120 00 Prague 2
Czech Republic
T: +420 571 633 510
F: +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Italia s.r.l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T: +39-051 6 13 24 84
F: +39-051 6 13 29 56
info@medi-italia.it
www.medi-italia.it

Tachezy Sanit Srl.
Via Druso 48/a
39100 Bolzano
Italy
T: +39 0471 288 068
F: +39 0471 282 773
info@tachezysanit.com
www.tachezysanit.com

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
Vestheimvegen 35
4250 Kopervik
Norway
T: +47 52 84 45 00
F: +47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi Polska Sp. z o.o.
ul. Łabędzka 22
44-121 Gliwice
Poland
T: +48-32 230 60 21
F: +48-32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal Lda.
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova Street 17
117342 Moscow
Russia
T +7-495 374 04 56
F +7-495 374 04 56
info@medirus.ru
www.medirus.ru

medi Trading (Shanghai) Co. Ltd.
Room 105, Building No. 4,
Lane 500 ShengXia Road
Pudong New District
201210 Shanghai
People's Republic of China
T: +86-21 61761988
F: +86-21 61769607
infocn@medi.cn

medi Sweden AB
Box 6034
192 06 Sollentuna
Sweden
T +46 8 96 97 98
F +46 8 626 68 70
info@medi.se
www.medi.se

Medi Turk Ortopedi Medikal İc ve Dis Tic. AS
Kültür mah. çaldıran sok.14/3
06420 Çankaya/Ankara
Turkey
T +90 312 435 20 26
F +90 312 434 22 82
info@medi-turk.com
www.medi-turk.com

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Evhen Sverstiuka str, 11.
Kiev 02002
Ukraine
T: +380 44 591 11 63
F: +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1-336 4 49 44 40
F +1-888 5 70 45 54
info@mediusa.com
www.mediusa.com

Headquarters
medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T + 49 921 912-0
F + 49 921 912-781
export@medi.de
www.medi.de/en



4 049772 376698

0097067/08.2019