



medi

# circaid® sizing guide

## Adjustable compression devices


Lymphoedema therapy  
circaid® reduction kit / circaid® juxtafit®

Wound care  
circaid® juxtacures® / circaid® juxtalite®

Vein therapy  
circaid® juxtalite®



medi. I feel better.

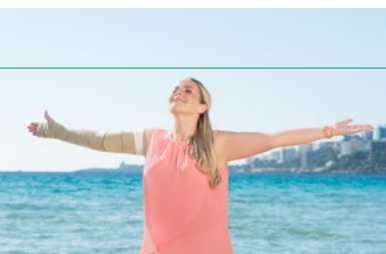



# Driving the future in compression therapy

**A strong brand. A wide-ranging product range.  
Convincing concepts.**

---

medi World of Compression: signpost to the future with more than 65 years' competence in the field of compression. Your partner for the highest demands on quality, innovation and design. Let's shape the future together and create positive brand experiences – for you and your clients!

# circaid® features



Built-In-Pressure

## Built-In-Pressure® system

To set and control the correct therapeutic compression range



Breathe-O-Prene

## Breathe-O-Prene

- Breathable and latex-free material
- Odour-inhibiting with antibacterial effect



Juxtaposed band

## Juxtaposed band system

- Optimum fit thanks to interlocking bands
- Each strap can be readjusted while on the limb



Variety

## Variety

- Wide range of care options
- Many sizes available



Easy handling

## Easy handling

- Simple donning and doffing
- Eases handling for the patient



Inelastic

## Inelastic

- Inelastic material for an effective therapy and pleasant wearing comfort
- Inspired by nature



Built-In-Tension

## Built-In-Tension System

- To set and control the correct therapeutic level of tension
- For circaid reduction kit products only



**Guaranteed  
hypoallergenic and  
latex-free**





## circaid® Built-In-Pressure® system (BPS)

With the BPS patients are able to set their prescribed compression range while donning their garment and to readjust the compression during the day as needed to maintain. Therefore, the appropriate compression range is ensured throughout the period of wear.


# medi Therapy Concept Lymphology

Effective therapy chain for treating lymphoedema.

---

## Note

Detailed information on the therapy concept lymphology and more product information please see our product catalogue lymphology (Art.No. 0097D84)

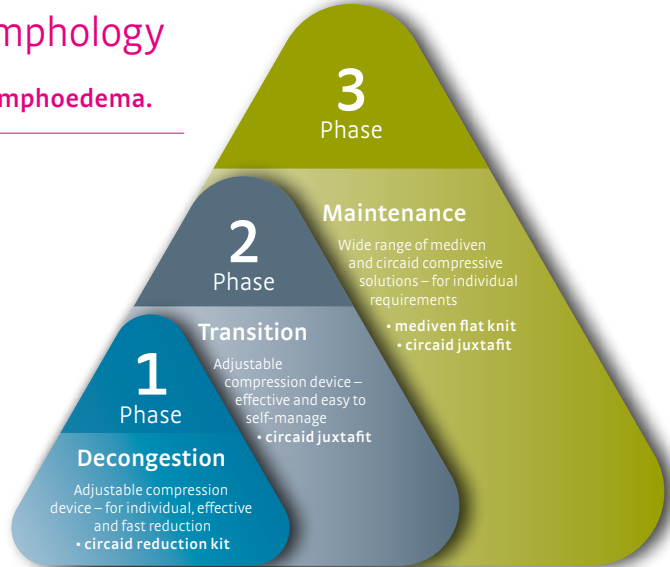

1

## Decongestion Phase



Great reduction results during the decongestion phase.

- circaid® reduction kit

2

## Transition Phase



Effective compression device throughout the transition phase.

- circaid® juxtafit®

3

## Maintenance Phase



Compression devices for maintaining best therapy results.

- mediven® flat knit
- circaid® juxtafit®

1

## circaid® reduction kit



### circaid® reduction kit leg


- Built-In-Tension System (BTS) to set and control the correct therapeutic tension level
- Adjustable sizes - garment is initially fitted to the patient's individual circumferences and limb shape
- Ability to reduce size of garment as the limb volume decreases
- Each band can be individually readjusted while on the limb
- Latex free and odour-inhibiting material with antibacterial effect





# Sizing chart

## circaid® reduction kit lower leg

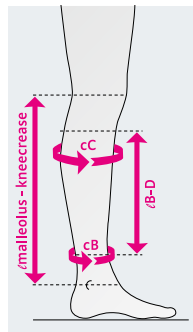


Circumferences in cm	standard	wide	
cC	< 65	< 85	
cB	< 55	< 65	
Actual garment length in cm	short	standard	long
ℓ B - D	30	35	40
Length measurement in cm	short	standard	long
ℓ malleolus - knee crease	< 32	32 – 36	> 36

All products contain the Built-In-Tension System card (BTS).

# Sizing chart

## circaid® reduction kit knee

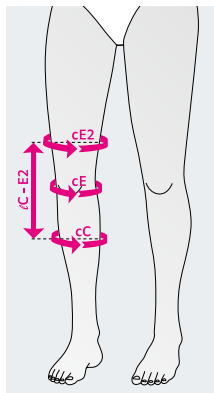


Circumferences in cm	standard
cE2*	< 110
cE	< 102
cC	< 80
Length in cm	standard
lC - E2	30

\* cE2 should not be more than 30 cm bigger than cC.  
cE2 is 10 cm above the patella

# Sizing chart

## circaid® reduction kit upper leg

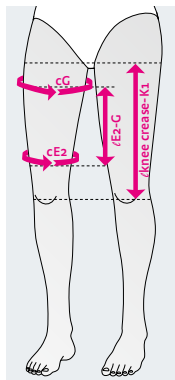

Circumferences in cm	standard	wide	
cG*	< 80	< 125	
cE2**	< 65	< 110	
Actual garment length in cm	short	standard	long
l E2 - G	30	35	40
Length measurement in cm	short	standard	long
l knee crease - K1	< 37	37 – 41	> 41

\* cG should not be 30 cm bigger than cE2

\*\* cE2 is 10 cm above the patella

All products contain the Built-In-Tension System card (BTS).

1

## circaid® reduction kit




### circaid® reduction kit arm

- Built-In-Tension System (BTS) to set and control the correct therapeutic tension level
- Adjustable sizes - garment is initially fitted to the patient's individual circumferences and limb shape
- Ability to reduce size of garment as the limb volume decreases
- Each band can be individually readjusted while on the limb
- Latex free and odour-inhibiting material with antibacterial effect



# Sizing chart

## circaid® reduction kit arm

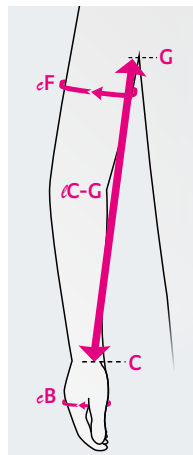

circaid  
reduction  
kit arm

Circumferences in cm	standard	wide	extrawide
cF	< 50	50 – 65	65 – 80
cB*	< 30	< 30	< 30
Length in cm	standard	long	
∠ C - G	< 45	> 45	

**All products contain the Built-In-Tension System card (BTS).**

\* If cB is > 30 cm, the enclosed circaid customizable hand wrap will not fit. But the circaid reduction kit arm garment may still be used. We recommend to use a circaid juxtafit made-to-measure hand option. For detailed information please see our product catalogue lymphology (Art.No. 0097D84).

### Note

Comes with one pair of undersleeves and a circaid customizable hand wrap.

2

3

## circaid® juxtafit® premium and essentials lower leg




### circaid® juxtafit® premium lower leg

- High quality band fasteners with reinforced landing straps
- BPS card to set and control the correct therapeutic compression range
- Also available as made-to-measure
- Twelve month lifespan



### circaid® juxtafit® essentials lower leg

- BPS card to set and control the correct therapeutic compression range
- Available in standard sizes only
- Six month lifespan

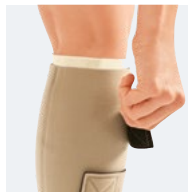


#### Note

Comes with a pac band, one pair of undersleeves and one pair of compression anklets.

# Sizing chart

circaid® juxtafit® premium and essentials lower leg

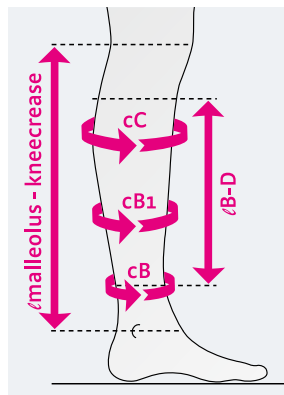

Circumferences in cm	S	M	M - X	L
cC	30 - 38	36 - 43	43 - 50	41 - 48
cB1	20 - 28	25 - 33	33 - 40	30 - 38
cB	15 - 23	20 - 28	20 - 28	25 - 33
	L - X	XL	XL - X	XXL
cC	48 - 56	41 - 51	51 - 61	53 - 64
cB1	38 - 46	33 - 41	41 - 51	43 - 53
cB	25 - 33	30 - 38	30 - 38	33 - 41

Actual garment length in cm	short	long
B - D	28	36
Sizing measurement in cm	short	long
malleolus - knee crease	< 36	> 36

All garments contain the circaid Built-In-Pressure System card (BPS).

2

3

## circaid® juxtafit® premium and essentials upper leg with knee



### **circaid® juxtafit® premium leg garments**

- High quality band fasteners with reinforced landing straps
- BPS card to set and control the correct therapeutic compression range
- Exclusively available as made-to-measure
- Twelve month lifespan

### **circaid® juxtafit® essentials upper leg with knee**

- BPS card to set and control the correct therapeutic compression range
- Available in in standard sizes as left or right limb option
- Six month lifespan


#### **Note**

We strongly recommend circaid juxtafit upper leg with knee in combination with circaid juxtafit lower leg and a circaid foot option.



# Sizing chart

## circaid® juxtafit® essentials upper leg with knee

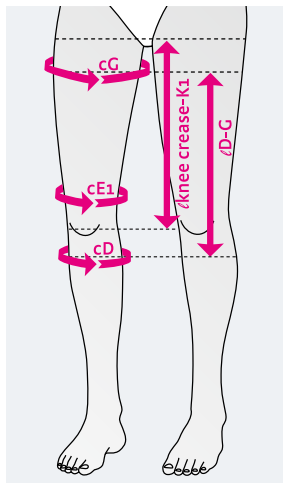

Circumferences in cm	XXS	XS	S	M	L	XL
cG	40 – 52	46 – 62	54 – 70	60 – 80	69 – 90	77 – 101
cE1*	32 – 42	37 – 48	42 – 55	50 – 65	58 – 76	66 – 87
cD	23 – 30	27 – 35	31 – 41	35 – 46	40 – 52	44 – 58
Actual garment length in cm		extra short	short	long		
l D – G			35**	45**	55**	
Sizing measurement in cm		extra short	short	long		
l knee crease – K1			< 25	25 – 35	> 35	

All garments contain the circaid Built-In-Pressure System card (BPS).

\* cE1 is 5 cm above the patella

\*\* Product length can be shortened by 5 cm

2

3

## circaid® juxtafit® essentials arm



### **circaid® juxtafit® essentials arm**


- BPS card to set and control the correct therapeutic compression range
- Available in standard sizes and as made-to-measure
- Six month lifespan

#### **Note**

We strongly recommend circaid juxtafit essentials arm in combination with a circaid juxtafit essentials hand wrap or made-to-measure circaid juxtafit essentials glove.

# Sizing chart

## circaid® juxtafit® essentials arm

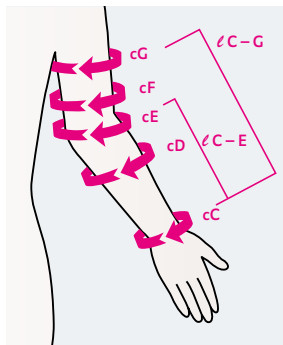

Circumferences in cm	1	2	3	4	5	6
cG	28–32	30–34	32–37	35–40	38–44	42–48
cF	26–30	28–32	30–35	33–38	36–41	39–45
cE	25–28	27–30	29–32	31–34	33–36	36–40
cD	23–26	25–28	27–30	29–32	31–34	33–37
cC	15–17	17–19	19–21	21–23	23–25	25–28
Actual garment length in cm		extra short	short	long		
l C–G			38	43	48	
Sizing measurement in cm		extra short	short	long		
l C–E			< 22	22–25	> 25	


All garments contain the circaid Built-In-Pressure System card (BPS).

# medi Wound Care

## Effective therapy chain for treating venous leg ulcer.

### Note

Detailed information on the medi wound care therapy chain and more product information please see our product catalogue wound care (Art.No. 0097D68)

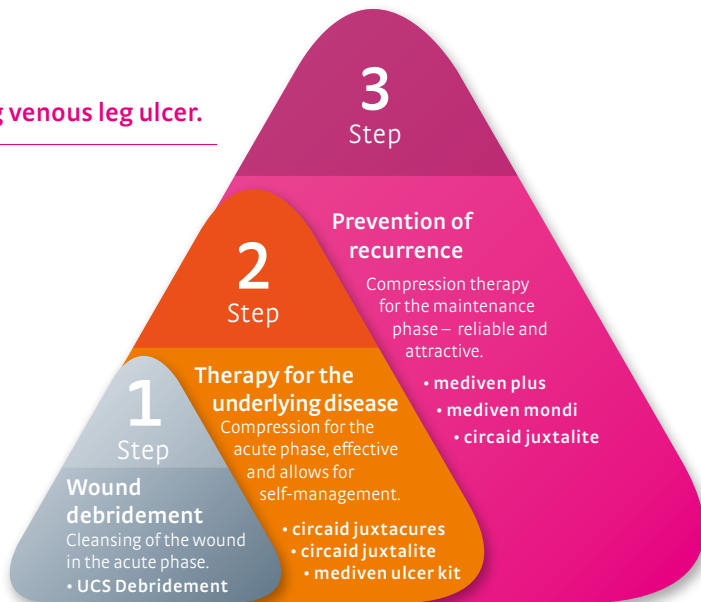

1

## Wound Debridement



Cleansing and care of the wound, wound edge and of the surrounding skin.

- UCS® Debridement

2

## Therapy for the underlying disease



Compression for treatment of chronic venous insufficiency.

- circaid® juxtacures®
- circaid® juxtalite®
- mediven® ulcer kit

3

## Prevention of recurrence



Compression to maintain treatment success.

- mediven® round knit
- mediven® flat knit
- circaid® juxtalite®



2

## circaid® juxtacures®




### circaid® juxtacures®

- BPS card to set and control the correct therapeutic compression range
- Customisable to the patient's individual circumferences and shape
- Adjustable to reducing leg circumferences during decongestion
- Six month lifespan

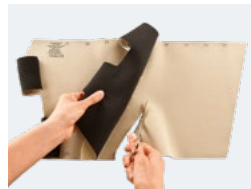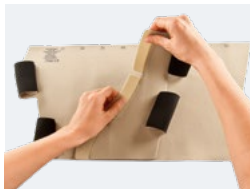

# Sizing chart

circaid® juxtacures®

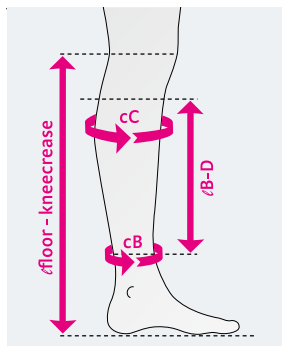

## Circumferences in cm

Adjustable to patient-individual circumferences.

## Minimum and maximum circumference measurements in cm

	minimum	maximum
cB	19	42
cC	25	64

## Actual garment length in cm

	short	standard	long
--	-------	----------	------

ℓ B - D	28	33	38
---------	----	----	----

## Sizing measurement in cm

	short	standard	long
--	-------	----------	------

ℓ floor - knee crease	< 44	44 - 49	> 49
-----------------------	------	---------	------

All garments contain the circaid Built-In-Pressure System card (BPS).

## Note

Comes with one pair of undersleeves and one pair of compression anklets.

2

3

## circaid® juxtalite® lower leg



### Note

circaid juxtalite is also applicable in the therapy for venous diseases. Offers an alternative for patients who have challenges with the proven medical compression stockings. For detailed information please see our product catalogue venous health (Art.No. 0097D67)

### circaid® juxtalite® lower leg

- BPS card to set and control the correct therapeutic pressure range
- Available in standard sizes
- Six month lifespan


### Note

Comes with an undersleeve and two compression anklets.



# Sizing chart

## circaid® juxtalite® lower leg

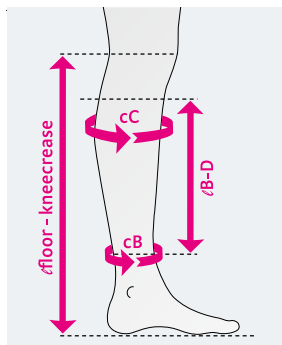

Circumferences in cm	S	M	M - X	L
cC	26 - 36	33 - 44	44 - 54	41 - 51
cB	19 - 22	22 - 27	22 - 27	25 - 32
	L - X	XL	XL - X	XXL
cC	51 - 61	37 - 48	48 - 61	53 - 64
cB	25 - 32	30 - 37	30 - 37	35 - 42
Actual garment length in cm	short	long		
ℓ B - D	28	33		
Sizing measurement in cm	short	long		
ℓ floor - knee crease	< 44	> 44		

All garments contain the circaid Built-In-Pressure System card (BPS).

## circaid® foot options



### **circaid® compression anklet**

- Provides compression to the foot and ankle
- Can be worn in combination with another foot option
- Available in standard and large




### **circaid®**

#### **power added compression band (pac band)**

- Targets the arch of the foot and around the malleoli
- Can be trimmed to fit almost every foot shape
- Lightweight and breathable material
- Bilateral use (left or right foot)
- Available in standard and large

## Sizing chart

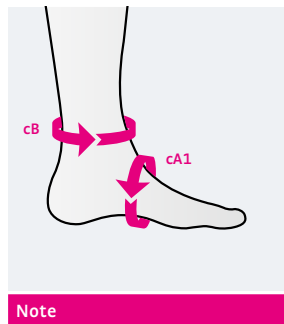

### Note

Circumferences in cm.

Circumference cB is around the ankle and cA1 around the arch.

circaid compression anklet	standard	large
cB	19 – 27	25 – 50
cA1	19 – 36	22 – 41

circaid pac band	standard	large	extra large
cB	19 – 27	25 – 50	45 – 56
cA1	19 – 36	22 – 41	35 – 44



**circaid®**

**customizable ankle foot wrap**

- Provides additional compression to the arch of the foot and around the malleoli
- Customisable to patient-individual circumferences and shape
- Adjustable to reducing circumferences during decongestion
- Interlocking heel band for foot length adjustment
- Heel band offers targeted compression to the Bisgaard's region and ankle




**circaid®**

**single band ankle foot wrap**

- Provides compression to foot and ankle
- Suitable for most foot sizes and shapes
- Ideal for individual bandaging of the foot

## Sizing chart

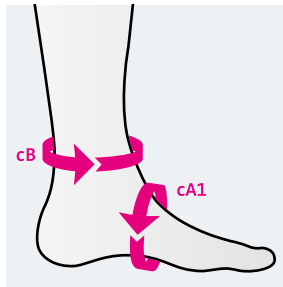

### Note

Circumferences in cm.

Circumference cB is around the ankle and cA1 around the arch.

### circaid customizable interlocking ankle foot wrap

cB	< 56
cA1	< 44

### circaid single band ankle foot wrap

Width	7,6 cm
Length	203 cm



**circaid® juxtafit® premium  
interlocking ankle foot wrap**


- Provides compression to foot and ankle
- Interlocking heel band allows foot length adjustment
- High quality band fasteners with reinforced landing straps
- Heel band offers targeted compression to the Bisgaard's region and ankle
- Available in standard sizes only



**circaid® juxtafit® premium  
ankle foot wrap**

- Provides compression to foot and ankle
- High quality band fasteners with reinforced landing straps
- Can be trimmed in length to provide perfect fit
- Closed heel
- Available in standard sizes and as made-to-measure

# Sizing chart

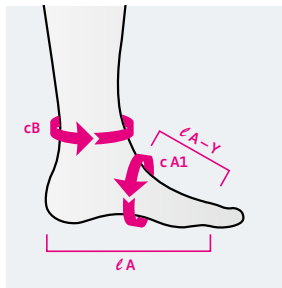

## Note

Circumferences in cm.

Circumference cB is around the ankle and cA1 around the arch.

circaid juxtafit premium interlocking ankle foot wrap	S	M	L
lA-Y	> 9	> 9	> 9
cB	< 36	< 36	< 41
cA1	19-24	24-29	29-34
lA	13-25	13-25	13-25
circaid juxtafit premium ankle foot wrap	S	M	L
lA-Y	> 9	> 9	> 9
cB	< 36	< 36	< 41
cA1	19-24	24-29	29-34
lA	> 16	> 16	> 16



**circaid® juxtalite®**  
**ankle foot wrap**

- Provides compression to foot and ankle
- Can be trimmed in length to provide perfect fit
- Closed heel
- Available in standard sizes only




**circaid®**  
**toe cap**

- Additional compression for the toes
- Provides light to moderate compression
- Individually adjustable length of the toes
- Designed for left or right limb use
- It is recommended to always wear the circaid toe cap with a circaid foot option
- Available in standard sizes



## Sizing chart

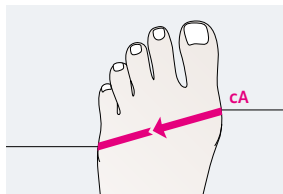

### Note

Circumferences in cm.

circaid juxtalite ankle foot wrap	S	M
cA1	19–26	24–31

circaid toe cap	2	3	4	5	6
cA	20–22	22–24	24–26	26–28	28–30

## circaid® hand options



### **circaid® customizable hand wrap**

- Provides additional compression to the hand from the wrist to the base of the fingers
- Customisable to patient-individual circumferences and shape
- Adjustable to decreasing circumferences during decongestion
- Bilateral use (left and/or right hand)
- Light and breathable material




### **circaid® juxtafit® essentials hand wrap**

- Compresses the hand from the wrist to the base of the fingers
- Adjustable for additional pressure on the dorsum of the hand
- Simple handling - ideal for self-care
- Bilateral use (left and/or right hand)
- Available in standard sizes only



## Sizing chart

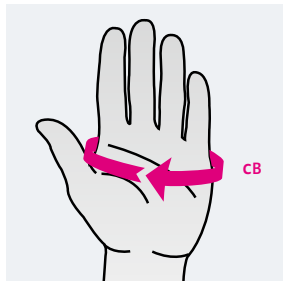

### Note

Circumferences in cm.

For made-to-measure circaid juxtafit hand options please see our product catalogue lymphology (Art.No. 0097D84).

### circaid customizable hand wrap

cC	< 30
----	------

cB	< 30
----	------

### circaid juxtafit® essentials hand wrap


	XS	S	M	L	XL
cB Palm	14 – 17	17 – 20	20 – 23	23 – 26	26 – 29



### **circaid® glove**

- Additional compression for the fingers
- Provides light to moderate compression
- Individually adjustable length of the fingers
- Designed for left or right hand use
- It is recommended to always wear the circaid glove with a circaid arm garment.
- Available in standard sizes only

## Sizing chart

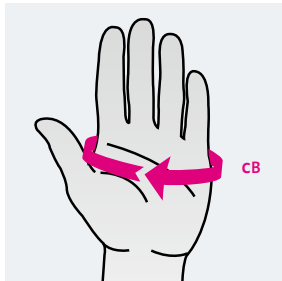

### Note

Circumferences in cm.

circaid glove					
	2	3	4	5	6
cB Palm	18 – 21	20 – 23	22 – 25	24 – 27	26 – 29



## Valuable hints on how to measure correctly

- Please take skin measurements.
- Patient should be measured in the morning when the limb has the smallest circumference.
- Patient's circumferences should remain constant from the time of taking the measurements until the time of product delivery.
- For all circaid leg products: patient should be measured in standing position, if possible.
- For all circaid arm products: patient's arm should be held straight.
- Please measure the lengths on the inside of the arm.
- Please use the up-to-date version of the order forms to ensure an easy and quick order process.

## medi care products



Intensive skin care plays a major role in compression treatment.

The medi skin care products are tailored to compression therapy. medi soft, medi day and medi night have proven to be very successful, and contribute to a pleasant skin sensation.

# circaid® intended purposes

## **Intended purpose circaid leg garments**

---

The compression system is designed to provide compression to the leg for patients with venous and lymphatic disorders.

## **Intended purpose circaid foot options**

---

The compression system is designed to provide compression to the foot and ankle for patients with venous and lymphatic disorders.

## **Intended purpose circaid toe cap**

---

The compression system is designed to provide compression to the toes for patients with venous or lymphatic disorders.

## **Intended purpose circaid arm garments**

---

The compression system is designed to provide compression to the arm for patients with venous and lymphatic disorders.

## **Intended purpose circaid hand options**

---

The compression system is designed to provide compression to the hand for patients with venous and lymphatic disorders.

## **Intended purpose circaid glove**

---

The compression system is designed to provide compression to the fingers for patients with venous and lymphatic disorders.



Headquarters  
medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-781  
export@medi.de  
www.medi.de/en

# Order hotline

**T +49 921 912-0**

Orders, delivery information, product information  
and advice

# Product service

**T +49 921 912-0**

**F +49 921 912-781**

Complaints processing standard, product suggestions

medi GmbH & Co. KG  
Medicusstraße 1  
95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-781  
[export@medi.de](mailto:export@medi.de)  
[www.medi.de/en](http://www.medi.de/en)



0097C63/04.2020