



medi

medi
product catalogue
Venous Health



medi. I feel better.

COMPRESSION for chronic venous disease and PAD / Diabetes?

Merino-Plush in the entire foot

Solid ribbed structure

Unique, soft padding from above the ankles down to the tip of the toes

Yes, certainly!*

mediven® angio – Safety down to the toes



Extra long, flat seam

mediven® angio

Safety down to the toes.


* Product safety proven by clinical trial: The use of the medical compression stocking mediven angio in patients with chronic venous diseases and concomitant mild to moderate peripheral artery disease (PAD) and / or Diabetes mellitus is safe.¹
¹ Rother U et al. Safety of medical compression stockings in patients with diabetes mellitus or peripheral arterial disease. BMJ Open Diab Res Care 2020;8:e001316.

The medi treatment traffic light

For a simple, safe and indication-appropriate treatment.

Step 1: Determine the indication and evaluate comorbidities

Step 2: Consider patient-individual criteria and comorbidities

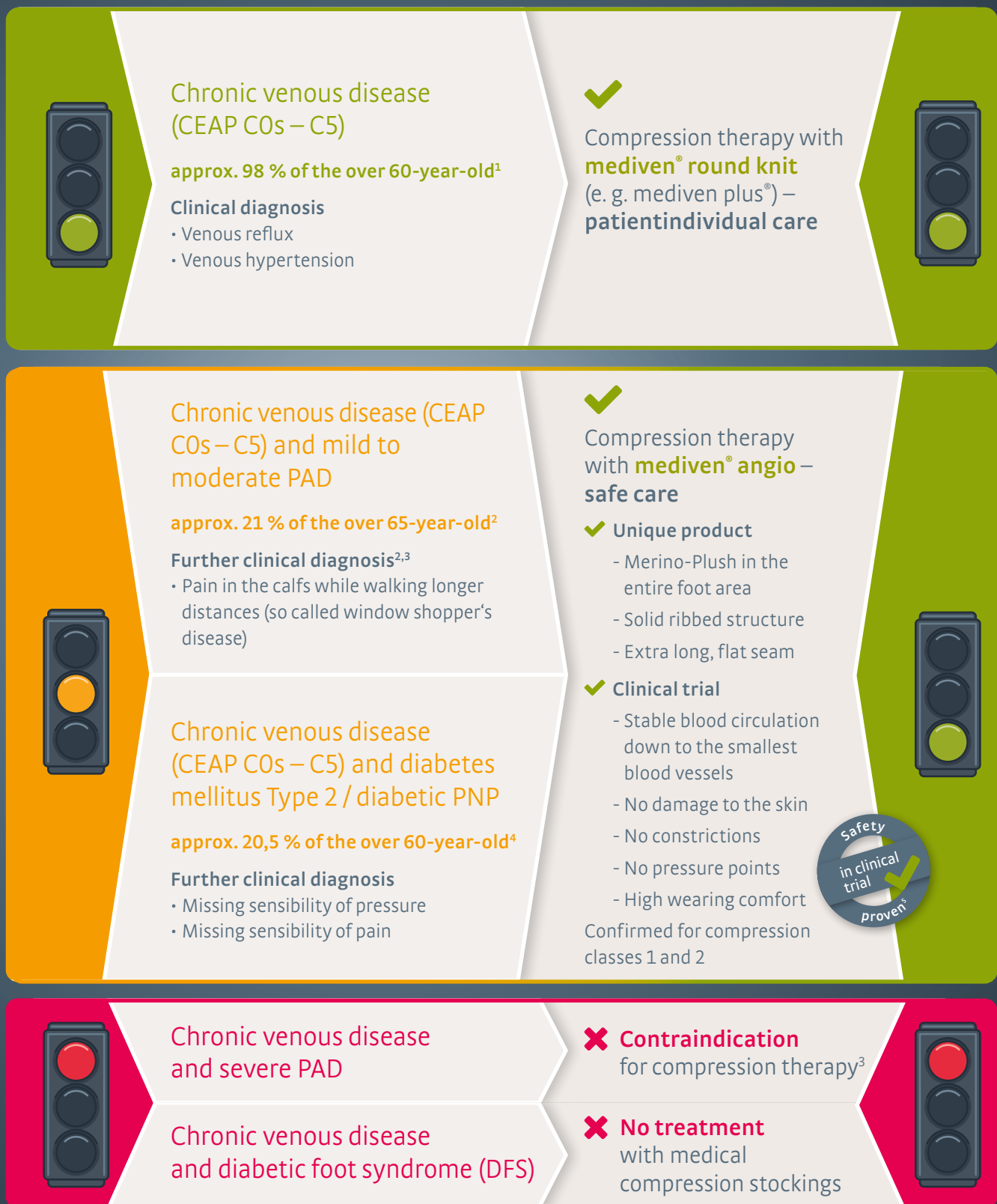

¹ Rabe E et al. Bonner Venenstudie der Deutschen Gesellschaft für Phlebologie. Phlebologie 2003;32:1–14.

² Lawall H et al. S3-Leitlinie zur Diagnostik, Therapie und Nachsorge der peripheren arteriellen Verschlusskrankheit. Available at: <https://www.awmf.org/leitlinien/detail/II/065-003.html> (Last access 09.12.2020)

³ Rabe E et al. S2k-Leitlinie: Medizinische Kompressionstherapie der Extremitäten mit Medizinischem Kompressionsstrumpf (MKS), Phlebologischem Kompressionsverband (PKV) und Medizinischen adaptiven Kompressionssystemen (MAK). Available at: <https://www.awmf.org/leitlinien/detail/II/037-005.html> (Last access 09.12.2020).

⁴ Bericht der nationalen Diabetes-Surveillance 2019. Diabetes in Deutschland. Robert Koch-Institut.

⁵ Rother U et al. Safety of medical compression stockings in patients with diabetes mellitus or peripheral arterial disease. BMJ Open Diab Res Care 2020;8:e001316.





Driving the future in compression therapy

A strong brand.

A wide-ranging product range.

Convincing concepts.

medi World of Compression: signpost to the future with more than 70 years' competence in the field of compression. Your partner for the highest demands on quality, innovation and design. Let's shape the future together and create positive brand experiences – for you and your clients!

Satisfied customers and new potential

Numerous possible combinations for more successful treatment.

mediven compression garments are available in many colours and variants. So medi offers the right garment for several phlebological indications. The broad assortment of add-ons helps patients enjoy wearing their stockings. Join medi in profiting from partner-like goodwill and helpful services.

medi World of Compression

medi convinces within compression by a holistic concept. For this reason you can trust in medi products. They are tailored down to the smallest detail to the needs of your customers and promise best possible treatment success.

To introduce you to medi World of Compression we created indication related catalogues. This enables you to find the correct product for all the different indications and situations in no time. Please find a short explanation of the different literature below:



Venous Health

mediven round-knit compression stockings are the cornerstone of the conservative treatment for patients with venous disorders. Discover more about the numerous styles and variants of mediven and circaid products for venous health.

Art. no.: 0097D67



Therapy Concept Lymphology

The medi Therapy Concept Lymphology shows the new approach in the treatment of lymphoedema. It combines the decongestion phase, the transition phase and the maintenance phase and provides the right products for every stage. The circaid and the mediven flat knit products make the treatment of lymphoedema more individual than ever before.

Art. no.: 0097D84



Therapy Concept Leg Ulcer

The medi Therapy Concept Leg Ulcer accompanies the patient during the complete treatment of venous leg ulcer by connecting the acute phase to the prevention of recurrence phase.

Available as pdf only.



Clinical Compression

The world of Clinical Compression is more than just thrombosis prophylaxis in the clinical environment. Learn more about the multiple application fields of medi's hospital products and benefit from over 40 years of experience with Clinical Compression.

Art. no.: 0099580


8

Individual patient care with mediven®

16

Overview mediven®

Compression classes, Length and features, Measurement charts, colour spectrum and product properties.

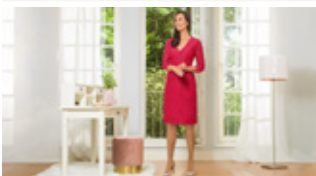

24

mediven sheer & soft®

medi's most translucent compression stocking.



28

mediven elegance®

Beauty that can be seen.



32

mediven® for men

Stylish compression socks for a fashionable business look.



36

mediven® comfort

Softness that can be felt.



40

mediven plus®

Reliability that endures.



46

mediven active®


Hard-wearing compression socks for active men.



50

mediven® forte

Protection that can be felt.

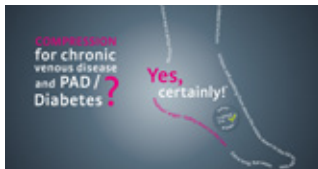

56

mediven® angio

Safety down to the toes.

72

**Special round knit models
Gusset versions**

Overview of the available features depending on fabric.



80

mediven® ulcer kit

Ulcer stocking system for the treatment of leg ulcer.



86

Basic care of medi

duomed®, duomed smooth



94

mediven harmony®

Round knit armsleeves in seven sizes.



98

circaid®

Adjustable compression garments for the treatment of venous diseases.



110

medi travel®

Over-the-counter travel socks for people with a healthy venous system.



114

medi accessories

127

Service

More than just a manufacturer

medi produces medical aids for a variety of conditions and life situations. But we don't see ourselves solely as a manufacturer. On the contrary, we handle our own research and development, technology, production, marketing and sales in-house – always keeping our focus on the market, its demands and variables, and the medi vision.

medi offers products in three business divisions: Compression, Orthopaedics and Footcare. These products are used and distributed by medical supply stores, hospitals and clinics, doctors' practices and therapists. In other words, specialists who know which users need which aids to improve their quality of life.

What we're proud of and what makes our products so special is that they're free of negative side effects.

Our range of products primarily includes medical compression stockings and adjustable compression for people with vein disorders, lipoedema and lymphoedema, supports and braces for the conservative and post-operative treatment of orthopaedic injuries and disorders, stockings for the prevention of thrombosis, compression garments for use after plastic and cosmetic surgery plus a variety of therapy-supporting products.

We support our partners in the sale and supply of our products and have developed intelligent concepts and services for this purpose.




Quality Certificates

As a manufacturer of medical products, medi has a high awareness for quality. Product and system certificates prove that all legal requirements are met.

CE mark

Our medi medical devices meet all of the requirements of the European Directive 93 / 42 / EEC and of the German Medical Devices Act. In accordance with regulations, our products are subject to the conformity assessment procedures and are granted the CE mark. As a kind of “passport for industrial products”, the CE mark guarantees free trade within the European Economic Area (EEA).

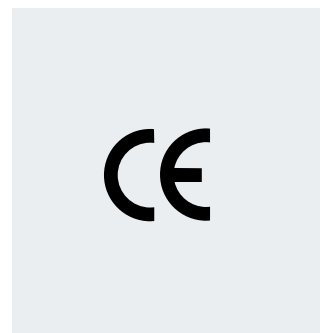


Quality label

Our medical compression hosiery is regularly checked by the Hohenstein Research Institute according to the quality and test provisions of the RAL-GZ 387. As confirmation that our products meet the quality conditions, you will find the quality label printed on the packaging.



Textile trust quality seal

All compression stockings and garments, whether circular-knit or flat-knit, thrombosis prophylaxis stockings and supports are regularly examined and rated according to strict standards by independent test centres for any toxic substances potentially damaging to human health. Granting of the Oeko-Tex Standard 100 certificate guarantees: tested quality for the protection of your health.

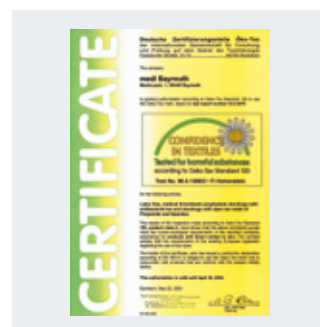

DIN EN ISO 9001 : 2000 / DIN EN 13485

medi maintains the quality management system DIN EN 13485. Since 1996, we have demonstrated our ISO 9000 status to our customers with a certificate. From this time, medi has exploited the opportunities offered by this standard to continually improve products from the business units Compression and Orthopaedics in order to organise safer and more efficient processes and procedures. We define customer satisfaction as a decisive measure of quality.



Hohenstein testing device – HOSY

Pressure is the most important feature of a medical compression stocking.



Individual patient care with mediven®

medi. I feel better.

With this claim, we strive to develop indication-appropriate products. Suitable fabrics designed for indications and lifestyle of individual patients with venous disorders.





The mediven® observational study - facts and figures

The study proves: High acceptance of mediven® medical compression stockings^{1,2}

High acceptance

Up to **98 %**
of patients

had no symptoms while wearing the
medical compression stockings

Wearing comfort

More than **70 %**
of patients

considered the medical compression
stockings to be comfortable

Improving symptoms

In up to **53.2 %**
of patients

the medical compression stockings
improved the subjective symptoms

Good compliance

Up to **75.6 %**
of patients

wore the medical compression stockings
on a daily basis

In **17.7 %**
of patients

the objective clinical findings improved
according to the CEAP class*

Up to **68.4 %**
of patients


wore the medical compression stockings
from morning until evening

¹ Schwahn-Schreiber et al. Phlebologie 2016;45(1): 15 – 24.

² Schwahn-Schreiber et al. Phlebologie 2016;45(4):207 – 214.

* CEAP: classification of chronic venous disorders

The gap between the medical care situation and patient-specific criteria

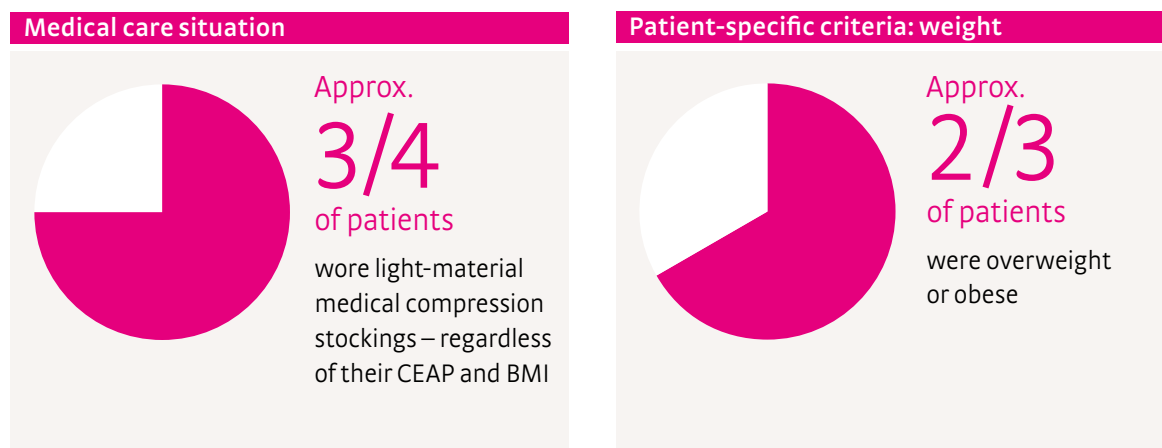

The term “material” encompasses various textile characteristics of a medical compression stocking and is defined by strength, rigidity and elasticity of the knitted fabric.

The mediven observational study has shown patient-specific criteria are rarely taken into account when prescribing medical compression stockings.

Although two thirds of the patients were overweight or obese, 73.8 percent of compression stockings prescribed were made from light material. In people with higher weight frequent constrictions occurred due to wearing medical compression stockings of light material.

2 steps to the perfect, individualised mediven compression stocking:

Step 1: Phlebological Indication Overview








Step 2: Consider patient-specific criteria

Step 1: Phlebological Indication Overview

medi compression aids improve

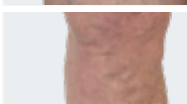- venous symptoms including pain reduction
- quality of life for sufferers of chronic venous diseases

and the following indications are to be considered (highest level of recommendation):¹

			Product (CCL)	mediven sheer & soft (15-20 mmHg, 20-30 mmHg)	mediven elegance (1, 2)	mediven for men (1, 2)	
Indication							
Chronic venous diseases according to CEAP							
C0s		No visible signs of a venous disease		•	•	•	
C1		Spider veins, telangiectasias, reticular varicose veins		•	•	•	
C2		Varicose veins with no signs of a CVI		•	•	•	
C3		Prevention of and therapy for venous oedema				•	
C4		Prevention and therapy of skin changes C4a Pigmentation, eczema C4b Lipodermatosclerosis, atrophy blanche C4c Corona phlebectatica					
C5		Healed venous ulcers (UCV), Prevention of UCV recurrence					
C6		Florid UCV					
Other							
Therapy for mixed venous ulcer							
Other chronic venous diseases							
Initial phase following varicose therapy				•	•		
Functional venous insufficiency due to obesity							
Other oedemas							
Congestion complaints and oedema in pregnancy, postoperative reperfusion oedema, post-traumatic, postoperative, occupational oedema, medicinally induced oedema, oedema due to immobility						•	
Thromboembolism							
• Superficial venous thrombosis				•	•	•	
• Deep vein thrombosis							
• Condition after thrombosis, post-thrombotic syndrome							
• Thrombosis prophylaxis in mobile patients				•	•	•	
• Arm vein thrombosis							
Other indications							
• Inflammatory dermatoses of the legs					•	•	
• Nausea, dizziness in pregnancy					•		

Note:

1. It is important to consider the different materials of medical compression stockings (MCS): light, medium, strong material. The term 'material' includes different textile MCS characteristics (strength, rigidity and elasticity). The usage of the three materials depends on the patients' characteristics (e. g. body weight, oedema, severity of the venous disorder, connective tissue): Keep in mind: The higher the body weight and / or the stronger the venous oedema and / or

medi tip: In respect to indications, we refer to the current S2k Guideline Medical Compression Therapy of the German Society for Phlebology.¹

¹ Rabe E et al. Medical compression therapy of the extremities with medical compression stockings (MCS), phlebological compression bandages (PCB), and medical adaptive compression systems (MAC): S2k guideline of the German Phlebology Society (DGP) in cooperation with the following professional associations: DDG, DGA, DGG, GDL, DGL, BVP. Hautarzt 2021;72(2):137-152.

	mediven comfort (1, 2)	mediven plus (1, 2, 3)	duomed (1,2)	mediven active (1, 2)	mediven forte (2, 3)	mediven harmony (1, 2)	circaid juxtalite (20-30 / 30-40 / 40-50 mmHg)	mediven ulcer kit (2 x 20 mmHg)	circaid juxtacures (20-30/30-40/ 40-50 mm Hg)
	•	•	•	•	•				
	•	•	•	•	•				
	•	•	•	•	•				
	•	•	•	•	•		•		
	•	•	•	•	•		•		
		•			•		•	•	
							•	•	•
							•	•	•
	•	•	•		•		•		
					•				
	•	•	•	•	•				
	•	•	•	•	•				
	•	•	•	•	•				
	•	•	•	•	•				
						•			
	•	•	•	•	•				
	•	•	•		•				

Decongestive potential

the more severe the venous disorder and / or the softer the connective tissue the firmer the MCS material.

- MCS of compression class I (CCL I) might be considered in elderly patients or in patients with an initial treatment, since it is well known, that the donning and doffing process of MCS is easier with CCL I compared to higher compression classes resulting in a better compliance.

Step 2: Consider patient-specific criteria

Recommendation – mediven® care compass:

- The higher the weight
 - The stronger the oedema (venous oedema)
 - The more severe the disease
 - The softer the connective tissue
- The stronger the stocking material should be.
- Already meeting a single criterion requires a stronger material.
- Please note! For lymphoedema or lipoedema, use mediven flat knit.

Case example: Obese, mild disease and light oedema → strong material





medi tip:

Safe care for concomitant diseases.
Customers with chronic venous disease and simultaneously


→ mild to moderate peripheral arterial disease (PAD) and / or

→ diabetes mellitus / diabetic polyneuropathy (PNP)


have special requirements for a medical compression stocking.
mediven angio was particularly developed for this purpose – and its safety was proven in a clinical trial. Actively ask your customers about these concomitant diseases.

Criteria	Material (e.g., mediven elegance)
1. Weight	
2. Oedema (venous oedema)	—
3. Severity of the disease	
Recommended mediven® compression stockings:	mediven sheer & soft
	mediven elegance
	mediven for men
	
	

medi tip:

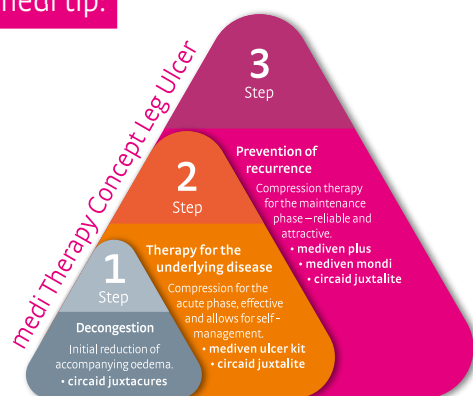

For C5 and C6, please also consider the medi Therapy Concept Leg Ulcer.
medi.biz/wound

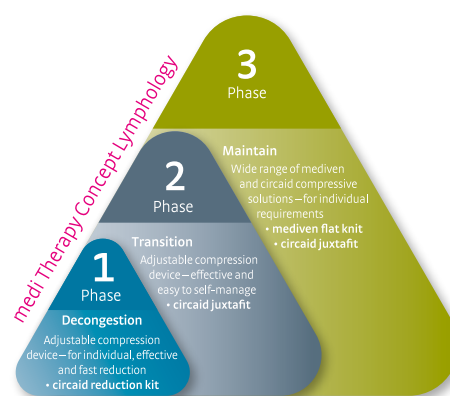

Please see the medi Therapy Concept Lymphology for the treatment of lip- and lymphoedema.
Art. no.: 0097D84

Medium
(e.g., mediven plus)

Strong
(e.g., mediven forte)



+

++

mediven comfort

mediven plus

mediven active

mediven forte




mediven® medical compression stockings

mediven round-knitted compression stockings are the cornerstone of the conservative treatment for patients with venous disorder. They are the basic form of therapy for patients with venous disorders.

In combination with a healthy lifestyle and physical activity, medical compression stockings improve the venous signs and symptoms. Your customers will soon notice that mediven compression stockings are good for them and their well-being, and that they are improving their quality of life.



mediven® product features

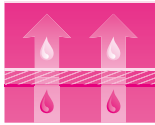

Clima Comfort

Clima Comfort*

- Wicks moisture away from the skin's surface quickly
- Guarantees a high level of breathability



Clima Fresh

Integrated Clima Fresh function

- Antibacterial effect
- Thus inhibits odour-forming bacteria



Soft Elastic

Soft Elastic

- Noticeably more toe freedom with closed toe
- Increases wearing comfort
- Less pressure is exerted on the end of the foot



Perfect Fit

Perfect Fit

- A very precise fit
- Medically effective pressure gradient
- Optimal care conditions in standard and made-to-measure version



Variety

A wide range of variants and special models

Colours for greater quality of life

- Large selection of standard colours and the latest trend colours available.

Hightech fibres for natural elasticity

- High double-stretch elasticity
- Optimal user comfort
- Easy to put on and take off
- Perfect, wrinkle-free fit
- Highly attractive transparency and refined feeling



cf. Ben Ammar, F. (DermScan): Clinical evaluation of the skin compatibility of a medical device - application testing realized under dermatological control, Tunis 2012/2013.



All mediven compression stockings are latex-free.



* Clima Comfort

* Scientifically tested by French institute IFTH. The test results show that Clima Comfort technology from medi assures climate control between the skin and the stocking. With 11/12 points, medi achieved an outstanding overall score.

mediven® ulcer kit product features



Softness

Softness**

- Soft material for a pleasant wearing comfort
- Slides smoothly and gently over the skin



Easy donning

Easy donning***

- Liner and outer stocking with both 20 mmHg for easy donning and doffing
- Soft liner stocking eases donning of the outer stocking



Antibacterial

Antibacterial**


- Knitted fabric with antibacterial effect
- Inhibits growth and nidation of malodorous bacteria

** These features only apply to mediven ulcer liner stocking

*** These features only apply to mediven ulcer kit


mediven® Measurement charts

Measurement chart

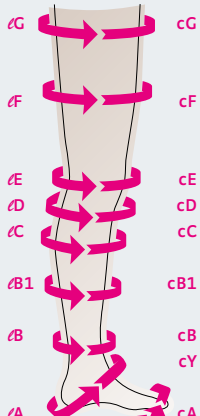
Standard fit without seam		I	II	III	IV	V	VI	VII
AG with topband, normal	cG	43–48	45–52	49–56	53–60	56–64	60–68	64–72
AG with topband, wide	cG	49–57	53–62	57–67	61–72	65–77	69–81	73–85
ℓ=Length c=Circumference								
	cG	43–57	45–62	49–67	53–72	56–77	60–81	64–85
	cF	39–52	41–56	44–60	47–65	50–69	53–73	57–78
	cE	30–37	33–40	35–43	37–45	39–48	41–51	42–53
	cD	27–33	29–36	32–39	34–42	36–45	38–48	40–50
	cC	28–34	30–37	33–40	35–43	37–46	39–49	41–51
	cB1	23–27	24–29	26–32	29–35	31–37	33–39	35–41
	cB	18–20	20–22	22–24	24–26	26–28	28–30	30–32
	cY	26–31	28–33	29–35	31–37	32–38	33–40	34–42
	cA	17–22	19–24	21–26	23–29	25–32	27–34	28–35
Circumference in cm								
Length measurements								
ℓG regular		72–83 cm						
ℓG petite		62–71 cm						
ℓD regular		39–44 cm						
ℓD petite		34–38 cm						
Foot		Highly elastic universal foot size, shoe size 36–43						

mediven elegance®

Measurement chart

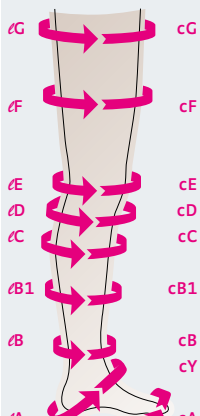Standard fit without seam		I	II	III	IV	V	VI	VII
AG with topband, normal	cG	43–48	45–52	49–56	53–60	56–64	60–68	64–72
AG with topband, wide	cG	49–57	53–62	57–67	61–72	65–77	69–81	73–85
ℓ=Length c=Circumference								
	cG	43–57	45–62	49–67	53–72	56–77	60–81	64–85
	cF	39–52	41–56	44–60	47–65	50–69	53–73	57–78
	cE	30–37	33–40	35–43	37–45	39–48	41–51	42–53
	cD	27–33	29–36	32–39	34–42	36–45	38–48	40–50
	cC	28–34	30–37	33–40	35–43	37–46	39–49	41–51
	cB1	23–27	24–29	26–32	29–35	31–37	33–39	35–41
	cB	18–20	20–22	22–24	24–26	26–28	28–30	30–32
	cY	26–31	28–33	29–35	31–37	32–38	33–40	34–42
	cA	17–22	19–24	21–26	23–29	25–32	27–34	28–35
Circumference in cm								
Length measurements								
ℓG regular		72–83 cm						
ℓG petite		62–71 cm						
ℓD regular		39–44 cm						
ℓD petite		34–38 cm						
Foot		Highly elastic universal foot size, shoe size 36–43						

mediven plus® • mediven® comfort

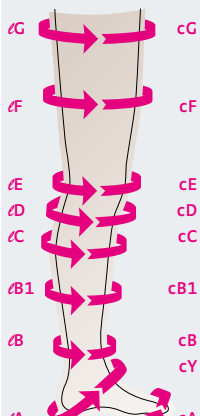Standard fit without seam		I	II	III	IV	V	VI	VII
AG with topband, normal	cG	43–48	45–52	49–56	53–60	56–64	60–68	64–72
AG with topband, wide	cG	49–57	53–62	57–67	61–72	65–77	69–81	73–85
ℓ=Length c=Circumference								
	cG	43–57	45–62	49–67	53–72	56–77	60–81	64–85
	cF	39–52	41–56	44–60	47–65	50–69	53–73	57–78
	cE	30–37	33–40	35–43	37–45	39–48	41–51	42–53
	cD	27–33	29–36	32–39	34–42	36–45	38–48	40–50
	cC	28–34	30–37	33–40	35–43	37–46	39–49	41–51
	cB1	23–27	24–29	26–32	29–35	31–37	33–39	35–41
	cB	18–20	20–22	22–24	24–26	26–28	28–30	30–32
	cY	26–31	28–33	29–35	31–37	32–38	33–40	34–42
	cA	17–22	19–24	21–26	23–29	25–32	27–34	28–35
Circumference in cm								
Length measurements								
ℓG regular		72–83 cm						
ℓG petite		62–71 cm						
ℓD regular		39–44 cm						
ℓD petite		34–38 cm						
Foot		Highly elastic universal foot size, shoe size 36–43						

Only with mediven plus AT		
We recommend a maxi panty top from a body circumference of:		
Size I–III	W = from 95 cm	H = from 110 cm
Size IV–V	W = from 120 cm	H = from 135 cm
Size VI–VII	W = from 140 cm	H = from 145 cm

mediven® Measurement charts

Measurement chart

mediven® forte

Standard fit without seam		I	II	III	IV	V	VI	VII
AG with topband, normal	cG	44–47	46–50	50–54	53–58	56–61	60–65	64–70
AG with topband, wide	cG	48–53	51–57	55–61	59–65	62–70	66–74	71–80
ℓ=Length								
c=Circumference								
ℓG	cG	44–53	46–57	50–61	53–65	56–70	60–74	64–80
ℓF	cF	40–47	42–51	45–55	48–58	51–62	54–67	58–72
ℓE	cE	30–37	33–40	36–42	38–44	40–46	42–48	44–50
ℓD	cD	28–33	30–35	32–37	34–39	36–41	38–43	40–45
ℓC	cC	29–34	31–36	33–38	35–40	37–42	39–44	41–46
ℓB1	cB1	24–26	26–28	28–31	30–33	32–35	34–37	36–39
ℓB	cB	18–20	20–22	22–24	24–26	26–28	28–30	30–32
ℓY	cY	28–30	29–32	31–34	33–36	34–38	36–40	37–41
ℓA	cA	18–21	20–23	22–25	24–27	26–29	28–31	30–33

Length measurements		Foot
ℓG regular	72–83 cm	2 foot lengths, closed toe
ℓG petite	62–71 cm	Small foot up to 23 cm ≅ Shoe size 36–38
ℓD regular	38–43 cm	Large foot from 23 cm and up ≅ Shoe size 38–43
ℓD petite	33–37 cm	

Measurement chart


mediven active® • mediven® for men mediven® angio

Standard fit without seam		I	II	III	IV	V	VI	VII
ℓ=Length								
c=Circumference								
ℓD	cD	27–33	29–36	32–39	34–42	36–45	38–48	40–50
ℓC	cC	28–34	30–37	33–40	35–43	37–46	39–49	41–51
ℓB1	cB1	23–27	24–29	26–32	29–35	31–37	33–39	35–41
ℓB	cB	18–20	20–22	22–24	24–26	26–28	28–30	30–32
ℓY	cY	26–31	28–33	29–35	31–37	32–38	33–40	34–42
ℓA	cA	17–22	19–24	21–26	23–29	25–32	27–34	28–35

Length measurements		Foot
mediven active		mediven active
ℓD regular	43–49 cm	closed toe shoe size 38–48
ℓD petite	37–42 cm	mediven for men
mediven for men		closed toe shoe size 39–46
mediven angio		mediven angio
ℓD regular	39–44 cm	2 foot lengths, closed toe
ℓD petite	34–38 cm	Small foot up to 25 cm ≅ Shoe size 36–40
		Large foot from 25 cm ≅ Shoe size 40–46

Correct fitting of round knit medical compression stockings


Note: Please measure length measurements to ℓG on the inside of the leg

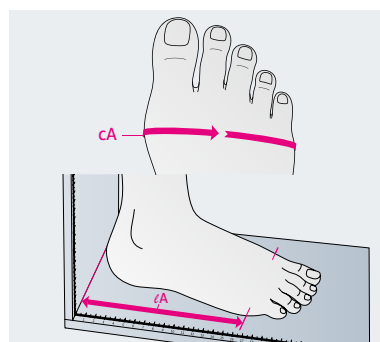

cA, lA, lZ


Take circumferential measurement **cA** just behind the joint at the base of the toes where the little toe begins.

The length of the foot for open toe stockings is determined by the length measurement **lA** which is measured from the bottom of the heel to measurement point **cA**. **The length of the foot for closed toe stockings** is measured from the heel of the foot to the tips of the toes. (**lZ-measurement**)

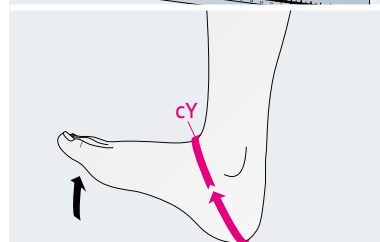


cY

Take the circumferential measurement **cY**, around the heel and the instep, with the foot in maximum dorsal extension.

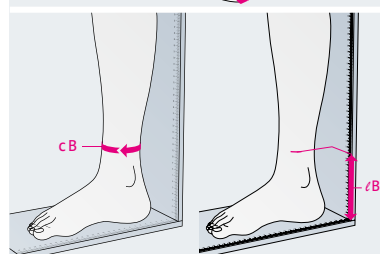


cB, lB

Take the circumferential measurement **cB** and length measurement **lB** at the narrowest circumference of the leg below the knee, directly above the ankle. This measurement above the ankle bones is especially important for selecting the stocking size, because the pressure here will be 100%.

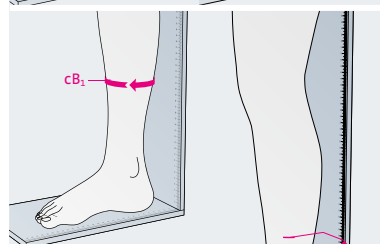


cB¹, lB¹

The circumferential measurement **cB¹** and the length measurement **lB¹** are at the point where the calf begins, or at the transition point to the Achilles tendon.

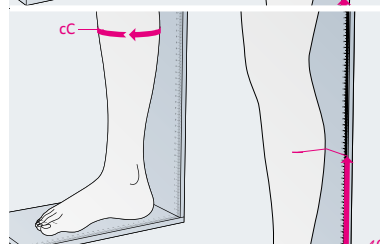


cC, lC

Take the **cC** and **lC** length measurements at the widest circumference of the calf.

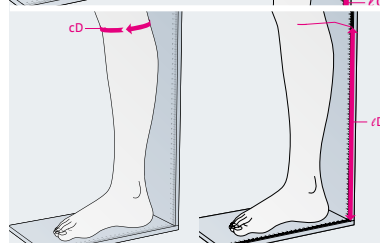


cD, lD

Take the circumference **cD** and length **lD** measurements about two finger-widths below the hollow of the knee at the level of the head of the fibula.

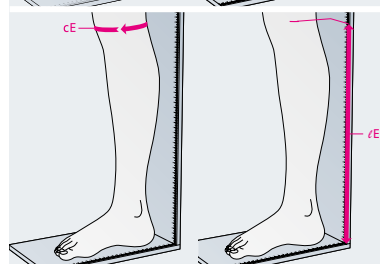


cE, lE

The circumferential measurement **cE** is the circumference of the knee measured through the hollow of the knee in the middle of the kneecap. Take the length measurement **lE** when the leg is straight.

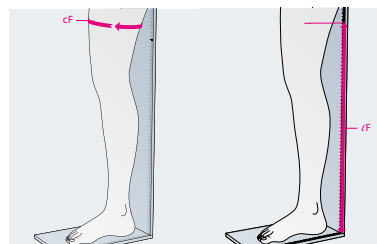


cF, lF

Circumferential measurement **cF** and length measurement **lF** are located where the inner thigh muscle begins or in the middle of the thigh.

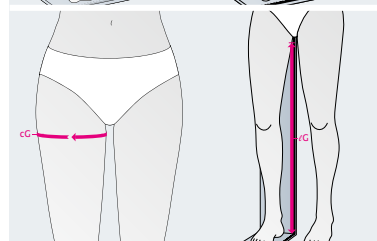


cG, lG

Take the **cG** and length measurement **lG** about 3 cm below the crotch.

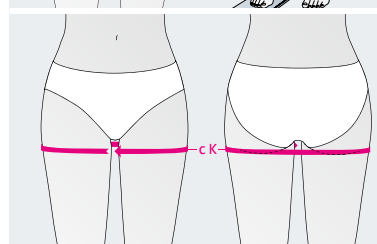


cK

Take the circumferential measurement **cK** at the height of the g measurement, around both legs.

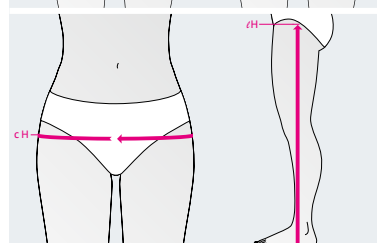


cH, lH

Take the circumferential measurement **cH** at hip level at the widest point of the buttocks.

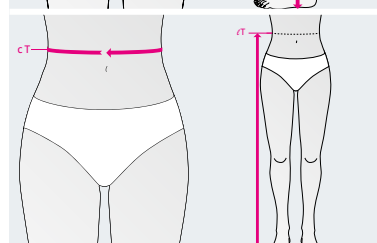


cT, lT

Take the circumferential measurement **cT** round the waist.

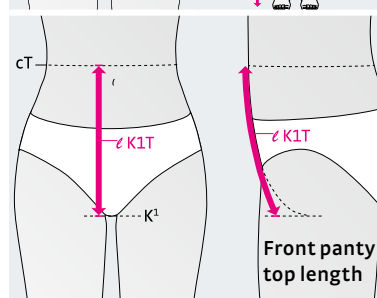

Front panty top length
Measure from the crotch to the waist. It is however more comfortable for the patient if you go 3 cm to the side of the crotch and measure the panty top length from there.

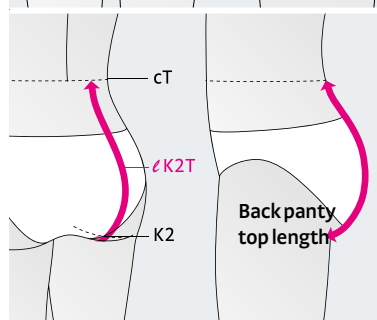

Back panty top length
Measure from the gluteal fold to the waist. If the patient requests a longer or shorter panty top, you must measure to the length the patient requests. The "waist" point is only the standard length.

mediven® colour spectrum

You can find all current trend colours for our round knit qualities at www.medi.de/en.

Colour	mediven elegance	mediven for men	mediven comfort	mediven plus	mediven active	mediven forte	mediven angio
Black	•	•	•	•	•	•	•
Anthracite	•	•	•	•	•	•	
Navy	•	•	•	•	•	•	
Cashmere	•		•	•		•	
Caramel	•		•	•		•	•
Rosé	•		○	•		○	
Bronze	•		○	•		○	
Beige	•		○	•		○	
medi Magenta	○			○			
White	○			○	•		
Brown		•			•		

○ Delivery period up to 10 working days.

mediven® Product range overview Length and fastening

Length	Fastening	mediven elegance	mediven for men	mediven comfort	mediven plus	mediven active	mediven forte	mediven angio
AD	Tricot border	•	•	•	•	•	•	•
	Silicone dot topband (NoB)	•		•	•		•	
	Narrow silicone dot topband (sNoB)	•		•	•		•	
AF	Tricot border	•			•		•	
	Silicone micro dot topband sensitive (MBs)	•		•	•		•	
	Plain silicone topband rose (FBR)	•		•	•		•	
	Silicone dot topband (NoB)	•		•	•		•	
	Silicone dot topband motif (NoM)	•		•	•		•	
AG	Tricot border	•			•		•	
	Hip fastening				•		•	
	Silicone micro dot topband sensitive (MBs)	•		•	•		•	
	Plain silicone topband rose (FBR)	•		•	•		•	
	Silicone dot topband (NoB)	•		•	•		•	
	Silicone dot topband motif (NoM)	•		•	•		•	

Length	Fastening	mediven elegance	mediven for men	mediven comfort	mediven plus	mediven active	mediven forte	mediven angio
AT	Standard panty top	•			•		•	
	Comfort panty top			•				
	Maxi panty top				•			
	Light compr. LT	•			•		•	
	Moderate compr. LT				•		•	
	highly compr. LT				•		•	
	Gusset selection page 59 – 68	•		•	•		•	
AT / U	closed LT	•			•		•	
	open LT	•			•		•	
	Light compr. LT	•			•		•	
	Gusset selection page 59 – 68	•			•		•	
AT / H	Slit across				•		•	
	Light compr. LT				•		•	
	Moderate compr. LT				•		•	
	highly compr. LT				•		•	
	Gusset selection page 59 – 68				•		•	
AT / E	right				•		•	
	left				•		•	
	Light compr. LT				•		•	
	Moderate compr. LT				•		•	
	highly compr. LT				•		•	
	Men's gusset				•		•	
	U-Gusset				•		•	
	Gusset selection page 59 – 68				•		•	

Compression Classes		Products
1 mild compression	Compression in mmHg: 18.0 – 21.0 Compression in kPa: 2.4 – 2.8	mediven elegance (open + closed toe) mediven for men (closed toe) mediven active (closed toe) mediven comfort (open + closed toe) mediven plus (open + closed toe) mediven angio (closed toe) Duomed (open + closed toe)
2 moderate compression	Compression in mmHg: 23.0 – 32.0 Compression in kPa: 3.1 – 4.3	mediven elegance (open + closed toe) mediven for men (closed toe) mediven active (closed toe) mediven comfort (open + closed toe) mediven plus (open + closed toe) mediven forte (open + closed toe) mediven angio (closed toe) Duomed (open + closed toe)
3 strong compression	Compression in mmHg: 34.0 – 46.0 Compression in kPa: 4.5 – 6.1	mediven plus (open + closed toe) mediven forte (open + closed toe)





mediven sheer & soft[®]

medi's most translucent compression stocking. Try a major advance in compression therapy. mediven sheer & soft takes comfort to a whole new level. Yet these compression stockings are so sheer that only you will know that they are medical compression stockings.

mediven sheer & soft®

medi's most translucent compression stocking.

medi tip¹:

Due to the material thickness, the fit and the product characteristics, we recommend mediven sheer & soft for:

- Women
- Normal weight
- No tendency to venous oedema
- Firm connective tissue

Indications²

- Chronic venous disorders: C0s to C2 according to CEAP
- Initial phase after varicose vein therapy
- Superficial venous thrombosis
- Thrombosis prophylaxis in mobile patients

Features

- Extremely sheer, not distinguishable from fashion stockings
- Soft and gentle on the skin
- Available with open toe³



Attractive topband



Knee length stocking (AD)



Panty (AT)



Open toe – detail

¹ Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer.

² Selecting the ideal compression stockings depends on the indication and the individual needs of the patients.

³ mediven sheer & soft, open toe in Ebony: delivery period up to 7 weeks.

Product benefits



Material


Light

Compression class

15 – 20 mmHg,
20 – 30 mmHg

Standard colours

S = Standard

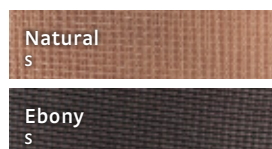

mediven sheer & soft

Styles	AD	AD pte.	AG	AG pte.
	calf	calf petite	thigh	thigh petite
15 – 20 mmHg	O140x11-7	O147x11-7	O161x11-7	O151x11-7
20 – 30 mmHg	O240x11-7	O247x11-7	O261x11-7	O251x11-7
Styles	AT	AT pte.		
	panty	panty petite		
15 – 20 mmHg	O110x11-7	O117x11-7		
20 – 30 mmHg	O210x11-7	O217x11-7		

See the last page for the article numbers

Standard styles

Shoe size 35 – 42

Material composition

	Polyamide (PA)	Elastane (EL)
15-20 mmHg	64 %	36 %
20-30 mmHg	64 %	36 %

Washing instructions

Wash-and-wear

Suitable for washing machine and drier

Measuring mediven sheer & soft

Size	Circumference in cm						
	I	II	III	IV	V	VI	VII
cG	42 – 57	45 – 61,5	49 – 66,5	52 – 73	56 – 79	60 – 84,5	64 – 87,5
cD	26 – 33,5	28 – 37	30 – 40	32 – 43	33,5 – 46,5	35,5 – 49,5	37,5 – 52,5
cC	28 – 35,5	30 – 39	32,5 – 42	35 – 45	37 – 48,5	39,5 – 51,5	41,5 – 54,5
cB	18 – 20	20,5 – 22,5	23 – 25	25,5 – 27,5	28 – 30	30,5 – 33	33,5 – 36
Length							
Calf stocking / panty petite:	34 – 38 cm			Thigh stocking / panty petite:			62 – 71 cm
Calf stocking / panty standard:	39 – 44 cm			Thigh stocking / panty standard:			72 – 83 cm



mediven elegance[®]

Is this really a medical compression stocking? For sure! This is a model that emphasises a woman's beauty. The mediven elegance pampers the legs with its compressive effect. In doing so, it nestles against the leg like a second skin. It can be combined with any item of clothing and perfectly accentuates the female figure. True to the motto: Beauty that can be seen.

mediven elegance. Effective compression has never looked so good.

mediven elegance[®]

All you see is beauty.
Effective compression has never
looked so good.

medi tip¹:

Due to the material thickness, the fit
and the product characteristics, we
recommend mediven elegance for:

- Women
- Normal weight
- No tendency to venous oedema
- Firm connective tissue





Indications²

- Chronic venous disease stage I
according to Widmer or COs – C2
according to CEAP
- Initial phase following varicose therapy
- Superficial venous thrombosis
- Thrombosis prophylaxis in mobile
patients
- Inflammatory dermatoses of the legs
- Nausea and dizziness in pregnancy

Features

- Transparent fine stocking style
- Soft Elastic foot tip guarantees more
freedom for the toes
- Wide and soft cuff for knee length style
- Smaller heel for wrinkle-free fit
- Attractive topband motifs for
thigh-length stockings

Washing instructions

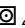Easy care    

Suitable for washing machine and drier.



Soft elastic foot tip



Knee length stocking (AD)
with soft cuff



Plain silicone topband rose (FBR)

¹ Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer.

² Selecting the ideal compression stockings depends on the indication and the individual needs of the patients.

Product benefits



cf. Ben Ammar, F. (DermScan): Clinical evaluation of the skin compatibility of a medical device - application testing realized under dermatological control, Tunis 2012/2013.

mediven elegance

Styles	AD	AD pte.	AG	AG pte.	AG / FBR
	calf narrow silicone topband	calf petite	thigh	thigh petite	thigh with topband
Class 1	0140x11-7	0147x11-7	0160x11-7	0150x11-7	016Hx11-7
Class 2	0240x11-7	0247x11-7	0260x11-7	0250x11-7	026Hx11-7
Styles	AG pte. / FBR	AT	AT pte.	AT / U	AT / U pte.
	thigh petite with topband	panty	panty petite	maternity panty	maternity panty petite
Class 1	015Hx11-7	0110x11-7	0117x11-7	0120x11-7	0127x11-7
Class 2	025Hx11-7	0210x11-7	0217x11-7	0220x11-7	0227x11-7

See the last page for the article numbers

Standard styles

Standard	Size I – VII, Length regular and petite, topband regular and wide
Made-to-measure	available
Special models	available, see pages 56 – 73
Border	AD tricot border, narrow silicone dot topband AF and AG tricot border, Silicone dot topband, Silicone dot topband motif, Plain silicone topband rose, Silicone micro dot topband sensitive All topbands are standard styles.
Gusset	AT Comfort gusset AT / U with closed panty top
Foot part	open toe, closed toe, Shoe size 36 – 43

Material composition

	Elastane (EL)	Polyamide (PA)
CCL 1	34%	66%
CCL 2	39%	61%

medi tip:



medi Butler donning made easier¹

medi skin and textile care offer support

More information on the medi accessories from page 112

¹ In general, the following applies for donning and doffing aids: Rabe E et al. Medical compression therapy of the extremities with medical compression stockings (MCS), phlebological compression bandages (PCB), and medical adaptive compression systems (MAC): S2k guideline of the German Phlebology Society (DGP) in cooperation with the following professional associations: DDG, DGA, DGG, GDL, DGL, BVP. Hautarzt 2021;72(2):137-152.

Material

Light



Compression class

CCL 1, 2

Standard colours

S = Standard

M = Made-to-measure

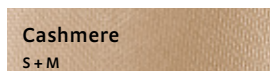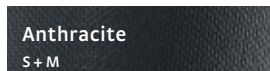

* Delivery period up to 10 working days.



Available with sparkling crystals



mediven® for men

It's generally more difficult to convince men to wear medical compression than women. However, the arguments of mediven for men speak for themselves and will help you.

This compression socks looks like a fashionable men's sock and is the perfect accompaniment for men. The stylish sock in a "business ribbed design" feels like a second skin.

mediven for men. For the fashionable business look.

mediven[®] for men

The elegant compression sock.

medi tip¹:

Due to the material thickness, the fit and the product characteristics, we recommend mediven for men for:

- Men
- Normal weight
- No / light tendency to venous oedema
- Also for soft connective tissue


Indications²

- Chronic venous disease stages I and II according to Widmer or C0s – C3 according to CEAP
- Postoperative reperfusion oedema
- Post-traumatic, postoperative, occupational, medicinally induced oedema
- Oedema due to immobility
- Superficial venous thrombosis
- Thrombosis prophylaxis in mobile patients
- Inflammatory dermatoses of the legs


Features

- Anatomical toe design with practical right/left markings for a perfect fit and user comfort
- Stylish Business Rib Design
- Added lanolin³
- Clima Fresh for long-lasting freshness on the leg and foot
- Extra-wide and especially soft comfort cuff
- Suitable for shoe sizes 39 – 46

Washing instructions

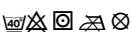Easy-care 

Suitable for washing machine and drier.

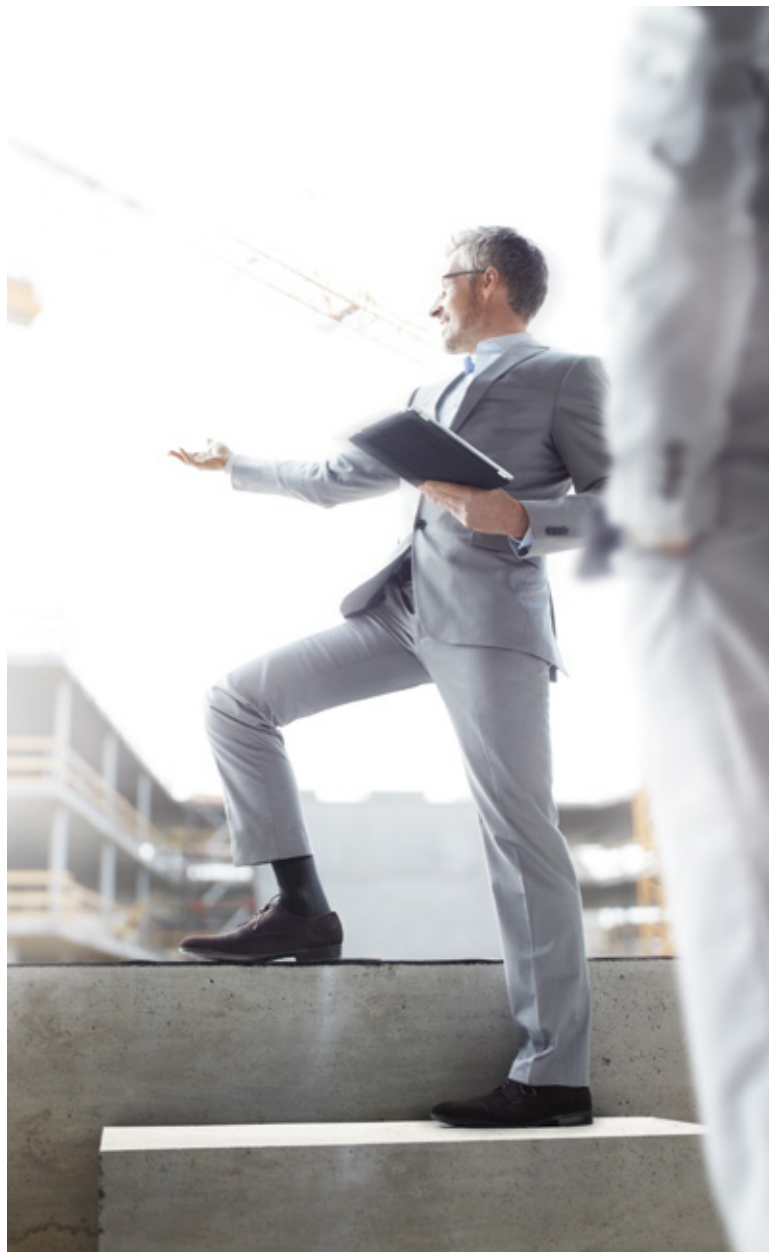

Knee length sock (AD) in fashionable Business Rib Design with comfort cuffs



Anatomically shaped foot with right and left markings

¹ Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer.

² Selecting the ideal compression stockings depends on the indication and the individual needs of the patients.

³ Natural substance used to care dry and chapped skin, has a broad range of applications, for example in child care and baby care.

Product benefits



Clima Comfort



Clima Fresh



Perfect Fit



cf. Ben Ammar, F. (Dermiscan): Clinical evaluation of the skin compatibility of a medical device - application testing realized under dermatological control, Tunis 2012/2013.

mediven for men

Styles

AD

AD pte.



calf

calf petite

Class 1	Article No.	3A40x11-7	3A47x11-7
Class 2	Article No.	3B40x11-7	3B47x11-7

See the last page for the article numbers

Standard styles

Standard	Size I – VII, Length standard and petite
Special models	not available
Border	AD Tricot border
Foot part	closed toe shoe size 39 – 46

Material composition

	Elastane (EL)	Polyamide (PA)
CCL 1	23%	77%
CCL 2	29%	71%

Material

Light

Compression class

CCL 1, 2

Standard colours

S = Standard

Black
S

Anthracite
S

Navy
S

Brown
S



medi tip:



medi Butler donning made easier¹ medi skin and textile care offer support

More information on the medi accessories from page 112

¹ In general, the following applies for donning and doffing aids: Rabe E et al. Medical compression therapy of the extremities with medical compression stockings (MCS), phlebological compression bandages (PCB), and medical adaptive compression systems (MAC): S2k guideline of the German Phlebology Society (DGP) in cooperation with the following professional associations: DDG, DGA, DGG, GDL, DGL, BVP. Hautarzt 2021;72(2):137-152.





mediven® comfort

mediven comfort is made of wonderfully soft material. The stocking in compression classes 1 and 2 makes the therapy and treatment of venous disorders as comfortable as possible. Thanks to its lanolin finish, it cares for the skin and treats it particularly gently. How do you win over your customers for mediven comfort? Let them feel for themselves!

mediven comfort. The cuddly side of compression.

mediven[®] comfort

All you feel is softness.

The cuddly side of compression.

medi tip¹:

Due to the material thickness, the fit and the product characteristics, we recommend mediven comfort for:

- Women
- Normal weight / overweight²
- Light to severe tendency to venous oedema²
- Firm connective tissue

Indications³

- Chronic venous disease stages I to II according to Widmer or C0s – C4 according to CEAP
- Initial phase following varicose therapy
- Congestion problems and oedema during pregnancy
- Postoperative reperfusion oedema
- Post-traumatic, postoperative, occupational, medicinally induced oedema
- Oedema due to immobility
- Superficial venous thrombosis
- Condition after thrombosis, post-thrombotic syndrome
- Thrombosis prophylaxis in mobile patients
- Inflammatory dermatoses of the legs
- Nausea and dizziness in pregnancy





Features

- Particularly soft material for easy donning and doffing
- Added lanolin⁴
- Double-knitted comfort sole for that enhanced feeling of comfort when walking
- Shaping comfort panty top



- Elastic and hard-wearing comfort gusset

Washing instructions

Easy-care    

Suitable for washing machine and drier.

¹ Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer.

² Use mediven comfort for serious oedema only for patients with normal weight. For people who are overweight and have severe oedema, medi recommends stronger materials to prevent lace-up grooves.

³ Selecting the ideal compression stockings depends on the indication and the individual needs of the patients.

⁴ Natural substance used to care dry and chapped skin, has a broad range of applications, for example in child care and baby care.



Comfort panty top



Double-knitted comfort sole



Silicone dot topband

Product benefits



Clima Comfort



Clima Fresh



Variety



Perfect Fit



mediven comfort

open toe

Styles	AD	AD pte.	AG / NoB	AG pte. / NoB	AT	AT pte.
	calf	calf petite	thigh with topband	thigh petite with topband	panty	panty petite
Class 1	Z540x01-7	Z547x01-7	Z569x01-7	Z559x01-7	Z510x01-7	Z517x01-7
Class 2	Z640x01-7	Z647x01-7	Z669x01-7	Z659x01-7	Z610x01-7	Z617x01-7

closed toe

Styles	AD	AD pte.	AG / NoB	AG pte. / NoB	AT	AT pte.
	calf	calf petite	thigh with topband	thigh petite with topband	panty	panty petite
Class 1	Z540x11-7	Z540x11-7	Z569x11-7	Z559x11-7	Z510x11-7	Z517x11-7
Class 2	Z640x11-7	Z640x11-7	Z669x11-7	Z659x11-7	Z610x11-7	Z617x11-7

See the last page for the article numbers

Standard styles

Standard	Size I – VII, Length standard and petite, topband standard and wide
Made-to-measure	available
Special models	available, see pages 56 – 73
Border	AD tricot border, narrow silicone dot topband AG Silicone dot topband, Silicone dot topband motif, Plain silicone topband rose, Silicone micro dot topband sensitive All topbands are standard styles.
Gusset	AT Comfort gusset
Foot part	open toe, closed toe, Shoe size 36 – 43

Material composition

	Elastane (EL)	Polyamide (PA)
CCL 1	32%	68%
CCL 2	35%	65%

medi tip:



medi Butler donning
made easier¹

medi skin and textile care
offer support

More information on the medi accessories from page 112

¹ In general, the following applies for donning and doffing aids: Rabe E et al. Medical compression therapy of the extremities with medical compression stockings (MCS), phlebological compression bandages (PCB), and medical adaptive compression systems (MAC): S2k guideline of the German Phlebology Society (DGP) in cooperation with the following professional associations: DDG, DGA, DGG, GDL, DGL, BVP. Hautarzt 2021;72(2):137-152.

Material

Medium

Compression class

CCL 1, 2

Standard colours

S = Standard
M = Made-to-measure

Black

S + M

Anthracite

S + M

Navy

S + M

Cashmere

S + M

Caramel

S + M

Rosé*

S + M

Bronze*

S + M

Beige*

S + M

* Delivery period up to 10 working days.



Available with sparkling crystals





mediven plus®

mediven plus has been established for over 40 years and caters for a variety of indications for men and women. The proven medical compression stocking in the compression classes 1, 2 and 3 with additional plus points can be relied upon, no matter what. True to the motto: Reliability that endures.

mediven plus. Compression has never proven so trustworthy.

mediven plus®

All you experience is reliability.
Compression has never proven so
trustworthy.

medi tip¹:

Due to the material thickness, the fit and the product characteristics, we recommend mediven plus for:

- Women / men
- Normal weight / overweight
- Light to heavy tendency to venous oedema
- Also for soft connective tissue


Indications²

- Chronic venous disease stages I to III according to Widmer or C0s – C5 according to CEAP
- Initial phase following varicose therapy
- Congestion problems and oedema during pregnancy
- Postoperative reperfusion oedema
- Post-traumatic, postoperative, occupational, medicinally induced oedema
- Oedema due to immobility
- Superficial venous thrombosis
- Deep leg vein thrombosis
- Condition after thrombosis, post-thrombotic syndrome
- Thrombosis prophylaxis in mobile patients
- Inflammatory dermatoses of the legs
- Nausea and dizziness in pregnancy

Features

- Very hard-wearing and opaque
- Optimal fit thanks to the plus heel, optimised toe, ribbed sole reinforcement and 5 cm wide cuff
- Especially recommended for heavier thighs
- Maxi panty top as standard version

Washing instructions

Easy-care 

Suitable for washing machine and drier.



Maxi panty top as standard



Plus-heel and sole reinforcement



5-cm-wide cuff

¹ Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer.

² Selecting the ideal compression stockings depends on the indication and the individual needs of the patients.

Product benefits



Clima Comfort



Clima Fresh



Variety



Perfect Fit



cf. Ben Ammar, F. (DermScan): Clinical evaluation of the skin compatibility of a medical device - application testing realized under dermatological control, Tunis 2012/2013.

mediven plus

Styles	AD	AD pte.	AG / NoB	AG / NoB pte.	AT	AT + maxi panty top	AT pte.	
	calf	calf petite	thigh with topband	thigh petite with topband	panty	panty with maxi panty top	panty petite	
Class 1	4A40x01-7	4A47x01-7	4A61x01-7	4A51x01-7	4A10x01-7	4AZ0x01-7	4A17x01-7	
Class 2	4B40x01-7	4B47x01-7	4B61x01-7	4B51x01-7	4B10x01-7	4BZ0x01-7	4B17x01-7	
Class 3	4C40x01-7	4C47x01-7	4C61x01-7	4C51x01-7	4C10x01-7	4CZ0x01-7	4C17x01-7	
Styles	ATE	ATE pte.		AT / U		AT / U pte.	CT	BT
	one legged panty	one legged panty petite		maternity panty		maternity panty petite	capri shorts	leggings
	right	left	right	left				
Class 1	4AE3x01-7	4AE2x01-7	4AV3x01-7	4AV2x01-7	4A20x01-7	4A27x01-7	made-to-measure	made-to-measure
Class 2	4BE3x01-7	4BE2x01-7	4BV3x01-7	4BV2x01-7	4B20x01-7	4B27x01-7		
Class 3	4CE3x01-7	4CE2x01-7	4CV3x01-7	4CV2x01-7	4C20x01-7	4C27x01-7		
Styles	ET	AT / H	AT / H pte.	AG	AG pte.			
	cycling short	men's leotard	men's leotard petite	thigh with waist attachment	thigh with waist attachment petite			
				right	left	right	left	
Class 1	made-to-measure	4A30x01-7	4A37x01-7	4A63x01-7	4A62x01-7	4A53x01-7	4A52x01-7	
Class 2		4B30x01-7	4B37x01-7	4B63x01-7	4B62x01-7	4B53x01-7	4B52x01-7	
Class 3		4C30x01-7	4C37x01-7	4C63x01-7	4C62x01-7	4C53x01-7	4C52x01-7	

See the last page for the article numbers

Material

Medium

Compression class

CCL 1, 2, 3

Standard colours

S = Standard

M = Made-to-measure

Black

S + M

Anthracite

S + M

Navy

S + M

Cashmere

S + M

Caramel

S + M

Rosé

S + M

Bronze

S + M

Beige

S + M

medi Magenta*

S + M

White*

S + M

* Delivery period up to 10 working days.



Available with sparkling crystals

Standard styles

Standard	Size I – VII, Length regular and short, topband regular and wide	
Made-to-measure	possible	
Special models	possible, see pages 56 – 73	
Border	AD	tricot border, narrow silicone dot topband
	AF and AG	Silicone dot topband, Silicone dot topband motif, Plain silicone topband rose, Silicone micro dot topband sensitive
	All topbands are standard styles	
Gusset	AT	Size I – III: Crotch gusset Size IV – VII: Narrow gusset, back to waist
	Maxi panty top:	Size I – III: narrow gusset, back to waist Size IV – V: wide gusset, back to waist Size VI – VII: extra-wide gusset, back to waist
	AT / U	with closed panty top
	AT / H	opening horizontal
Foot part	open toe, closed toe, shoe size 36 – 43	

Material composition

	Elastane (EL)	Polyamide (PA)
CCL 1	25%	75%
CCL 2	31%	69%
CCL 3	37%	63%

medi tip:



medi Butler donning made easier¹

medi skin and textile care offer support

More information on the medi accessories from page 112

¹ In general, the following applies for donning and doffing aids: Rabe E et al. Medical compression therapy of the extremities with medical compression stockings (MCS), phlebological compression bandages (PCB), and medical adaptive compression systems (MAC): S2k guideline of the German Phlebology Society (DGP) in cooperation with the following professional associations: DDG, DGA, DGG, GDL, DGL, BVP. Hautarzt 2021;72(2):137-152.





mediven active[®]

Men who move around a lot, for example for professional reasons, need a robust medical compression stocking. It must withstand high loads and fit perfectly in the shoe. The men's stocking in a sporty fine-ribbed look is very soft and under no circumstances can it be recognised as a medical compression stocking. Show your male customers that there are compression stockings that also look masculine.

mediven active. For the active man.



mediven active®

The hard wearing compression sock.

medi tip¹:

Due to the material thickness, the fit and the product characteristics, we recommend mediven active for:

- Men
- Normal weight / overweight
- Light to heavy tendency to venous oedema
- Also for soft connective tissue

Indications²

- Chronic venous disease stages I and II according to Widmer or COs – C4 according to CEAP
- Postoperative reperfusion oedema
- Post-traumatic, postoperative, occupational, medicinally induced oedema
- Oedema due to immobility
- Superficial venous thrombosis
- Condition after thrombosis, post-thrombotic syndrome
- Thrombosis prophylaxis in mobile patients
- Inflammatory dermatoses of the legs

Features

- Imperceptible and barely visible seam
- Fine-ribbed design – barely distinguishable from men's socks
- Comfort sole with heel reinforcement, withstands even the toughest conditions
- Soft and wide cuff for increased wearing comfort
- Very robust knitted fabric
- Suitable for shoe sizes 38 – 48

Washing instructions

Easy-care     

Suitable for washing machine and drier.



Foot tip without seam



Comfort sole

¹ Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer.

² Selecting the ideal compression stockings depends on the indication and the individual needs of the patients.

Product benefits



Clima Comfort



Perfect Fit



cf. Ben Ammar, F. (DermScan): Clinical evaluation of the skin compatibility of a medical device - application testing realized under dermatological control, Tunis 2012/2013.

Material

Strong

Compression class

CCL 1, 2

Standard colours

S = Standard



mediven active

Styles		AD	AD pte.
		calf	calf petite
Class 1	Article No.	3540x11-7	3547x11-7
Class 2	Article No.	3640x11-7	3647x11-7

See the last page for the article numbers

Standard styles

Standard	Size I – VII, Length standard and petite
Special models	not available
Border	AD Tricot border
Foot part	closed toe shoe size 38 – 48

Material composition

	Elastane (EL)	Polyamide (PA)
CCL 1	10%	90%
CCL 2	11%	89%

medi tip:



medi Butler donning made easier¹ medi skin and textile care offer support

More information on the medi accessories from page 112

¹ In general, the following applies for donning and doffing aids: Rabe E et al. Medical compression therapy of the extremities with medical compression stockings (MCS), phlebological compression bandages (PCB), and medical adaptive compression systems (MAC): S2k guideline of the German Phlebology Society (DGP) in cooperation with the following professional associations: DDG, DGA, DGG, GDL, DGL, BVP. Hautarzt 2021;72(2):137-152.



mediven® forte

This stocking keeps chronic indications in check. After all, it is all about ensuring that customers with significant vein issues actively participate in life while accepting as few compromises as possible.

The mediven forte delivers maximum pressure stability throughout the day. Its strong material achieves the required depth effect. The stocking is also very hard-wearing and durable.

mediven forte. Pleasant compression has never been so powerful.



mediven® forte

All you feel is protection.

Pleasant compression has never been so powerful.

medi tip¹:

Due to the material thickness, the fit and the product characteristics, we recommend mediven forte for:

- Women / men
- Overweight / obese
- Light to heavy tendency to venous oedema
- Also for soft connective tissue

Indications²

- Chronic venous disease stages I to III according to Widmer or C0s – C5 according to CEAP
- Initial phase following varicose therapy
- Functional venous insufficiency due to obesity
- Congestion problems and oedema during pregnancy
- Postoperative reperfusion oedema
- Post-traumatic, postoperative, occupational, medicinally induced oedema
- Oedema due to immobility
- Superficial venous thrombosis
- Deep leg vein thrombosis
- Condition after thrombosis, post-thrombotic syndrome
- Thrombosis prophylaxis in mobile patients
- Inflammatory dermatoses of the legs
- Nausea and dizziness in pregnancy

Features

- Extremely hard-wearing seamless material for optimum wearing comfort
- Maximum pressure stability (high working pressure for a deep functional effect)
- Stable soft cuff at the AD finish
- Opaque
- Clima Fresh over the whole leg for a feeling of freshness around the clock
- Two foot-lengths for series with foot tip:
 - Small foot up to 23 cm
 - Large foot from 23 cm

Washing instructions

Easy-care    

Suitable for washing machine and drier.



Open toe – detail



Double-knitted fabric



Stable soft cuff at the AD finish

¹ Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer.

² Selecting the ideal compression stockings depends on the indication and the individual needs of the patients.

Product benefits



Clima Fresh



Variety


















Perfect Fit



cf. Ben Ammar, F. (DermScan): Clinical evaluation of the skin compatibility of a medical device - application testing realized under dermatological control, Tunis 2012/2013.

mediven forte

Styles	AD	AD pte.	AG / NoB	AG pte./NoB	
					
	calf	calf petite	thigh	thigh petite	
Class 2	F240x01-7	F247x01-7	F260x01-7	F250x01-7	
Class 3	F340x01-7	F347x01-7	F360x01-7	F350x01-7	
Styles	AG	AG pte.			
					
	thigh with waist attachment	thigh petite with waist attachment			
	right	left	right	left	
Class 2	F263x01-7	F262x01-7	F253x01-7	F252x01-7	
Class 3	F363x01-7	F362x01-7	F353x01-7	F352x01-7	
Styles	AT	ATE	AT pte.	AT / U	BT
					
	panty	one legged panty	panty petite	maternity panty	leggings
Class 2	F210x01-7	made-to-measure	F217x01-7	made-to-measure	made-to-measure
Class 3	F310x01-7		F317x01-7	measure	measure
Styles	CT	ET	AT / H	AT / H pte.	
					
	capri shorts	cycling short	men's leotard	men's leotard petite	
Class 2	made-to-measure	made-to-measure	made-to-measure	made-to-measure	
Class 3	measure	measure	measure	measure	

See the last page for the article numbers

Material

Strong

Compression class

CCL 2, 3

Standard colours

S = Standard

M = Made-to-measure

Black

S + M

Anthracite

S + M

Navy

S + M

Cashmere

S + M

Caramel

S + M

Rosé*

S + M

Bronze*

S + M

Beige*

S + M

* Delivery period up to 10 working days.



Available with sparkling crystals

Standard styles

Standard	Size I – VII, Length regular and petite, topband regular and wide	
Made-to-measure	available	
Special model	available, see pages 56 – 73	
Border	AD	Tricot border, narrow silicone dot topband
	AF and AG	Tricot border, Silicone dot topband, Silicone dot topband motif, PI silicone topband rose, Silicone micro dot topband sensitive
	All topbands are standard styles	
Gusset	AT	Sizes I – III groin gusset Sizes IV – VII narrow gusset at the back up to the waist,
	AT / U	with closed panty top
	AT / H	opening horizontal
Foot part	open toe, closed toe	

Material composition

	Elastane (EL)	Polyamide (PA)
CCL 2	21%	79%
CCL 3	26%	74%

medi tip:



medi Butler donning made easier¹

medi skin and textile care offer support

More information on the medi accessories from page 112

¹ In general, the following applies for donning and doffing aids: Rabe E et al. Medical compression therapy of the extremities with medical compression stockings (MCS), phlebological compression bandages (PCB), and medical adaptive compression systems (MAC): S2k guideline of the German Phlebology Society (DGP) in cooperation with the following professional associations: DDG, DGA, DGG, GDL, DGL, BVP. Hautarzt 2021;72(2):137-152.



COMPRESSION for chronic venous disease and PAD / Diabetes?

Merino-Plush in the entire foot

Solid ribbed structure

Yes, certainly!*

mediven® angio – Safety down to the toes



Unique, soft padding from above the ankles down to the tip of the toes

Extra long, flat seam

mediven®[®] angi

Compression therapy in patients with venous disorders and concomitant diseases, have special requirements for a compression stocking. Therefore mediven angi was particularly developed and is the optimum solution for patients with chronic venous disease and concomitant peripheral arterial disease (PAD) and / or diabetes mellitus.

This compression stocking is unique. mediven angi is the first and only medical compression stocking whose safety was scientifically evaluated and proven in patients with chronic venous disease and concomitant mild to moderate PAD and / or diabetes mellitus.¹

mediven angi. Safety in treatment – down to the toes.

mediven[®] angio



Safety in treatment – down to the toes.

medi tip²:

Due to product features and the results of the clinical trial¹ we recommend mediven angio for:

- Women / men with chronic venous disease
- Concomitant mild to moderate peripheral arterial disease (PAD)
- Concomitant Diabetes mellitus

Indications³

- Chronic venous disease stages I bis III according to Widmer or C0s – C5 according to CEAP

Features

- Unique and soft merino-plush padding in the entire foot
- Natural merino wool for optimum thermoregulation and additional wearing comfort
- Extra long and flat seam clings to the toes and thus forms an anatomic tip shape for optimum pressure relief at the toes and their base joints
- Solid ribbed structure for an ideal balance of resting- and working pressure and as a visual guide for the correct application
- Micro-plush in the entire lower leg for a high wearing comfort
- 90°-heel ensures an even distribution of the knitted fabric in the heel and instep area

Washing instructions

Easy care

Suitable for washing machine and dryer



medi tip:



medi Butler donning
made easier⁴

medi skin and textile care
offer support

More information on the medi accessories from page 112

^{*} Product safety proven by clinical trial: The use of the medical compression stocking mediven angio in patients with chronic venous diseases and concomitant mild to moderate peripheral artery disease (PAD) and / or Diabetes mellitus is safe.¹

¹ Rother U et al. Safety of medical compression stockings in patients with diabetes mellitus or peripheral arterial disease. BMJ Open Diab Res Care 2020;8:e001316.

² Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer.

³ Selecting the ideal compression stockings depends on the indication and the individual needs of the patient.

⁴ In general the following applies for donning and doffing aids: Rabe E et al. Medical compression therapy of the extremities with medical compression stockings (MCS), phlebological compression bandages (PCB), and medical adaptive compression systems (MAC): S2k guideline of the German Phlebology Society (DGP) in cooperation with the following professional associations: DDG, DGA, DGG, GDL, DGL, BVP. Hautarzt 2021;72(2):137-152.

mediven[®] angio


The solution for patients with chronic venous disease and concomitant PAD and / or diabetes mellitus.



Extra wide cuff

Secure hold and large area pressure distribution

- ✓ Prevention of constrictions

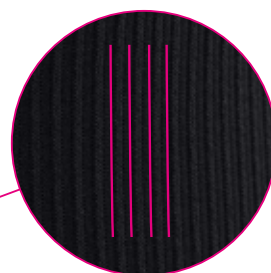

Solid ribbed structure

Easy-grip structure with high longitudinal elongation and fine plush material

- ✓ Easy to apply and comfortable to wear



Merino-Plush in the entire foot

Unique, soft padding from the tip of the toes up to above the ankle

- ✓ Reduction of pressure peaks in sensibility or skin disorders and optimum thermoregulation

Extra long, flat seam





Special seam which ends behind the base joints of the toes

- ✓ Optimum pressure relief of the toes and their base joints



mediven[®] angio

Safety in treatment – down to the toes.

mediven angio				
Styles	AD	AD	AD pte.	AD pte.
				
	calf small foot	calf large foot	calf petite small foot	calf petite large foot
CCL 1	3X40 Δ 11-7	3X40 Δ 21-7	3X47 Δ 11-7	3X47 Δ 21-7
CCL 2	3Y40 Δ 11-7	3Y40 Δ 21-7	3Y47 Δ 11-7	3Y47 Δ 21-7

Δ = Please add 1 for colour caramel and 5 for colour black
Please choose the size on the last digit of the article number from 1 to 7.

Standard styles	
Standard sizes	Size I – VII Length normal und petite
Made-to-measure	possible
Special models	not possible
Border	extra wide cuff
Foot part	closed toe small foot up to 25 cm $\hat{=}$ shoe sizes 36 – 40 large foot from 25 cm $\hat{=}$ shoe size 40 – 46

Material composition			
	Polyamide (PA)	Elastane (EL)	Merino wool
CCL 1	71 %	22 %	7 %
CCL 2	70 %	23 %	7 %




Material

Medium

Compression class

CCL 1, 2

Product benefits

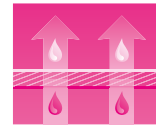

Clima Comfort



Clima Fresh



Perfect Fit



Variety



Standard colours

S = Standard

M = Made-to-measure*

Black
S+M

Caramel
S+M

* Delivery period up to 10 working days.

Note

For more information please visit:
medi.biz/angio

Unique novelty

mediven® angio – the first and only medical compression stocking whose safety was scientifically evaluated and proven in patients with chronic venous disease and concomitant PAD and / or diabetes mellitus.*

Unique product features

mediven angio was particularly developed for patients with chronic venous disease and mild to moderate PAD and / or diabetes mellitus and optimally adapted to their needs.

Unique safety

In a clinical trial¹ with both types of patients, the safety of mediven angio was evaluated and scientifically proven:

- ✓ Stable blood circulation down to the smallest blood vessels
- ✓ No damage to the skin
- ✓ No constrictions
- ✓ No pressure points
- ✓ High wearing comfort

Confirmed for compression classes 1 and 2



* Product safety proven by clinical trial: The use of the medical compression stocking mediven angio in patients with chronic venous diseases and concomitant mild to moderate peripheral artery disease (PAD) and / or Diabetes mellitus is safe.¹

¹ Rother U et al. Safety of medical compression stockings in patients with diabetes mellitus or peripheral arterial disease. BMJ Open Diab Res Care 2020;8:e001316.



medi[®] special models round knit

Special versions are all variants of the stocking fabrics that are available above and beyond the standard range of mediven fabrics. This overview will help you choose and offer the right add-on in order to provide the best solution for every unique patient and make it easier for you to hold active sales talks. Special versions of mediven take into account the individuality of patients with venous disease and ensure pleasant and comfortable wear combined with easier treatment. Compliance means high patient loyalty to treatment! Increase your chance of successful repeat sales and thus long-term customer loyalty.



Standard models

Foot tips

Open toe

medi tip:

- Open toe is available at the patient's request, if the patient prefers this



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Standard	Standard	Standard	Standard
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge

Closed toe

medi tip:

- Universal foot size: mediven elegance, mediven plus and mediven comfort
Shoe size 36 – 43
- Two foot sizes for mediven forte:
 - Small, up to 23 cm
 - Large, from 23 cm



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Standard	Standard	Standard	Standard
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge

Special models

Foot tips

Soft toe

medi tip:

- For customers who are sensitive in the toe region (e.g. hammer toe) and would, therefore, like to wear a particularly soft toe

Note: Due to the various knit materials, slight colour deviations may occur.



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Extra charge	Extra charge	Extra charge	Extra charge
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge

Hallux valgus toe section

medi tip:

- For customers who require freedom of movement for their toes



	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Extra charge	Extra charge	Extra charge	Extra charge
In case of modification	–	–	–	–

Anatomical toe

medi tip:

- For customers who are hypersensitive in the toe area
- Particularly well suited to narrow shoes (e.g. business shoes)

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Extra charge	Extra charge	Extra charge	Extra charge
In case of modification	—	—	—	—



Jersey foot tips can be lengthened or shortened according to specifications

medi tip:

- Individual supply according to customer-specific measurements

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge



Jersey toe
lengthened according to specifications



Jersey toe
shortened according to specifications

Jersey border at the foot lengthened according to specifications

medi tip:

- Individual supply according to customer-specific measurements

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge



Foot length shortened according to specifications

medi tip:

- Individual supply according to customer-specific measurements

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge



Anti-slip-segments

The anti-slip-segments offer more confidence. This makes walking in stockings safer and offer your patients enhanced shoe grip.

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Extra charge	Extra charge	Extra charge	Extra charge



Panty top


Pantyhose with compressive panty top

medi tip:

Secure hold and greater user comfort

- For customers who require a large panty top circumference
- For customers who have soft binding tissue
- With weakness of the abdominal muscles or diastases

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	–	–
Made-to-measure	Extra charge	Extra charge	–	Extra charge

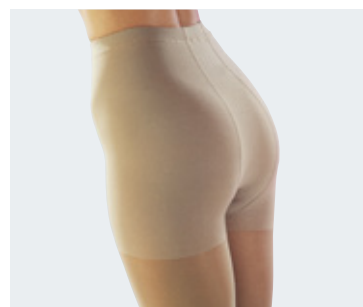

Panty top longer / shorter

medi tip:

Secure hold and greater user comfort

- Extended for customers with a stronger body or larger buttocks

medi tip:

Secure hold and greater user comfort

- Shortened for customers with less developed buttocks (waist-high wearing possible)

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	–	–
Made-to-measure	no surcharge	no surcharge	–	no surcharge
In case of modification	Extra charge	Extra charge	–	Extra charge



Panty top front / back, longer / shorter

medi tip:

Secure hold and greater user comfort

- Lengthened at the front or shortened at the back for customers with less developed buttocks, but a stronger body


medi tip:

Secure hold and greater user comfort

- Shortened at the front or lengthened at the back for customers with overly developed buttocks but less body

Note: The difference between the front and back panty top length can be a maximum of 9 cm.

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	–	Extra charge
Made-to-measure	no surcharge	no surcharge	–	no surcharge
In case of modification	Extra charge	Extra charge	–	Extra charge

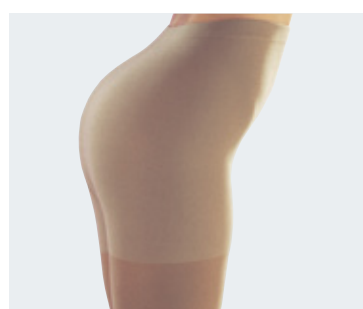

Pantyhose with open crotch

medi tip:

Enhanced user comfort and better suitability for everyday use

- For customers with highly restricted freedom of movement
- For incontinent patients
- For patients with permanent catheters
- For customers with gynaecological mycoses

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	–	–
Made-to-measure	no surcharge	no surcharge	–	no surcharge
In case of modification	Extra charge	Extra charge	–	Extra charge



Pantyhose with stoma opening

medi tip:

Enhanced user comfort and better suitability for everyday use

- For stoma patients

Please note:

- For initial treatment, first fit the trousers to the customer and then mark the stoma opening.
- Send the trousers to medi to work in the opening.
In this way it will be positioned exactly.
- Please send the “Template” in the event of a repeat treatment.

	mediven elegance	mediven plus	mediven comfort	mediven forte
Standard	Extra charge	Extra charge	Extra charge	Extra charge
Made-to-measure	Extra charge	Extra charge	Extra charge	Extra charge
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge




One-leg pantyhose

medi tip:

- As an alternative to hip fastening or topband

	mediven elegance	mediven plus*	mediven comfort	mediven forte*
Standard	–	Extra charge	–	–
Made-to-measure	–	Extra charge	–	Extra charge

* Surcharge regulation always based on the price AG with tricot border

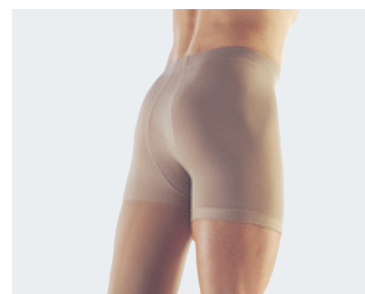

Standard models Topbands

Silicone dot topband, Silicone dot topband motif, Silicone micro dot topband sensitive, Plain silicone topband rose

Plain silicone topband rose (FBR)

medi tip: tried-and-tested, reliable grip thanks to flat coating. Elegant, feminine design. Fine, thin topband quality.

Silicone dot topband (NoB)

medi tip: tried-and-tested, reliable grip thanks to dotted structure. Good air circulation. Neutral design. Sturdy, robust topband quality.

Silicone dot topband motif (NoM)

medi tip: tried-and-tested, reliable grip thanks to nubbed structure. Good air circulation. Fashionable design. Sturdy, robust topband quality.

Silicone micro dot topband sensitive (MBs)

medi tip: barely noticeable, fine micro-dots on the reverse, for a reliable grip. Very good air circulation. Neutral design. Fine, thin topband quality.



Plain silicone topband rose (FBR)



Silicone dot topband (NoB)



Silicone dot topband motif (NoM)



Silicone micro dot topband sensitive (MBs)



Vgl. Ben Ammar, F. (DermScan):
Klinische Untersuchung der Haut-
verträglichkeit eines Medizinprodukts –
Anwendungstest unter dermatolo-
gischer Kontrolle, Tunis, 2012 / 2013.



Quelle:
Final Clinical Security Test –
Eurofins DERMSCAN,
Bucharest, 2019.

mediven elegance*	(FBR) Plain silicone topband rose	(NoB) Silicone dot topband	(NoM) Silicone dot topband motif	(MBs) Silicone micro dot topband sensitive
Standard	Standard	Standard	Standard	Standard
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge
mediven comfort*				
Standard	Standard	Standard	Standard	Standard
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge
mediven plus*				
Standard	Standard	Standard	Standard	Standard
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge
mediven forte*				
Standard	Standard	Standard	Standard	Standard
Made-to-measure	Standard	Standard	Standard	Standard
In case of modification	Extra charge	Extra charge	Extra charge	Extra charge

* Surcharge regulation always based on the price with topband

Waist attachment

medi tip:

- Secure hold and greater user comfort
- For customers with heavy or conical thighs
- For very soft thighs
- For allergy sufferers (alternative to silicone-coated topband)
- For customers who, due to the mechanical action of top bands, have a tendency to experience skin reactions

	mediven elegance	mediven plus*	mediven comfort	mediven forte*
Standard	–	Extra charge	–	Extra charge
Made-to-measure	–	Extra charge	–	Extra charge
In case of modification	–	Extra charge	–	Extra charge


* Surcharge regulation always based on the price AG with tricot border



Stocking suspenders

Stocking suspenders with strap, adjustable with 2 or 4 suspenders. Can be used with all styles.

One-sided in the sizes	1	2	3
Waist	80 – 90 cm	95 – 105 cm	110 – 120 cm
Double-sided in the sizes	1	2	3
Waist	80 – 90 cm	95 – 105 cm	110 – 120 cm

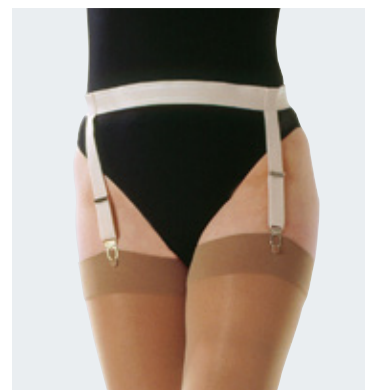

Yard ware (1 m, 5 m)

Can be ordered as required.

- Silicone dot topband (NoB)
- Silicone dot topband motif (NoM)
- Plain silicone topband rose (FBR)
- Silicone micro dot topband sensitive (MBs)

In all colours, composition: silicone, polyamide, polyester, elastane





mediven[®]

Gusset variants

Gussets are knitted into the groin area of compression pantyhose. The following pages will give you an overview of the “standard gusset” that is used in all the round knit models.

In addition to this, you can choose from further gusset variants for standard sizes and made-to-measure garments for your customers, since a size in centimetres says little about the anatomy of patients with venous disease or the relationship between abdomen and bottom. Besides the pictures, a description simplifies the correct choice of gusset. The optimum compression pantyhose gives the wearer absolute comfort in the panty top.





mediven® Gusset standard variants

Standard Gusset	mediven elegance	mediven plus	mediven forte	mediven comfort
Comfort gusset	Sizes I – VII cT 60 – 125 cH 80 – 145			Sizes I – VII cT 60 – 125 cH 80 – 135
Crotch gusset Small abdomen and small bottom For women with a less pronounced abdomen and bottom		Sizes I – III cT 60 – 95 cH 80 – 110	Sizes I – III cT 60 – 95 cH 80 – 110	
Narrow gusset at the back Small abdomen and large bottom For women with a pronounced bottom, but a relatively flat abdomen		Sizes IV – VII cT 80 – 125 cH 100 – 145 Maxi panty top Sizes I – III cT 95 – 140 cH 110 – 150	Sizes IV – VII cT 80 – 125 cH 100 – 145	
Wide gusset at the back Large abdomen and very large bottom For women with a pronounced bottom but a comparatively flat abdomen		Maxi panty top Sizes IV – V cT 120 – 150 cH 135 – 160		
Extra wide gusset at the back Very large abdomen and very large bottom For women with a pronounced bottom and abdomen		Maxi panty top Sizes VI – VII cT 140 – 165 cH 145 – 170		
Maternity panty AT / U, closed Growing abdomen Suitable for pregnant women with a large, growing abdominal circumference	Sizes I – VII cT from 60 cH from 80	Sizes I – VII cT from 60 cH from 80		
Men's leotard, horizontal fly For men with fly		Sizes I – VII cT 60 – 125 cH 80 – 145		

Comfort gusset

Standard gusset




Standard model	
Models	mediven elegance, mediven comfort
Sizes	I – VII
Made-to-measure	
Models	mediven elegance, mediven comfort
Please order	"Comfort crotch"

Crotch gusset

Standard gusset

For women with a small abdomen and bottom.

Standard model	
Models	mediven plus, mediven forte
Sizes	I – III
Made-to-measure	
Models	mediven plus, mediven forte
Please order	"Small gusset in crotch only"

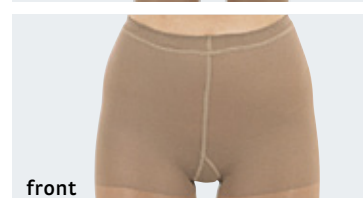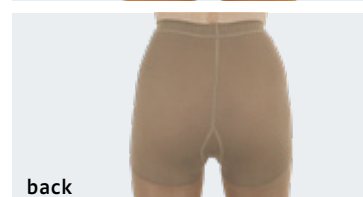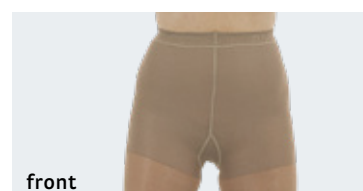

Maxi-Panty top

For women with large hip and waist circumferences.

Special model

Models	mediven plus
Sizes	I – VII
Please order	“Maxi-Panty top”

Narrow gusset, up to the waist at the back


For women with large bottoms and a relatively flat abdomen.

Made-to-measure

Models	mediven elegance, mediven plus, mediven forte
--------	---

Special model

Models	mediven elegance, Size IV – VII mediven plus, Size I – III
Please order	“Narrow gusset up to waist at back” Can also be incorporated in the AT / U. Caution: this needs a transverse seam in the anterior crotch area.

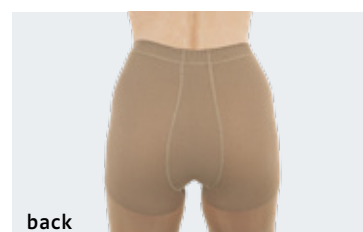

Narrow gusset, up to the waist at the front


For women with a large abdomen and a relatively flat bottom.

Made-to-measure

Models	mediven elegance, mediven plus, mediven forte
--------	---

Special model

Models	mediven elegance, Size I – III mediven plus, Size I – VII
Please order	“Narrow gusset, up to the waist at the front”

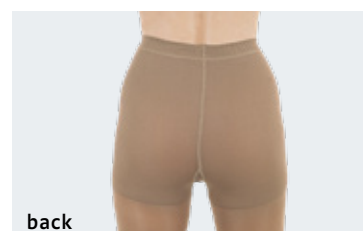

Wide gusset, up to the waist at the back

For women with a far more pronounced bottom and, in contrast, a flat abdomen.

Made-to-measure

Models mediven elegance, mediven plus, mediven forte, mediven comfort


Special model

Models mediven elegance, mediven plus, mediven comfort

Please order “Wide gusset, up to waist at back”
Can also be incorporated in the AT / U. **Caution:** this needs a transverse seam in the anterior crotch area.



front

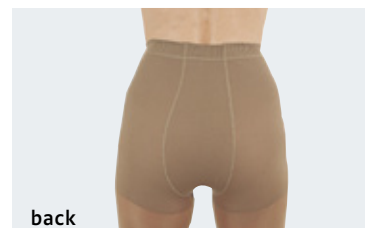

back

Wide gusset, up to the waist at the front

For women with a pronounced abdomen and a relatively flat bottom.

Made-to-measure

Models mediven elegance, mediven plus, mediven forte, mediven comfort


Special model

Models mediven elegance, mediven plus, mediven comfort

Please order “Wide gusset, up to the waist at the front”



front

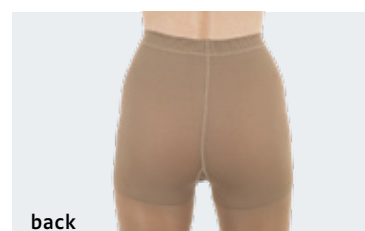

back

Variable gusset, back

For women with a narrow waist, a far more pronounced bottom and, in contrast, a flat abdomen.

Made-to-measure

Models mediven elegance, mediven plus, mediven forte


Special model

Models mediven elegance, mediven plus

Please order “Variable gusset, back”



front

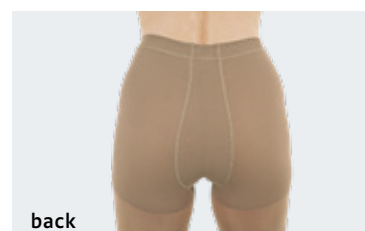

back

Variable gusset, front

For women with a narrow waist, a far more pronounced abdomen and, in contrast, a flat bottom.


Made-to-measure

Models mediven elegance, mediven plus, mediven forte

Special model

Models mediven elegance, mediven plus

Please order "Variable gusset, front"

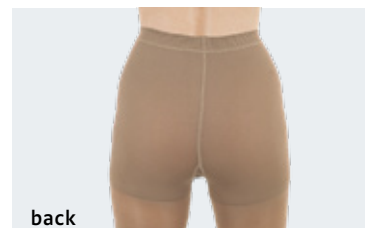

Continuous gusset, narrow

For women who are sensitive to pressure and have a slightly larger bottom and abdomen.


Made-to-measure

Models mediven elegance, mediven plus, mediven forte

Special model

Models mediven elegance, mediven plus

Please order "Continuous gusset, narrow"

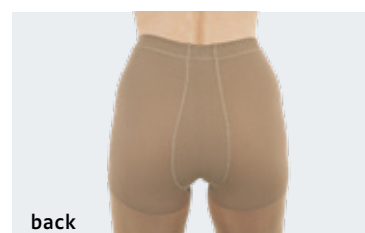

Continuous gusset, front narrow, back wide

For women with a large abdomen and a particularly pronounced bottom.



Made-to-measure

Models mediven elegance, mediven plus, mediven forte

Special model

Models mediven elegance, mediven plus

Please order "Continuous gusset, front narrow, back wide"

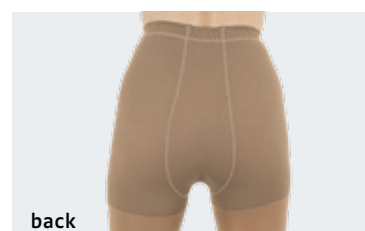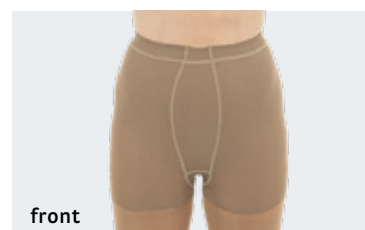

Continuous gusset, front wide,
back narrow

For women with a large abdominal circumference and a particularly pronounced bottom.

Made-to-measure	
Models	mediven elegance, mediven plus, mediven forte
Special model	
Models	mediven elegance, mediven plus
Please order	“Continuous gusset, front wide and back narrow”



Continuous gusset, wide

For women with large abdomen and hips.

Made-to-measure	
Models	mediven elegance, mediven plus, mediven forte, mediven comfort
Special model	
Models	mediven elegance, mediven plus, mediven comfort
Please order	“Continuous gusset, wide”



Maternity panty AT / U, open

For pregnant women with large and growing abdominal circumferences and tenderness to pressure.

Made-to-measure	
Models	mediven elegance, mediven plus, mediven forte
Special model	
Models	mediven elegance, mediven plus
Please order	“Maternity panty AT / U, open”



Maternity panty AT / U, closed

Standard gusset

For pregnant women with large and growing abdominal circumferences.

Standard model

Models mediven elegance, mediven plus

Sizes I – VII


Made-to-measure

Models mediven elegance, mediven forte

Please order “Maternity panty AT / U, closed”
The AT / U is possible with a narrow (No. 4) or with a wide (No. 6) gusset up to the waist at the back.
Caution: this needs a transverse seam in the anterior gusset area.



front

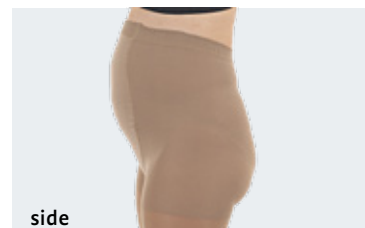

side

Maternity panty AT / U, closed

Standard gusset

For pregnant women with large and growing abdominal circumferences.

- Soft and supple gusset, the material is visually brighter
- Pantytop has more stretch for a better fit

Standard model


Models mediven plus

Sizes I – VII


Made-to-measure

Models mediven elegance, mediven forte

Please order “Maternity panty AT / U, closed”
The AT / U is possible with a narrow (No. 4) or with a wide (No. 6) gusset up to the waist at the back.
Caution: this needs a transverse seam in the anterior gusset area.

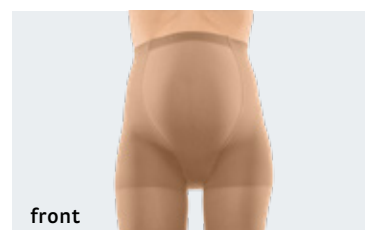

front

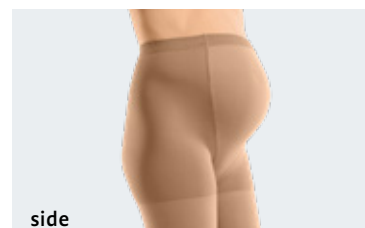

side

Men's gusset with vertical / horizontal fly

For men, with fly.

Standard model

Models mediven plus (standard horizontal fly)

Sizes I – VII


Made-to-measure

Models mediven plus, mediven forte

Please order “Men's crotch with horizontal fly” or
“Men's crotch with vertical fly”



front

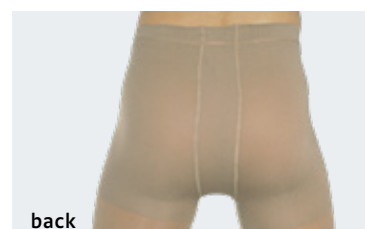

back

mediven[®] ulcer kit

Therapy for leg ulcer requires a sophisticated therapy concept. With the mediven ulcer kit patients suffering from a leg ulcer can be treated efficiently. The ulcer kit consists of two components worn on top of each other: the mediven ulcer and mediven ulcer plus. The liner as well as the outer stocking each provide 20 mmHg and thus allow for a therapeutically effective compression therapy day and night.





mediven[®] ulcer kit

Ulcer stocking system with 2 x 20 mmHg for effective compression therapy for up to 24 hours a day.^{1,2}

medi tip^{*}:

Due to product features we recommend mediven ulcer kit for:

- Women / Men
- After initial oedema decongestion
- Suitable for 24 hour compression therapy





Indications²





- Therapy of venous leg ulcer (CEAP Classification C6)
- Therapy for mixed ulcers (due to arterial and venous issues), taking the contraindications into account
- Prevention of recurrence of venous leg ulcer (CEAP Classification C5)

Features

- Already offers effective therapy overnight^{1,2}
- Easy donning liner stocking material makes application of outer stocking easier³
- Antibacterial, odour reducing effect of the liner stocking allows for optimum hygiene
- Refined appearance
- Pleasant wearing comfort whilst good durability
- Fulfills the EWMA** requirements regarding therapeutical necessary pressure (40 mmHg)⁴
- High wall stability thanks to double layer

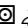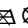Washing instructions

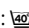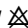Liner stocking:    

Outer Stocking:    

Suitable for washing machine and drier.



Note

For more information and donning video please visit:
www.medi.biz/ulcerkit

* Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer

1 Partsch H et al. Compression stockings for treating venous leg ulcers: measurement of interface pressure under a new ulcer kit. Phlebology 2008;23(1):40-46.

2 Wolff O et al. The effect of compression ulcer stockings on the capillary filtration rate and the formation of oedema. Phlebologie 2011;5:245-250.

3 Tickle J. Managing venous leg ulcers and oedema using compression hosiery. Nursing Standard 2015;30(8):57-63.

4 Franks P et al. Management of Patients with Venous Leg Ulcer Challenges and Current Best Practice, J Wound Care 2016;25 Suppl 6:1-67.

** European Wound Management Association

Therapy for the underlying disease mediven® ulcer kit

Ulcer stocking system with 2 x 20 mmHg for effective compression therapy for up to 24 hours a day.^{1,2}

Liner stocking mediven® ulcer

NEW

Therapeutically effective compression with 20 mmHg during the night.^{1,2}

Flat topband will stay in place without constriction

Soft material with antibacterial effect for pleasant wearing comfort and easy donning

Sewn-in instep area allows for optimum fit and reduces wrinkling when foot is bent

Special heel construction offers optimum pressure relief and colour marking for easy positioning

Outer stocking mediven® ulcer plus

Additional compression of 20 mmHg during the day.

Wide topband ensures optimum wearing comfort

Medium material strength for comfortable yet effective working pressure

Comfortable plus-heel and reinforced foot sole for good durability

Also available as closed toe

mediven[®] ulcer kit

Ulcer stocking system with 2 x 20 mmHg for effective compression therapy for up to 24 hours a day.^{1,2}

Delivery contents

- 2 mediven ulcer liner stockings
- 1 mediven ulcer plus outer stocking
- 1 easy-on-slipper donning aid (standard item for mediven ulcer kit open toe)

mediven ulcer kit



Article	Art.-No.	
closed toe	petite	regular
mediven ulcer kit	381708Δ000	381008Δ000
open toe		
mediven ulcer kit	381707Δ000	381007Δ000
Refill (2 pieces per PU)		
mediven ulcer Refill	8X4718Δ000	8X4018Δ000

Δ = Please add size 1 - 7


Compression

2 x 20 mmHg

Product benefits



*These features only apply to mediven ulcer

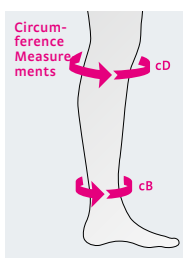

Measurements / Size table		Size I	Size II	Size III	Size IV	Size V	Size VI	Size VII
Leg measurements (in cm)	Ankle cB	18 – 20	20 – 22	22 – 24	24 – 26	26 – 28	28 – 30	30 – 32
	Below knee cD	27 – 33	29 – 36	32 – 39	34 – 42	36 – 45	38 – 48	40 – 50
Colour marking	Heel	pink	blue	yellow	red	grey	green	brown
Length (in cm)		petite		regular				
ID		34 – 38		39 – 44				

¹ Partsch B et al.: Compression stockings for treating venous leg ulcers: measurement of interface pressure under a new ulcer kit, Phlebology 2008, 23(1), S. 40–46.
² Wolff, O., et al.: The effect of compression ulcer stockings on the capillary filtration rate and the formation of oedema, Phlebologie 5/2011, S. 245–250.







Basic care of medi

duomed® and duomed smooth are the economic solutions for a reliable compression therapy. The cooperative partnership with medi enables you to address all customer needs and cover the complete product range of basic care, mediven quality products and premium special models. Simply obtain everything from medi through one provider.

The economic solution for venous problems.

medi tip¹:

Due to the material thickness, the fit and the product characteristics, we recommend duomed for:

- Women / men
- Overweight / obese
- Light to heavy tendency to venous oedema
- Also for soft connective tissue

Indications²


- Chronic venous disorders: C0s to C4 according to CEAP
- Initial phase after varicose vein therapy
- Oedema during pregnancy
- Postoperative reperfusion oedema
- Post-traumatic, post-operative, occupational, drug-related oedema
- Oedema due to immobility
- Superficial venous thrombosis
- Condition after thrombosis, postthrombotic syndrome
- Thrombosis prophylaxis in mobile patients
- Inflammatory dermatoses of the legs
- Sickness, dizziness during pregnancy

Features

- Combines ideal wearing properties and appropriate medical compression
- Your economic solution for a reliable compression therapy



Knee length stocking (AD) with closed toe

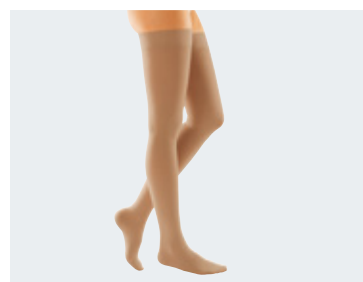

Thigh length stocking (AC) with closed toe



Panty (AT) with open toe

¹ Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer.

² Selecting the ideal compression stockings depends on the indication and the individual needs of the patients.

Product benefits



Clima Fresh



Material

Medium

Compression class

CCL 1, 2

Standard colours

S = Standard

Black
s

Beige
s

duomed					
Styles	open toe	AD	AD pte.	AG	AG pte.
		calf	calf petite	thigh	thigh petite
Class 1	Article No.	V140x02-6	V147x02-6	V160x02-6	V150x02-6
Class 2	Article No.	V240x02-6	V247x02-6	V260x02-6	V250x02-6
Styles	open toe	AG / SN	AG pte. / SN	AT	AT pte.
		thigh with topband	thigh petite with topband	panty	panty petite
Class 1	Article No.	V161x02-6	V151x02-6	V110x02-6	V117x02-6
Class 2	Article No.	V261x02-6	V251x02-6	V210x02-6	V217x02-6
		AG		AG pte.	
		thigh with waist attachment right	thigh petite with waist attachment left		
Class 1	Article No.	V163x02-6	V162x02-6	V153x02-6	V152x02-6
Class 2	Article No.	V263x02-6	V262x02-6	V253x02-6	V252x02-6
Styles	closed toe	AD	AD pte.	AG	AG pte.
		calf	calf petite	thigh	thigh petite
Class 1	Article No.	V140x12-6	V147x12-6	—	—
Class 2	Article No.	V240x12-6	V247x12-6	—	—
Styles	closed toe	AG / SN	AG pte. / SN	AT	AT pte.
		thigh with topband	thigh petite with topband	panty	panty petite
Class 1	Article No.	V161x12-6		V110x12-6	V117x12-6
Class 2	Article No.	V261x12-6		V210x12-6	V217x12-6

See the last page for the article numbers


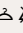
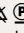


Standard styles

Standard sizes	S – XXL
Made -to-measure	not available
Special models	not available
Border	–
Gusset	–
Foot part	With open or closed toe

Material composition


	Elastane (EL)	Polyamide (PA)
CCL 1	37 %	63 %
CCL 2	43 %	57 %

Washing instructions

Wash-and-wear     
Suitable for washing machine and drier

Measuring duomed

Size	cB Ankle measurement	cC Calf measurement	cG Thigh measurement	Measurements for silicone topband
S	19 – 21	28 – 34	42 – 57	47 – 54
M	22 – 24	32 – 38	48 – 64	52 – 60
L	25 – 27	36 – 42	54 – 71	56 – 65
XL	28 – 30	40 – 46	60 – 78	62 – 72
XXL	30 – 32	42 – 50	65 – 85	65 – 76

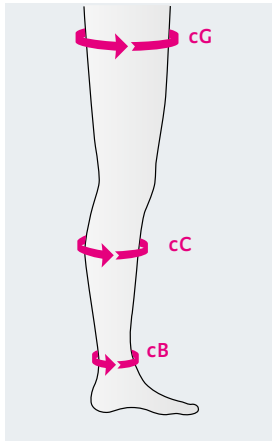

Length

ℓG regular	72 – 83
ℓG petite	62 – 71
ℓD regular	39 – 44
ℓD petite	34 – 38

Toe

For all closed toe stockings:
only one highly elastic foot size



duomed smooth

Shining beauty for your legs.

medi tip¹:

Due to the material thickness, the fit and the product characteristics, we recommend duomed smooth for:


- Women
- Normal weight
- No / light tendency to venous oedema

Indications²

- Chronic venous disorders: COs – C3 according to CEAP
- Postoperative reperfusion oedema, post-traumatic, postoperative, occupational oedema, medically induced oedema
- Oedema due to immobility
- Initial phase following varicose vein therapy
- Superficial venous thrombosis
- Thrombosis prophylaxis in mobile patients
- Inflammatory dermatoses of the legs
- Nausea and dizziness in pregnancy

Features

- Economic and trendy
- Appropriate medical compression
- Smooth and slightly shiny material

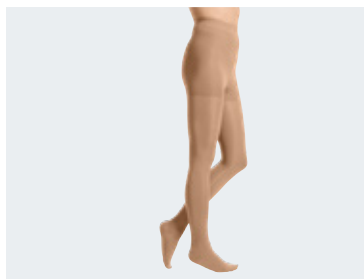

Pantyhose



Thigh-length stocking



Open toe



Below-knee stocking

¹ Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer.

² Selecting the ideal compression stockings depends on the indication and the individual needs of the patients.

Product benefits



Clima Comfort

Clima Fresh



Material

Light

Compression class

CCL 1, 2

Standard colours

Black

Caramel

duomed smooth

Styles	open toe	AD pte.	AD	AG pte.	AG	AT pte.	AT
	calf petite	calf	thigh petite with topband	thigh with topband	panty petite	panty	

Class 1 Article No. XV47_0_ XV40_0_ XV57_0_ XV61_0_ XV17_0_ XV10_0_

Class 2 Article No. XW47_0_ XW40_0_ XW57_0_ XW61_0_ XW17_0_ XW10_0_

Styles	closed toe	AD pte.	AD	AG pte.	AG	AT pte.	AT
	calf petite	calf	thigh petite with topband	thigh with topband	panty petite	panty	

Class 1 Article No. XV47_1_ XV40_1_ XV57_1_ XV61_1_ XV17_1_ XV10_1_


Class 2 Article No. XW47_1_ XW40_1_ XW57_1_ XW61_1_ XW17_1_ XW10_1_

Standard styles

Standard sizes	S – XXL
Made-to-measure	not available
Special models	not available

Material composition

	Polyamide (PA)	Elastane (EL)
Class 1	57 %	43 %
Class 2	51 %	49 %

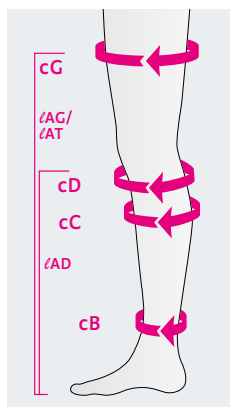

Circumference in cm

Sizes	cB Ankle	cC Calf	cG Thigh	silicone topband
S	19 – 21	28 – 34	42 – 57	47 – 54
M	22 – 24	32 – 38	48 – 64	52 – 60
L	25 – 27	36 – 42	54 – 71	56 – 65
XL	28 – 30	40 – 46	60 – 78	62 – 72
XXL	30 – 32	42 – 50	64 – 85	65 – 76

Length in cm

Length	AD Calf stocking	AG Thigh stocking
petite	34 – 38	62 – 71
standard	39 – 44	72 – 83



mediven harmony[®]

The comfortable “no seam” armsleeve is suitable for less severe, post-traumatic oedema.



mediven harmony®

Comfortable round-knit armsleeve.





Indications³

- Shoulder arm vein thrombosis
- Postoperative and post-traumatic oedema (please note: not suitable for pronounced oedema!)

Features


- The double ellipses on the elbow point and in the crease of the elbow offer freedom of movement. This reduces wrinkles to a minimum to prevent soreness.
- The anatomical shape of the armsleeve ensures easy and correct donning and doffing
- The silicone topband is knitted to the armsleeve with a flat and inconspicuous seam which secures hold whilst retaining air permeability.
- The skin feels comfortable all year round with Clima Comfort
- Added lanolin²

Washing instructions

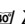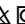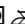Easy-care     Suitable for the washing machine and tumble dryer



Handpiece

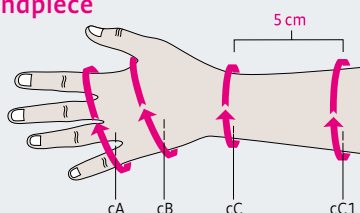

Available variants

AC1 handpiece
fingerless, with
thumb piece

AC1 seamless
glove with
compressive fingers



without fingers

CCL 1: Art.No.:
2Y90x02-7000

CCL 2: Art.No.:
2Y90x02-7000

with fingers

CCL 1: Art.No.:
2Z90x02-7000

CCL 2: Art.No.:
2Z91x02-7000


Material composition handpiece

	Elastane (EL)	Polyamide (PA)
CCL 1	18%	82%
CCL 2	22%	78%

Circumference / Handpiece (cm)

Sizes	II	III	IV	V	VI	VII
cC1*	15.0 – 18.0	17.0 – 20.0	19.0 – 21.5	21.0 – 23.5	23.0 – 26.0	25.5 – 30.0
cC	13.5 – 15.5	15.0 – 18.0	17.0 – 20.0	19.0 – 21.0	20.0 – 23.5	22.5 – 27.5
cB	15.5 – 20.0	17.5 – 21.0	19.5 – 23.0	20.5 – 24.0	22.5 – 26.0	24.5 – 30.0
cA	15.0 – 19.0	17.0 – 20.0	19.0 – 22.0	20.0 – 23.0	22.0 – 25.0	24.0 – 29.0

medi tip:

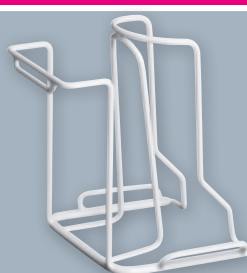

medi Butler donning made easier¹

medi skin and textile care offer support

More information on the medi accessories from page 112

* Measuring point cC1 located 5 cm above cC

¹ In general, the following applies for donning and doffing aids: Rabe E et al. S2k Guideline: Medizinische Kompressionstherapie der Extremitäten mit Medizinischem Kompressionsstrumpf (MKS), Phlebologischem Kompressionsverband (PKV) und Medizinischen adaptiven Kompressionssystemen (MAK). (Medical Compression Therapy of the Extremities with Medical Compression Stockings (MCS), Phlebological Compression Bandage (PCB) and Medical Adaptive Compression Systems (MAC)). Published online at: <https://www.awmf.org/leitlinien/detail/II/037-005.html> (Last accessed: 02.03.2020)

² Natural substance to care dry and rough skin, used in many ways, for example in child and baby care.

³ Please note that this recommendation does not replace a doctor's diagnosis or professional advice from a specialist medical retailer.

Product benefits



Clima Comfort




Perfect Fit



Double ellipses



Armsleeve

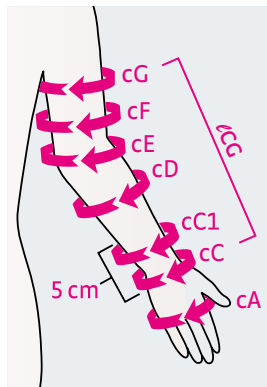

Measurement points mediven harmony

Standard styles

Colours	Sand, Caramel
Standard sizes	I – VII, Standard and extra-wide
Made-to-measure	not possible
Border	Dotted topband, narrow, 2.5 cm
Handpiece	Fingerless, with thumb piece

Material composition

Arm care	Elastane (EL)	Polyamide (PA)
CCL 1	34%	66%
CCL 2	43%	57%

Length (cm)

Standard length	41 - 47
-----------------	---------

Compression class

CCL 1, 2

Standard colours

S = Standard

Sand
s


Caramel
s

Black
s

Versions (all versions available in standard and extra-wide)

AG armsleeve with integrated hand piece and topband cuff

CG armsleeve with topband cuff

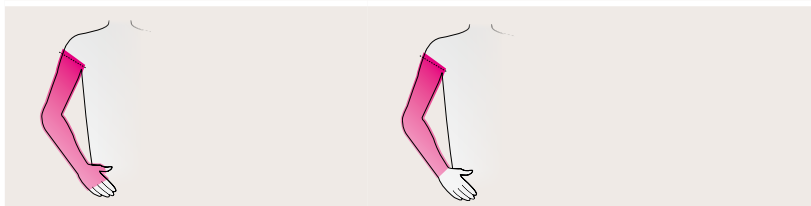

Armsleeve	Size I		Size II		Size I – III		Size IV	
Circumference (cm)	Standard	Extra-wide	Standard	Extra-wide	Standard	Extra-wide	Standard	Extra-wide
cG	25.5 – 28.0	28.5 – 31.0	28.0 – 30.5	30.5 – 33.5	30.0 – 33.0	33.0 – 36.0	32.0 – 35.0	35.0 – 38.5
cF	23.5 – 25.5	25.5 – 28.0	25.0 – 27.5	27.5 – 30.0	27.5 – 30.0	30.0 – 33.0	29.5 – 32.0	32.0 – 35.0
cE	23.0 – 25.0	24.0 – 26.0	24.5 – 26.5	26.5 – 29.0	26.0 – 28.5	28.5 – 31.0	27.5 – 30.0	30.0 – 33.0
cD	22.0 – 24.0	24.0 – 26.0	23.0 – 25.0	25.0 – 27.5	24.0 – 26.0	26.0 – 28.5	25.0 – 27.5	28.0 – 30.5
cC1 = 100 %*	15.0 – 16.5		16.5 – 18.0		18.0 – 19.5		19.5 – 21.0	
cC	13.5 – 14.5		14.5 – 15.5		16.0 – 17.5		17.5 – 19.0	
cA	15.0 – 18.0		16.0 – 19.0		17.0 – 20.0		19.0 – 22.0	
Armsleeve	Size V		Size VI		Size VII			
Circumference (cm)	Standard	Extra-wide	Standard	Extra-wide	Standard	Extra-wide		
cG	33.5 – 36.5	37.0 – 40.5	36.0 – 39.5	39.5 – 43.0	38.0 – 41.5	41.5 – 45.5		
cF	31.0 – 34.0	34.0 – 37.0	33.0 – 36.0	36.0 – 39.5	35.0 – 38.5	38.5 – 42.0		
cE	29.0 – 31.5	31.5 – 34.5	30.5 – 33.5	33.5 – 36.5	32.5 – 35.5	35.5 – 39.0		
cD	27.0 – 29.5	29.5 – 32.0	28.5 – 31.0	31.0 – 34.0	30.5 – 33.5	33.5 – 36.5		
cC1 = 100 %*	21.0 – 23.0		23.0 – 25.0		25.0 – 27.5			
cC	19.0 – 20.5		20.5 – 22.5		22.5 – 24.5			
cA	20.0 – 23.0		22.0 – 25.0		24.0 – 27.0			

* Measuring point cC1 located 5 cm above cC





circaid[®] adjustable compression

Adjustable compression garments for the treatment of venous diseases.

circaid garments are a compressive solution for those patients who have challenges with the established medical compression stockings. Its easy donning and doffing process promotes self-management and thus an increased patient concordance as well as an improved quality of life. The ability to instantly readjust the circaid system helps to consistently apply compression to a limb and avoids sliding down of the garment for better treatment outcomes and satisfied patients.



Compression therapy by the specialist

**medi offers the most comprehensive line
of compression garments for individual needs.**

The inelastic range of circaid is the cost-effective alternative to short stretch and multi-layer bandage systems. circaid products guarantee a constant compression level and feature a fast and easy donning and doffing system. The ability to instantly readjust the circaid system helps to consistently apply compression to a reducing limb. This is not possible with bandaging without completely removing the layers. circaid garments promote self-management and thus an increased patient compliance as well as an improved quality of life.



Inspiration by nature.

circaid products have an animal role model: the giraffe with its specific skin structure. Giraffes have venous pressures of more than 250 mmHg at their front and hind legs – three times more than humans. But they do not suffer from lymphoedema or venous diseases, even though they are on their feet the whole day and sometimes also sleep while standing. But why? In contrast to humans, giraffes have inelastic skin, meaning it does not stretch.* The founder of circaid was inspired from this “natural compression” and designed the first patented inelastic circaid leg garment. This first garment has been further developed to today’s circaid products (short for circulation aid).

* Sathar F, Ludo Badlangana N, Manger PR. Variations in the thickness and composition of the skin of the giraffe. Anatomical record 2010;293(9):1615-1627.

circaid® features

Overview of circaid characteristics.



Built-In-Pressure

Built-In-Pressure® system

To set, control and re-adjust the correct therapeutic compression range



Breathe-O-Prene

Breathe-O-Prene

- Breathable and latex-free material
- Odour-inhibiting with antibacterial effect



Juxtaposed band

Juxtaposed band system

- Optimum fit thanks to interlocking bands
- Each strap can be readjusted while on the limb



Variety

Variety

- Wide range of care options
- Many sizes available



Easy handling

Easy handling

- Simple donning and doffing
- Eases handling for the patient



Inelastic

Inelastic

- Inelastic material for an effective therapy and pleasant wearing comfort
- Inspired by nature



All circaid products are latex-free.



circaid® Built-In-Pressure System (BPS)

circaid inelastic compression garments are engineered with the ability to be set to precise compression ranges with a graduated compression profile from the ankle up the leg.

The user-friendly design of the circaid Built-In-Pressure System (BPS) enables patients to achieve the necessary compression ranges needed for the treatment and management of chronic venous disease.

With the circaid Built-In-Pressure System (BPS) guide card patients are able to set their prescribed compression range while donning their garment and to readjust the compression range during the day as needed. Therefore, the appropriate compression range is ensured throughout the period of wear. Simultaneously, it is very simple to verify the correct range of applied pressure.





circaid[®] juxtalite[®]

Adjustable compression garment for treating venous diseases after initial oedema decongestion.

medi tip*:

Due to product features we recommend circaid juxtalite for:

- Women / men
- Patients who have challenges with medical compression stockings
- 24 hour compression therapy
- Mobile / immobile patients

Indications

- Chronic venous insufficiency
- Varicose veins
- Post Thrombotic Syndrome
- Venous Stasis Ulcer / Mixed Leg Ulcer
- Post Sclerotherapy
- Dependent oedema

Product features

- Easy handling – simple donning and doffing
- circaid Built-In-Pressure System (BPS): adjustable, controllable and re-adjustable compression
- Breathe-O-Prene: breathable material with high working pressure
- Latex-free, odour-inhibiting material with anti-bacterial effect
- Re-adjustability of the individual bands avoids pressure loss and thus sliding down of the garment^{1,2}
- **New:** Also available with new compressive undersock
 - Applies graduated compression to the foot and ankle
 - Graduated compression continues into the leg portion of the undersock

Benefits

- Offering an alternative therapy concept for patients who are not able to apply or tolerate ulcer compression stockings
- Faster healing rates of venous leg ulcer^{1,2}
- Promotes self-management thanks to easy handling¹


Adjustable compression ranges

20 – 30 mmHg

30 – 40 mmHg

40 – 50 mmHg

Product benefits

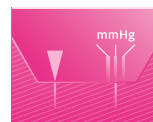

Built-In-Pressure



Breathe-O-Prene



Juxtaposed band



Variety



Easy handling



Inelastic

Colours

S = Standard



Note

* Please consider, that this recommendation does not replace the diagnosis and consultation of a medical healthcare professional.

Studies

¹ Mosti G et al. Adjustable compression wrap devices are cheaper and more effective than inelastic Bandages for venous leg ulcer healing. A Multicentric Italian Randomized Clinical Experience. Phlebology 2020;35(2):124-133.

² Stather PW et al. Review of adjustable velcro wrap devices for venous ulceration. In Wound J 2019;16(4):903-908.



circaid® juxtalite®

Your order includes

- 1 inelastic adjustable compression garment
- 1 circaid Built-In-Pressure System (BPS) guide card
- 1 pair of circaid compression anklets
- 1 circaid undersleeve lower leg




circaid juxtalite

circaid juxtalite			
Styles		AD	AD
			
		short	long
Size / Article No.			
	S	JU15702000	JU15002000
	M	JU15703000	JU15003000
	L	JU15704000	JU15004000
	XL	JU15705000	JU15005000
	XXL	JU15706000	JU15006000
	M extra wide	JU15713000	JU15013000
	L extra wide	JU15714000	JU15014000
	XL extra wide	JU15715000	JU15015000

Measurement chart circaid® juxtalite®

Circumferences in cm				
	S	M	M – X	L
cC	26 – 36	33 – 44	44 – 54	41 – 51
cB	19 – 22	22 – 27	22 – 27	25 – 32
	L – X	XL	XL – X	XXL
cC	51 – 61	37 – 48	48 – 61	53 – 64
cB	25 – 32	30 – 37	30 – 37	35 – 42
	Short		Long	
Sizing measurement in cm				
floor – knee crease	< 44		> 44	

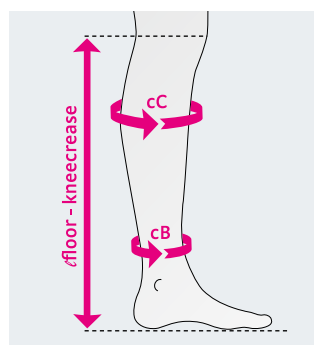



circaid® juxtalite® with circaid® compressive undersock

Your order includes

- 1 inelastic adjustable compression garment
- 1 circaid Built-In-Pressure System (BPS) guide card
- 1 pair of circaid compressive undersocks moderate




circaid juxtalite with circaid compressive undersock

NEW: circaid juxtalite with compressive undersock			
Styles		AD	AD
			
		short	long
Size / Article No.			
	S	JU1M702000	JU1M002000
	M	JU1M703000	JU1M003000
	L	JU1M704000	JU1M004000
	XL	JU1M705000	JU1M005000
	XXL	JU1M706000	JU1M006000
	M extra wide	JU1M713000	JU1M013000
	L extra wide	JU1M714000	JU1M014000
	XL extra wide	JU1M715000	JU1M015000

Measurement chart circaid® juxtalite®

Circumferences in cm				
	S	M	M – X	L
cC	26 – 36	33 – 44	44 – 54	41 – 51
cB	19 – 22	22 – 27	22 – 27	25 – 32
	L – X	XL	XL – X	XXL
cC	51 – 61	37 – 48	48 – 61	53 – 64
cB	25 – 32	30 – 37	30 – 37	35 – 42
	Short	Long		
Sizing measurement in cm				
floor – knee crease	< 44	> 44		

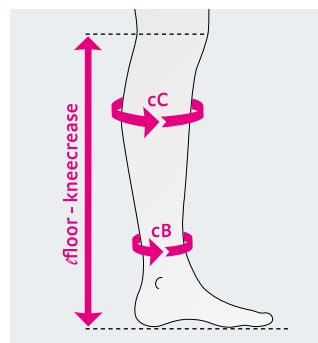

circaid® foot options

The following foot options are recommended to wear in combination with circaid juxtalite.

Note

It is recommended to always wear a circaid foot option in combination with a circaid leg garment. The indications of the circaid foot options are identical to the indications of the corresponding circaid leg garment. Please order separately.

circaid® juxtalite® ankle foot wrap

Adjustable compression device with closed heel for venous diseases.

Product details


- Provides compression to foot and ankle
- Can be trimmed in length to provide perfect fit
- Easy handling
- Available in standard sizes only

Measurement chart

Circumferences in cm		
	S	M
cA1	19 – 26	24 – 31
Article No.	JU1W002000	JU1W003000



circaid juxtalite ankle foot wrap

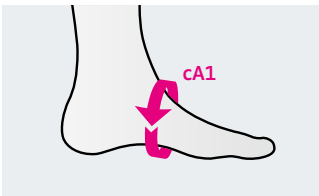

circaid® compression anklet

Standard compression option for foot and ankle.

Product details


- Provides compression to the foot and ankle
- Can be combined with other circaid foot options
- Available in standard and large

Measurement chart

Circumferences in cm		
	Standard	Large
cB	19 – 27	25 – 50
cA1	19 – 36	22 – 41
Article No.	JU51001000	JU51002000



circaid compression anklet

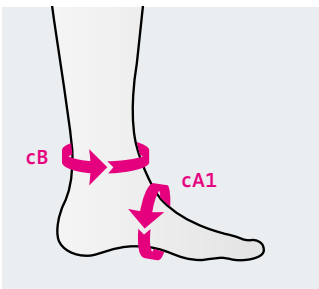

circaid® power added compression band (pac band)

Additional compression for the arch of the foot and ankle.

Product details


- Additional compression to the foot with mild to severe tendency for oedema
- Targets the arch of the foot and around the malleoli
- Can be trimmed to fit almost every foot shape
- Easy handling
- Lightweight and breathable material
- Available in standard and large
- Bilateral use (left or right foot)

Measurement chart

Circumferences in cm			
	Standard	Large	Extra Large
cB	19–27	25–50	45–56
cA1	19–36	22–41	22–41
Article No.	JU5P001000	JU5P002000	JU5P003000



circaid power added compression band (pac band)

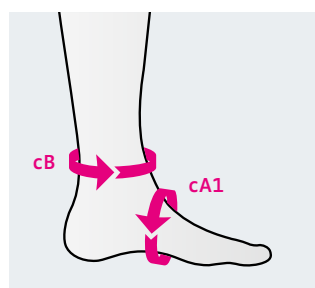

circaid® compressive undersock

Easy-to-apply liner with targeted compression for foot and ankle.

Product details


- Applies graduated compression to the foot and ankle
- Graduated compression continues into the leg portion of the undersock
- Soft material for pleasant wearing comfort
- Available in 15–25 mmHg (moderate compression) and 25–35 mmHg (firm compression)
- Available in standard and large

Measurement chart

Circumferences in cm		
	Standard	Large
cA1	19–32	24–41
cB	19–27	25–42
cC	26–54	41–64
Article No.		
Moderate	JU5M001000	JU5M002000
Firm	JU5F001000	JU5F001000



circaid compressive undersock

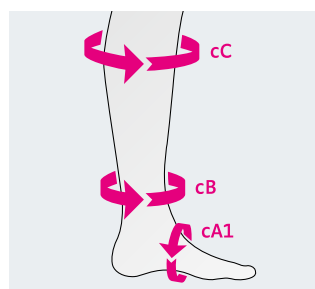





medi travel® Travel socks

People with a healthy venous system do not immediately consider the topic of thrombosis prophylaxis when travelling on holiday or on business. On long car journeys or when flying, however, this is particularly important. Travel socks are an essential part of every travel checklist. They can reduce the risk of traveller's thrombosis¹.

Integrate into your talks to customers the topic of thrombosis prophylaxis when travelling and recommend medi travel socks – as a healthy holiday companion for fit and healthy legs.

¹ Hsieh HF, Lee FP. Graduated compression stockings as prophylaxis for flight-related venous thrombosis: systematic literature review. J Adv Nurs 2005;51(1):83-98.





medi travel[®] men

The sporty travel socks for HIM.



Features

- Opaque knee-high stocking in attractive ribbed look
- Controlled, upwardly decreasing pressure distribution
- Can reduce the risk of travel thrombosis¹
- Extra-soft material with reinforced sole
- Comfortable cuff
- Clima Comfort: for the rapid removal of moisture from the skin's surface and high breathability
- For people with a healthy venous system
- Cannot be prescribed

Washing instructions

Easy-care    

Suitable for washing machine and drier

Styles		Measuring chart	
AD			
		Circumference 18 – 20 cm 20 – 23 cm 23 – 26 cm 26 – 29 cm 29 – 32 cm Ankle circumference	Size S M L XL XXL
Length standard and petite	The length is measured from the heel to below the knee.	You determine the right size of the medi travel by measuring the ankle circumference.	

Standard styles	
Foot	closed toe, shoe size 38 – 48
Series	Size S – XXL, Length standard (43 – 49 cm) and petite (37 – 42 cm)

Material composition	
Elastane (EL)	Polyamide (PA)
10%	90%

Product benefits



Clima Comfort



Perfect Fit



Hightech fibres

Highest longitudinal and transverse elasticity ensures optimum user comfort on the leg



medi travel complies with all applicable EU Directives. All the prescribed conformity assessment procedures have been completed.

Colours

S = Standard

Black
S

Anthracite
S



¹ Hsieh HF, Lee FP. Graduated compression stockings as prophylaxis for flight-related venous thrombosis: systematic literature review. J Adv Nurs 2005;51(1):83-98.

medi travel[®] women

The fashionable travel socks for HER.




Features

- Knee socks with a semi-transparent appearance
- Controlled, upwardly decreasing pressure distribution
- Can reduce the risk of travel thrombosis¹
- Comfortable cuff
- Clima Comfort: for the rapid removal of moisture from the skin's surface and high breathability
- Clima Fresh has an antibacterial effect, thereby inhibiting odour-forming bacteria
- For people with a healthy venous system
- Cannot be prescribed

Washing instructions

Easy-care

Suitable for washing machine and drier

Styles		Measuring chart	
AD			
		<p>Circumference</p> <p>18 – 20 cm</p> <p>20 – 23 cm</p> <p>23 – 26 cm</p> <p>26 – 29 cm</p> <p>29 – 32 cm</p> <p>Ankle circumference</p>	<p>Size</p> <p>S</p> <p>M</p> <p>L</p> <p>XL</p> <p>XXL</p> 
Length standard and petite	The length is measured from the heel to below the knee.	You determine the right size of the medi travel by measuring the ankle circumference.	
Standard styles			
Foot	closed toe, shoe size 36 – 43		
Series	Size S – XXL, Length standard (39 – 44 cm) and petite (34 – 38 cm)		
Material composition			
Elastane (EL)	Polyamide (PA)		
25%	75%		

Product benefits



Hightech fibres

Highest longitudinal and transverse elasticity ensures optimum user comfort on the leg



medi travel complies with all applicable EU Directives. All the prescribed conformity assessment procedures have been completed.

Colours

S = Standard



¹ Hsieh HF, Lee FP. Graduated compression stockings as prophylaxis for flight-related venous thrombosis: systematic literature review. J Adv Nurs 2005;51(1):83-98.



A photograph of a person's legs from the knees down, resting on a light-colored, textured carpet. The person is sitting on a white armchair, which is partially visible in the background. A large, brown, textured cushion is also on the carpet. A semi-transparent dark blue rectangular box is overlaid on the right side of the image, containing white text.

medi accessories

medi offers medical compression stockings for the round and flat knit sector as well as complementary products. The skin, in particular, is always eager for special care when it comes to compression therapy. Do your customers complain about itchy or dry skin? Recommend the care products from medi. These have been specially designed for vein and oedema patients and pamper the skin thanks to their precious design characteristics.

Care products for compression stockings protect the fabric and ensure a long service life for the stocking as well as caring for the skin.

Do your customers need help putting on their medical compression stockings? There are different aids for donning and doffing medical compression stockings, which make their daily use that much easier.

medi skin care

Offer your customers more, such as silky smooth skin during compression treatment.

medi's skin care products have many beneficial ingredients. They are the perfect accompaniment to venous treatment and ensure nourished, healthy skin.

Medical compression stockings are the main component of venous treatment. Proper skin care is an important part of treatment, since the garments are generally worn every day. This can stress the skin. The medi skin care products are specially designed for compression treatment and keep the skin balanced.

An important tip for patients with allergies:

All products meet the highest demands and are produced free of parabens (preservatives) and wool fat alcohols, and are latex free.

The medi care products do not have a use-by or expiry date. They are marked with an „opened tub“ and a month. This means that as soon as the tube has been opened and the contents are being used, the product should be used within 6, 9, or 12 months, as with shower gels, shampoos, etc.



medi care products are cosmetic products and may only be used on intact skin.

medi day® – cooling gel with horse chestnut

- Refreshing and cooling effect
- Well-nourished, smooth skin
- Reduces the feeling of tightness
- Supports user comfort of medicinal compression stockings

50 ml tube / 150 ml tube / 500 ml dispenser

Article	PU	Art. no.
medi day, 50 ml tube	6 pcs	P400025000
medi day, 150 ml tube	6 pcs	P400026000
medi day, 500 ml dispenser	1 pcs	P400027000
medi day, Sachet 8 ml	20 pcs	P931009000



You can find more information
about the test carried out at
www.dermatest-guarantee.de



medi day

medi night – skincare cream with regenerative Hamamelis

- To soothe and rejuvenate during the night.
- Soothes the skin and also supports regeneration

50 ml tube / 150 ml tube

Article	PU	Art. no.
medi night, 50 ml tube	6 pcs	P500025000
medi night, 150 ml tube	6 pcs	P500026000
medi day, Sachet 8 ml	20 pcs	P941009000



You can find more information
about the test carried out at
www.dermatest-guarantee.de



medi night

medi fresh – refreshing spray with menthol

- For a hint of freshness throughout the day
- Feel refreshed straight away
- Can be used several times a day
- Gentle on the stocking fabric
- Easy to use everyday

100 ml bottle

Article	PU	Art. no.
medi fresh, 100 ml bottle	6 pcs	P900026000



You can find more information
about the test carried out at
www.dermatest-guarantee.de



medi fresh

medi fabric care

The special medi hand wash is designed especially for mediven compression garments. It is suitable for daily washing and gently and effectively cleans the fabric. The medi spot cleaner removes shoe abrasion from the heel and tip of the foot. It is applied to the required areas before washing (allow to work for five minutes).

Features

- Protects the colour
- Gentle care
- Easy handling
- Odourless
- Dermatologically compatible



medi clean – hand detergent for medical compression stockings

Article	PU	Art. no.
medi clean, 200 ml	6 pcs	P920026000
medi clean, 200 ml	20 pcs	P951009000



medi clean

medi spot ex – removal stick

Article	PU	Art. no.
medi spot ex, 15 ml	6 pcs	P930005000



medi spot ex

medi laundry net

Article	PU	Art. no.
medi laundry net	15 pcs	1020585000



medi laundry net

medi

Donning and doffing aids

medi Butler

Your benefits:

- Wide selection of Butlers: the right solution for all of your customer requirements
- Increase add-on sales
- Time saving with uncomplicated donning and doffing

Benefits for your customers:

- Quick and simple donning of compression stockings
- Less effort required
- Wide choice for the most individual demands

medi Export Butler with long handles

The ideal donning aid for all compression stockings wearers who can comfortably use a donning aid for their stockings without assistance. The long handle ensures maximum stability when putting on compression stockings and pantyhose. The handle length is 32 cm. The stirrup width is 11 cm.

Article No. 6900041000




medi Export Butler

medi Butler with long handles

The handle length is 35 cm. The stirrup width is 11 cm.

Article No. 6900052000

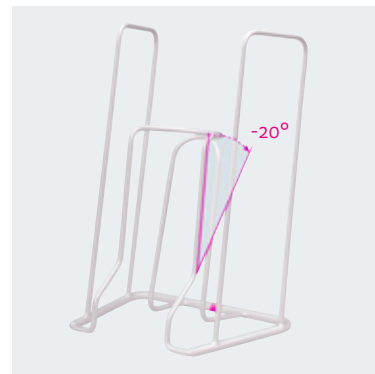


medi Butler with long handles

medi Butler with variable handles

The donning aid for stocking wearers with considerably restricted freedom of movement. The handle length can be adjusted between 37 and 58 cm. The stirrup width is 11 cm.

Article No. 6900051000

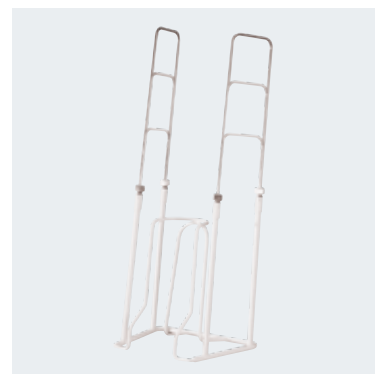


medi Butler with variable handles

medi Big (XXL) Butler

The stirrup of the medi Big Butler has a larger diameter. This makes allowance for larger circumferences which are often associated with less freedom of movement. The handle length is 44 cm. The stirrup width is 15 cm.

Article No. 6900102000

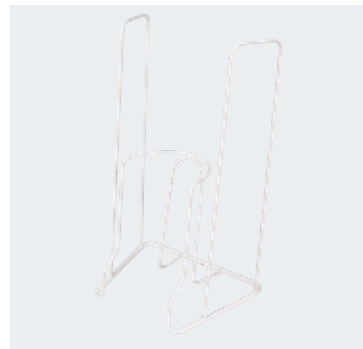


medi Big (XXL) Butler

medi Butler for pantyhose

The medi pantyhose Butler eases donning of compression pantyhose. The handle length is 48 cm. The stirrup width is 11 cm.

Article No. 6900055000

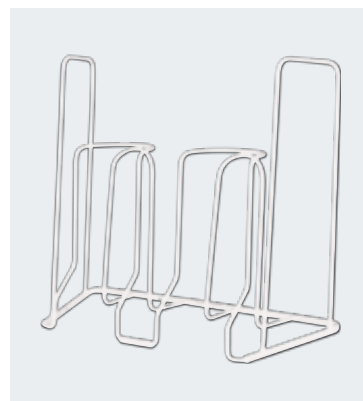

medi Butler for pantyhose

medi Hospitalbutler

Adjustable diameter with hard paint finish coating. The stirrup width is 11 cm.

Article No. 6900062000



medi Hospitalbutler

medi Arm Butler

The medi Arm Butler eases donning of compression armsleeves.

Article No. 6900057000




medi Arm Butler

medi Butler Off

Quick and easy doffing of stockings, socks and medical compression stockings, without bending over. Not recommended for mediven forte.

Article No.	Description
6900362000	medi Butler Off

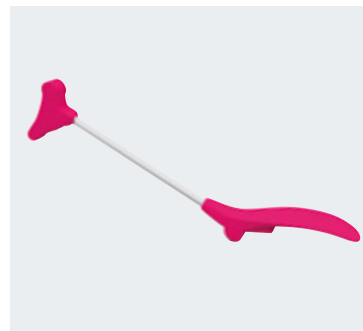

medi Butler Off

Dressing aid

Easy-on-slipper, one size fits all.

Article No.	Description
6900022000	Dressing aids (unit with 40 pieces)



Easy-on-slipper

medi 2in1

- Your compression stocking glides up over your heel effortlessly
- For all garment lengths
- For open and closed toes
- Hard-wearing and tear-resistant
- Light and folds up – very compact and portable

Article No.	Description
6900013000	medi 2in1



medi 2in1

Rubber gloves

- Sizes: small, medium, large and x-large
- Material: 100 % Latex

Article No.	Description			
6901022000	rubber gloves	S	6,5-7	1 pair
6901023000	rubber gloves	M	7,5-8	1 pair
6901024000	rubber gloves	L	8,5-9	1 pair
6901025000	rubber gloves	XL	9,5-10	1 pair
6901122000	rubber gloves	S	6,5-7	12 pairs / carton
6901123000	rubber gloves	M	7,5-8	12 pairs / carton
6901124000	rubber gloves	L	8,5-9	12 pairs / carton
6901125000	rubber gloves	XL	9,5-10	12 pairs / carton




Rubber gloves

medi textile gloves with latex

- These gloves facilitate putting on, arranging and positioning compression garments on legs and arms.
- Material: 40% polyester, 60% latex

Article No.	Description		
6903022000	medi textile gloves	small	1 pair
6903023000	medi textile gloves	medium	1 pair
6903024000	medi textile gloves	large	1 pair
6903122000	medi textile gloves	small	12 pairs / carton
6903123000	medi textile gloves	medium	12 pairs / carton
6903124000	medi textile gloves	large	12 pairs / carton

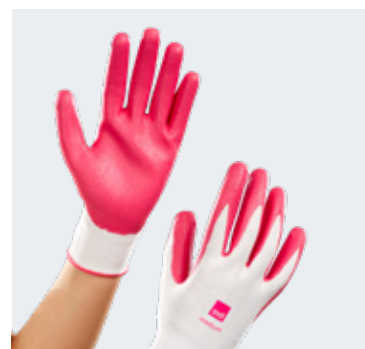

medi textile gloves

Treatment aids

medi fix

Article	PU	Art. no.
medi fix, 50 ml	6 pcs	1023025000

Adhesive lotion for compression garments with a tricot border.



You can find more information
about the test carried out at
www.dermatest-guarantee.de



medi fix







Service

On the following pages, you will find all necessary information for your orders. “medi. I feel better”. This is reflected in our service, in addition to the outstanding quality of our products: 95 percent of orders are shipped within three working days.

Your electronic path to medi starts here

medi e-shop

With the medi e-shop you can work flexibly from your tablet wherever you are – at your workplace or on the road. Round the clock. Simple and straightforward.

How do I access the medi e-shop?

Simply access the link to the medi e-shop:

<https://eshop.medi.de>

Details on using the medi e-shop:

If you have internet access, you can get started right away. Once you fill out the required fields in the registration, you will receive an email with access data, and you will then be able to log on with your user login.



Intended purposes

Round-knitted medical compression stocking:

Round-knitted medical compression stocking used for compression of the lower extremities, mainly for the treatment of disorders of the venous system.

medi Butler (leg):

Donning aids for medical/clinical compression socks and stockings.

medi Butler (arm):

Donning aid for medical/clinical compression sleeves.

medi 2in1:

Donning and doffing aids for medical/clinical compression socks and stockings.

mediven harmony:

Round-knitted medical compression stocking used for compression of the upper extremities, mainly for the treatment of disorders of the venous and lymphatic systems.

mediven harmony handpart:

Flat-knitted medical compression stocking used for compression of the upper extremities, mainly for the treatment of disorders of the venous and lymphatic systems.

mediven ulcer kit:

Circular-knit compression stocking system used for compression of the lower extremities, primarily in the treatment of disorders of the venous system.

medi travel:

Round knitted compression stocking of the lower extremities for physical thrombosis prophylaxis for travellers with healthy veins.

circaid juxtalite:

The compression system is designed to provide compression to the leg for patients with venous and lymphatic disorders.

circaid foot options:

The compression system is designed to provide compression to the foot and ankle for patients with venous and lymphatic disorders.

Notes

Notes



medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth / Germany
T +49 921 912-0
F +49 921 912-781
export@medi.de
www.medi.de/en

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61 2 9890 8696
F +61 2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16 / 7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium NV
Staatsbaan 77 / 0099
3945 Ham
Belgium
T +32 11 24 25 60
F +32 11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Canada Inc / médi Canada Inc
597, Rue Duvernay, Verchères
QC Canada J0L 2R0
T +1 450 583 3317 / +1 800 361 3153
F +1 888 583 6827
service@medicanada.ca
www.medicanada.ca

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbaek Strand
Denmark
T +45 70 25 56 10
F +45 70 25 56 20
kundeservice@sw.dk
www.meditandmark.dk

medi Bayreuth Espana SL
C / Canigo 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 932 60 04 00
F 34 932 60 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z. I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1 371-0090
F +36 1 371-0091
info@medi.hu
www.medi.hu

medi Italia s. r. l.
Via Giuseppe Ghedini, 2
40069 Zola Predosa (BO)
Italy
T +39 051 6 13 24 84
F +39 051 6 13 29 56
info@medi-italia.it
www.medi-italia.it

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Norway AS
Vestheimvegen 35
4250 Kopervik
Norway
T +47 52 84 45 00
F 47 52 84 45 09
post@medinorway.no
www.medinorway.no

medi Polska Sp. z o. o.
ul. Łabędzka 22
44-121 Gliwice
Poland
T +48 32 230 60 21
F +48 32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
Rua do Centro Cultural, no. 43
1700-106 Lisbon
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LLC
Business Center NEO GEO
Butlerova Street 17
117342 Moscow
Russia
T +7 495 374 04 56
F +7 495 374 04 56
info@medirus.ru
www.medirus.ru

medi Sweden AB
Box 6034
19206 Sollentuna
Sweden
T +46 8 969798
F +46 8 6266870
info@medi.se
www.medi.se

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
Great Britain
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Evhen Sverstiuka str, 11.
Kiev 02002
Ukraine
T +380 44 591 11 63
F +380 44 392 73 73
info@medi.ua
http://medi.ua

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1 336 4 49 44 40
F +1 888 5 70 45 54
info@mediusa.com
www.mediusa.com

medi Trading (Shanghai) Co. Ltd.
Room 1703, T1, CES WEST BUND CENTER,
No. 277, Long Lan Road
Xuhui District
200232 Shanghai
People's Republic of China
T +86 21 50582319
F +86 21 50582319

Medi Turk Ortopedi Medikal İç ve Dış Tic. AS
Mustafa Kemal Mah. 2156 Sokak No: 16 / 4 – 5
06510 Çankaya Ankara
Turkey
T +90 312 435 20 26
F +90 312 434 17 67
info@medi-turk.com
www.medi-turk.com

MAXIS a. s.
medi group company
Slezská 2127 / 13
120 00 Prague 2
Czech Republic
T +420 571 633 510
F +420 571 616 271
info@maxis-medica.com
www.maxis-medica.com

Tachezy Sanit Srl.
Via Druso 48 / a
39100 Bolzano
Italy
T +39 0471 288 068
F +39 0471 282 773
info@tachezysanit.com
www.tachezysanit.com

medi Japan K. K.
5-1 Nihonbashi Kabutocho Chuoku
103-0026 Tokyo
Japan
T +81 3 6778 2590
F +81 3 5847 7901
info@medi-japan.jp
www.medi-japan.co.jp

Definition of article numbers phlebology

Please choose your size on the last (7th) digit of the article number from 2 to 6 (travel men, travel women, duomed) or from 1 to 7 (all other qualities). Please choose open or closed toe (6th digit: 0 = open toe, 1 = closed toe) for all qualities except mediven plus, mediven for men, mediven active, medi travel men / women. Please choose your colour (5th digit) instead of x.

mediven sheer & soft

0 = Natural

5 = Ebony

mediven elegance

C = Cashmere

A = Anthracite

K = Rosé*

5 = Black

1 = Caramel

M = medi Magenta

Z = Bronze*

7 = White

0 = Beige

3 = Navy

* Please choose open or closed toe (6th digit: 7 = open toe, 8 = closed toe).

mediven for men / mediven active

3 = Navy

5 = Black

6 = Brown

7 = White (only mediven active)

A = Anthracite

mediven comfort

C = Cashmere

0 = Beige

K = Rosé

3 = Navy

1 = Caramel

A = Anthracite

Z = Bronze

5 = Black

mediven plus

C = Cashmere

3 = Navy

K = Rosé

A = Anthracite

1 = Caramel

5 = Black

Z = Bronze

M = medi Magenta

0 = Beige

7 = White

mediven forte

C = Cashmere

0 = Beige

K = Rosé

3 = Navy

1 = Caramel

Z = Anthracite

X = Bronze

5 = Black

duomed

0 = Beige

5 = Black

duomed smooth

1 = Caramel

5 = Black



Headquarters
medi GmbH & Co. KG
Medicusstraße 1
95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-781
export@medi.de
www.medi.de/en

Product Catalogue



4 049772 376698

0097067 / 02.2022